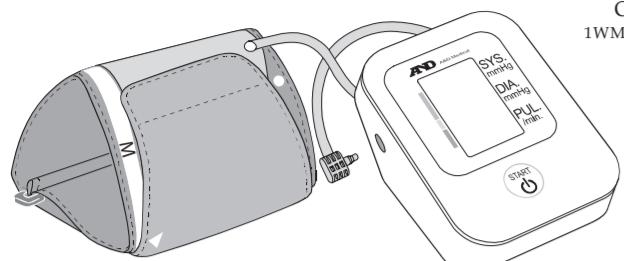
611 PLUS Blood Pressure Monitor

UA-611

 Instruction Manual
 ORIGINAL
 1WMPD4002687B

Congratulations on purchasing a state-of-the-art A&D blood pressure monitor, one of the most advanced monitors available today. Designed for ease of use and accuracy, this device will facilitate your daily blood pressure regimen. We recommend that you read through this manual carefully before using the device for the first time.

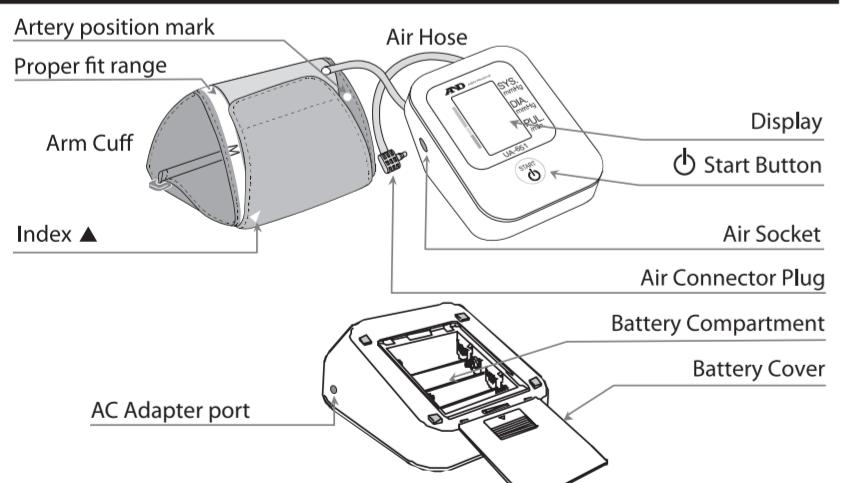
PRELIMINARY REMARKS

- This device conforms to the European Directive 93/42 EEC for Medical Products. This is made evident by the CE 0123 mark of conformity. (0123: The reference number to the involved notified body)
- The device is designed for use on adults only. Do not use on newborns or infants.
- This device is intended for home healthcare environment.
- This device is designed to measure blood pressure and pulse rate of people for diagnosis.

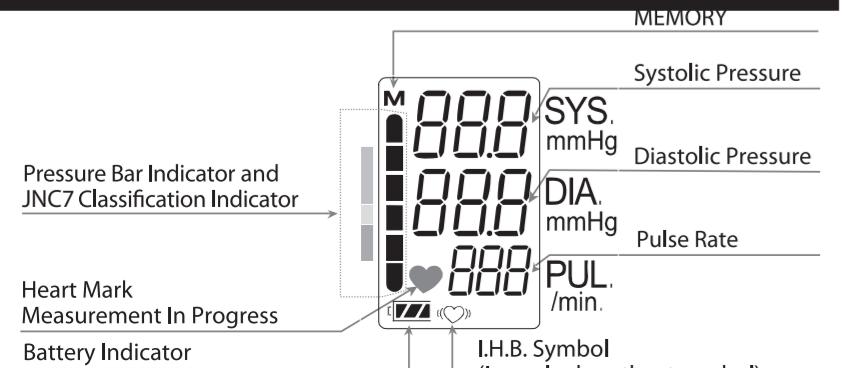
PRECAUTIONS

- Precision components are used in the construction of this device. Extremes in temperature, humidity, direct sunlight, shock or dust should be avoided.
- Clean the device and cuff with a dry, soft cloth or a cloth dampened with water and a neutral detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.
- Avoid tightly folding the cuff or storing the hose tightly twisted for long periods, as such treatment may shorten the life of the components.
- The device and cuff are not water resistant. Prevent rain, sweat and water from soiling the device and cuff.
- Measurements may be distorted if the device is used close to televisions, microwave ovens, cellular telephones, X-ray or other devices with strong electrical fields.
- When reusing the device, confirm that the device is clean.
- Used equipment, parts and batteries are not treated as ordinary household waste, and must be disposed of according to the applicable local regulations.
- Do not modify the device. It may cause accidents or damage to the device.
- To measure blood pressure, the arm must be squeezed by the cuff hard enough to temporarily stop blood flow through the artery. This may cause pain, numbness or a temporary red mark to the arm. This condition will appear especially when measurement is repeated successively. Any pain, numbness, or red marks will disappear with time.
- People who have a severe circulatory deficit in the arm must consult a doctor before using the device, to avoid medical problems.
- Do not self-diagnose the measurement results and start treatment by yourself. Always consult your doctor for evaluation of the results and treatment.
- Do not apply cuff on an arm with other medical equipment attached. Neither equipment may function properly.
- Do not apply the cuff on an arm with an unhealed wound.
- Do not apply the cuff on an arm receiving an intravenous drip or blood transfusion. It may cause injury or accidents.
- Do not use the device where flammable gases such as anesthetic gases are present. It may cause an explosion.
- Do not use the device in highly concentrated oxygen environments, such as a high-pressure oxygen chamber or an oxygen tent. It may cause a fire or explosion.
- Do not excessively bend or squeeze the air hose.

MONITOR COMPONENTS

DISPLAY

SYMBOL IDENTIFICATION

Symbols printed on the device case:

Symbol	Function / Meaning	Recommended Action
●	Standby and turn the device on	Set the ● mark on the artery of the upper arm or in line with the ring finger on the inside of the arm.
▲	Index	_____
REF	Catalog number	_____
M	Proper fit range for the medium cuff. It's printed on the cuff.	_____
—	Over range printed on the medium cuff and AccuFit™ Plus cuff.	Use the large cuff instead of the medium cuff or the AccuFit™ Plus cuff.
W	Proper fit range for the AccuFit™ Plus cuff. It's printed on the AccuFit™ Plus cuff.	_____
L	Proper fit range for the large cuff. It's printed on the large cuff and AccuFit™ Plus cuff.	_____
S	Under range printed on the medium cuff and AccuFit™ Plus cuff.	_____
M	Under range printed on the large cuff.	Use the medium cuff instead of the large cuff.
LOT	Lot number	_____

CUFF SYMBOL IDENTIFICATION

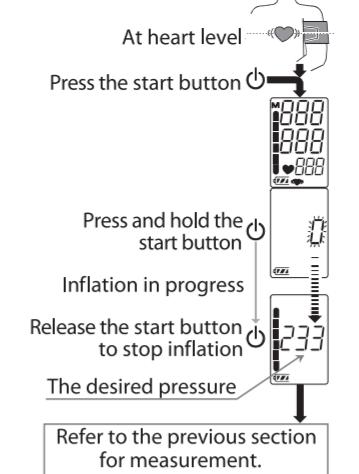
Symbols printed on the cuff:

Symbol	Function/Meaning	Recommended Action
●	Artery position mark	Set the ● mark on the artery of the upper arm or in line with the ring finger on the inside of the arm.
▲	Index	_____
REF	Catalog number	_____
M	Proper fit range for the medium cuff. It's printed on the cuff.	_____
—	Over range printed on the medium cuff and AccuFit™ Plus cuff.	Use the large cuff instead of the medium cuff or the AccuFit™ Plus cuff.
W	Proper fit range for the AccuFit™ Plus cuff. It's printed on the AccuFit™ Plus cuff.	_____
L	Proper fit range for the large cuff. It's printed on the large cuff and AccuFit™ Plus cuff.	_____
S	Under range printed on the medium cuff and AccuFit™ Plus cuff.	_____
M	Under range printed on the large cuff.	Use the medium cuff instead of the large cuff.
LOT	Lot number	_____

Measurement with a Desired Systolic Pressure

If your systolic pressure is expected to exceed 230 mmHg, use this procedure.

1. Place the cuff on the arm (preferably the left arm).
2. Press the ○ button.
3. During the zero blinks, press and hold the ○ button until a number about 30 to 40 mmHg higher than your expected systolic pressure appears.
4. When the desired number is reached, release the ○ button to start measurement. Then continue to measure your blood pressure as described in the previous section.

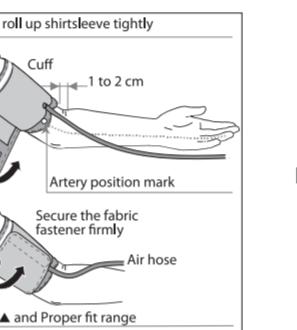
TIPS FOR TAKING READINGS

- Sit down in a comfortable position. Place the arm to be used for the measurement on a table or other support so that the center of the cuff will be at the same height as your heart.
- Relax before taking a measurement. If you are excited or depressed by emotional stress, the measurement will reflect this stress as a higher (or lower) than normal blood pressure reading and the pulse reading will usually be faster than normal.
- An individual's blood pressure varies constantly, depending on what you are doing and what you have eaten. What you drink can have a very strong and rapid effect on your blood pressure.
- This device bases its measurements on the heartbeat. If you have a very weak or irregular heartbeat, the device may have difficulty determining your blood pressure.
- Should the device detect a condition that is abnormal, it will stop the measurement and display an error symbol. Refer to the section titled Display Symbols for the description of the symbols.
- This device is intended for use by adults only. Consult with your physician before using this device on a child. A child should not use this device unattended.

APPLYING THE ARM CUFF

1. Wrap the cuff around the upper arm, about 1 to 2 cm above the inside of the elbow, as shown.
2. Constriction of the upper arm, caused by rolling up a shirt sleeve, may prevent accurate readings.
3. Confirm that the index ▲ points within the proper fit range.

HOW TO TAKE ACCURATE MEASUREMENTS

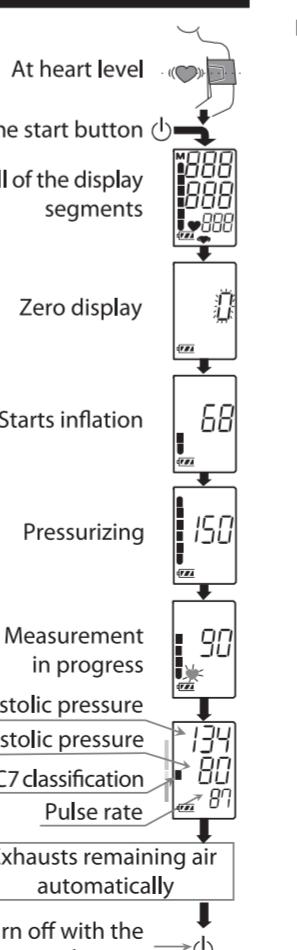
- For the most accurate blood pressure measurement:
- Sit comfortably on a chair. Rest your arm on the table. Do not cross your legs. Keep your feet on the floor and straighten your back.
 - Relax for several minutes before measurement.
 - Place the center of the cuff at the same level as your heart.
 - Remain still and keep quiet during measurement.
 - Do not measure immediately after physical exercise or a bath. Rest for twenty or thirty minutes before taking the measurement.
 - Try to measure your blood pressure at the same time every day.

DURING MEASUREMENT

- During measurement, it is normal for the cuff to feel very tight. (Do not be alarmed).
- AFTER MEASUREMENT**
- After measurement, press the START button to turn the device off. After one minute of non-operation, the device will turn off automatically.
 - Remove the cuff and record your data.

TAKING A MEASUREMENT

- Model UA-611 is designed to detect the pulse and to inflate the cuff to a systolic pressure level automatically.
- If your systolic pressure is expected to exceed 230 mmHg, read "Measurement With A Desired Systolic Pressure" below.
- Normal Measurement**
1. Place the cuff on the arm (preferably the left arm). Sit quietly during measurement.
 2. Press the ○ button. All of the display segments are displayed. Zero is displayed blinking briefly. Then the display changes, as indicated in the figure at the right, as the measurement begins. The cuff starts to inflate. It is normal for the cuff to feel very tight. A pressure bar indicator is displayed, as in the figure at the right, during inflation.
 3. When inflation is complete, deflation starts automatically and the ● (heart mark) blinks, indicating that the measurement is in progress. Once the pulse is detected, the mark blinks with each pulse beat.
 4. If an appropriate pressure is not obtained, the device starts to inflate again automatically.
 5. When the measurement is complete, the cuff exhausts the remaining air and deflates completely.
 6. Press the ○ button again to turn off the power.
- Note:** Model UA-611 is provided with an automatic power shut-off function.

TROUBLESHOOTING

Problem	Possible Reason	Recommended Action
Nothing appears on the display, even when the power is turned on.	Batteries are drained.	Replace all batteries with new ones.
Battery terminals are not in the correct position.	Reinstall the batteries with negative and positive terminals matching those indicated on the battery compartment.	
The cuff does not inflate.	Battery voltage is too low. (LOW BATTERY mark) blinks. If the batteries are drained completely, the mark does not appear.	Replace all batteries with new ones.
The cuff is not applied properly.	Apply the cuff correctly.	
You moved your arm or body during measurement.	Make sure you remain still and quiet during measurement.	
The device does not measure.	Sit comfortably and still. Place your arm on a table with your palm facing upward and the cuff at the same level as your heart.	
The cuff position is not correct.	If you have a very weak or irregular heartbeat, the device may have difficulty in determining your blood pressure.	
Other	Remove the batteries. Place them back properly and take another measurement.	

Note: If the actions described above do not solve the problem, contact the dealer. Do not attempt to open or repair this product, as any attempt to do so will make your warranty invalid.

MAINTENANCE

Do not open the device. It uses delicate electrical components and an intricate air unit that could be damaged. If you cannot fix the problem using the troubleshooting instructions, contact the authorized dealer in your area or our customer service department. A&D customer service will provide technical information, spare parts and units to authorized dealers. It is generally recommended to have the monitor inspected every 2 years to ensure proper functioning and accuracy.

SPECIFICATIONS

MODEL	UA-611
MEASUREMENT METHOD	Oscillometric measurement
MEASUREMENT RANGE	Pressure: 0 - 299 mmHg Systolic pressure: 60 - 279 mmHg Diastolic pressure: 40 - 200 mmHg Pulse: 40 - 180 beats / minute
MEASUREMENT ACCURACY	Pressure: ±3 mmHg Pulse: ±5 %
POWER SUPPLY	4 x 1.5V batteries (R6P, LR6 or AA)
NUMBER OF MEASUREMENTS	Approx. 700 times LR6 (alkaline batteries) Approx. 200 times R6P (manganese batteries) With pressure value 180 mmHg, room temperature 23 °C
CLASSIFICATION	Internally powered ME equipment (Supplied by batteries). Continuous operation mode.
CLINICAL TEST	According to ANSI / AAMI SP-10 1992
EMC	IEC 60601-1-2: 2007
MEMORY	Last 15 measurements
OPERATING CONDITIONS	50 °F to 104 °F / 15 % RH to 85 % RH +10 °C to +40 °C / 15 % RH to 85 % RH 800 to 1060hPa
TRANSPORT/STORAGE CONDITIONS	-4 °F to 140 °F / 10 % RH to 95 % RH -20 °C to +60 °C / 10 % RH to 95 % RH
DIMENSIONS	Approx. 3.8" (W) x 2.7" (H) x 5.1" (D)
WEIGHT	Approx. 9.1 oz., excluding the batteries
INGRESS PROTECTION	Device: IP21
APPLIED PART	Cuff Type BF
USEFUL LIFE	Device: 5 years (when used six times a day) Cuff: 2 years (when used six times a day)

Note: Specifications are subject to change without prior notice.
EMC table information is listed on our website:
http://www.aandd.jp/products/manual/medical/emc_en.pdf

Accessories sold separately

Part Number	Cuff Size	Arm Size
UA-420	AccuFit™ Plus Cuff	8.6"-16.5" (22-42cm)
UA-290	Medium Cuff	9.0"-14.6" (23-37cm)
UA-291	Large Cuff	12.2"-17.7" (31-45cm)

Arm size: The circumference at the biceps.

WARRANTY

This blood pressure monitor and cuff carries different warranties in different countries of purchase. Please see below for warranty length and contact information where you purchased this product.

In the United States of America:

5 year warranty
A&D Engineering, Inc.
1756 Automation Parkway
San Jose, CA 95131 U.S.A.
www.andmedical.com
1-888-726-9966

In Canada:

5 year warranty
Distributed in Canada by:
Auto Control Medical, Inc.
6695 Millcreek Drive, Unit 6
Mississauga, Ontario, L5N 5R8 Canada
www.lifesourcecanada.com
1-800-461-0991

In Latin America:

No warranty
Manufactured for:
A&D Medical
A division of A&D Engineering, Inc.
1756 Automation Parkway
San Jose, CA 95131
Toll-Free: 1-888-726-9966
www.andmedical.com

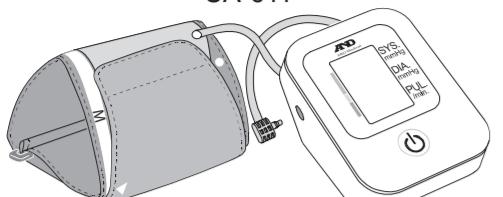
CONTACT INFORMATION

Distributed by:
A&D Engineering, Inc.
1756 Automation Parkway
San Jose, CA 95131
1-888-726-9966
www.andmedical.com

©2014 A&D Medical. All rights reserved. Specifications subject to change without notice.
I-MAN-UA-611-072016<br

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