

Quick Start Guide

1 CHARGING THE BATTERY: Plug unit into a wall outlet using the supplied power cord and USB charger. The unit will charge until the green light turns off. A full charge may take up to 4 hours. When you see the green light is off, the unit is fully charged. You are ready to use the NUWAVE®. **IMPORTANT NOTE:** If you attempt to charge the unit and you see the green light momentarily flash and then turn off, your unit is fully charged and you DO NOT NEED TO CHARGE THE UNIT. You are ready to use the NUWAVE®.

NOTE: You can operate the NUWAVE® unit with the power cord in if you chose. For detailed charging instructions, see page 7 of the enclosed instruction manual.

2 Connect your CPAP tube to the NUWAVE® sanitizing chamber. Then connect your tube to the NUWAVE® generator (see pages 3-7 of the enclosed instruction manual for details).

3 Load your CPAP mask, headgear and or the humidification chamber and fully zip close the NUWAVE® sanitizing chamber (see page 4 of the enclosed instruction manual for details).

4 OPERATING THE UNIT: Press and hold the “On” button for 2 seconds then release to cleaning cycle. You should see the blue light illuminate for the next 35 minutes. Leave the CPAP equipment in the sanitizing chamber for the next 2 hours and remove. You are ready to use your sanitized CPAP equipment.

5 OZONE FILTER: Each box contains a special blue ozone filter containing activated carbon. It is packaged in a bag within your NUWAVE® carton near the plastic flange fitting. Remove the blue filter from the bag and also remove the labels on the filter. Put filter inside the pouch of your sanitizing chamber. Replace every 6 months.

OZONE NOTE: If you smell any residual ozone on your mask, connect your CPAP tubing and mask assembly to your CPAP machine and run your unit for 30 seconds. This will flush any residual ozone out of your tubing and mask. If you sense that you are sensitive to ozone, discontinue use.



Manufactured by:
Spirit Medical
5000 Township Parkway
Saint Paul, MN 55110 USA
support@spiritmedical.net
www.spiritmedical.net
Phone: 866-578-4889

More information: www.nuwavecpap.com

© 05/2019 Spirit Medical. All rights reserved.
NUWAVE® is a trademark of Spirit Medical.

Part number: LBL-QSG-NW (Rev. C)



Manufactured by:
Spirit Medical
5000 Township Parkway
Saint Paul, MN 55110 USA
support@spiritmedical.net
www.spiritmedical.net
Phone: 866-578-4889

More information: www.nuwavecpap.com

© 05/2019 Spirit Medical. All rights reserved.
NUWAVE® is a trademark of Spirit Medical.

Part number: LBL-QSG-NW (Rev. C)

Quick Start Guide

1 CHARGING THE BATTERY: Plug unit into a wall outlet using the supplied power cord and USB charger. The unit will charge until the green light turns off. A full charge may take up to 4 hours. When you see the green light is off, the unit is fully charged. You are ready to use the NUWAVE®. **IMPORTANT NOTE:** If you attempt to charge the unit and you see the green light momentarily flash and then turn off, your unit is fully charged and you DO NOT NEED TO CHARGE THE UNIT. You are ready to use the NUWAVE®.

NOTE: You can operate the NUWAVE® unit with the power cord in if you chose. For detailed charging instructions, see page 7 of the enclosed instruction manual.

2 Connect your CPAP tube to the NUWAVE® sanitizing chamber. Then connect your tube to the NUWAVE® generator (see pages 3-7 of the enclosed instruction manual for details).

3 Load your CPAP mask, headgear and or the humidification chamber and fully zip close the NUWAVE® sanitizing chamber (see page 4 of the enclosed instruction manual for details).

4 OPERATING THE UNIT: Press and hold the “On” button for 2 seconds then release to cleaning cycle. You should see the blue light illuminate for the next 35 minutes. Leave the CPAP equipment in the sanitizing chamber for the next 2 hours and remove. You are ready to use your sanitized CPAP equipment.

5 OZONE FILTER: Each box contains a special blue ozone filter containing activated carbon. It is packaged in a bag within your NUWAVE® carton near the plastic flange fitting. Remove the blue filter from the bag and also remove the labels on the filter. Put filter inside the pouch of your sanitizing chamber. Replace every 6 months.

OZONE NOTE: If you smell any residual ozone on your mask, connect your CPAP tubing and mask assembly to your CPAP machine and run your unit for 30 seconds. This will flush any residual ozone out of your tubing and mask. If you sense that you are sensitive to ozone, discontinue use.