

# 5539 Osteo Arthritis Knee Brace



## Features

- For mild to moderate unicompartmental knee osteoarthritis.
- Single side hinge provides a 3 point pressure system to shift the joint forces from the degenerative area to a healthy compartment.
- Hinge angle can be adjusted with a simple tool.
- Lightweight frame and anti-slip soft padding.
- Buckle for easy fastening and release.
- One size. Right or Left.

## Indications

For mild to moderate unicompartmental knee osteoarthritis.

## Warning

If rash, hives, itchiness or any irritation occurs, discontinue use and consult a physician.

## Size

ONE SIZE  
RIGHT OR LEFT

## Composition

55% Nylon	14% Steel	8% Foam
7% Aluminum	7% POM	4% Polyester
3% Silicone	1% PVC	1% EVA

## Fitting Instructions

1. Sit on a stable chair, keep knee slightly bent.
2. Centre the condyle pad to the knee cap.
3. Fasten straps 1 and 2 in sequence as indicated.
4. Adjust the thumb screw so that the proximal portion of the brace comfortably meets the thigh portion of the leg. Then attach and tighten straps 3 and 4.
5. Tighten the adjustment screw with the wrench provided to apply adequate pressure to relieve knee joint pain.
6. Once satisfied with good brace fit and function, trim the straps to the correct length and reapply the Y-hook strap ends.

