### Light-Activated Composite for Core Buildups

#### ENGLISH INSTRUCTIONS FOR USE SUPERCURE®

##### Light-Activated composite for core buildups.

Store at room temperature (10°C - 22°C, 50°F - 72°F)

Store away from light! Do not freeze!

##### SUPERCURE®

REF 310140 Natural 30 x .25g REF 310142 Contrast 30 x .25g  
REF 310141 Natural 30 x .50g REF 310143 Contrast 30 x .50g

##### SUPERCURE®-Q (harder-cutting, quartz reinforced)

REF 310146 Natural 30 x .50g REF 310147 Contrast 30 x .50g

##### DO NOT USE IN THE PRESENCE OF EUGENOL!

Do not use on patients with known allergies to Bis-GMA, benzoyl peroxide or other components.

- 1) Isolate the tooth with a rubber dam or cotton rolls.
- 2) Remove all decay and existing restorative materials, including eugenol cements or sealers.
- 3) Place an adhesive bonding agent system for dentin that is compatible with light-activated composites. Follow manufacturers' specific instructions for use.
- 4) Remove cap (or sheath) and insert the SUPERCURE® tip into a Centrix C-R® syringe. Test flow by expressing a small amount of material onto a pad.
- 5) Inject a 2mm layer of material into the prep, over the surface coated by the bonding agent. Light-activate for 20 seconds, using any halogen, LED or PAC light source.
- 6) Place a second layer on top, not to exceed a thickness of 6 to 8mm. Light activate for 40 sec.
- 7) Cut and prep the core, using water-cooled diamonds or burs.
- 8) Remove used SUPERCURE® tip from syringe and dispose of properly. Sterilize or disinfect syringe gun after use.

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## SAFETY DATA SHEET (SDS)

In accordance with Hazard Communication Standard (HCS), 29CFR 1910.1200(g), 2012 revision

### SECTION I - PRODUCT / COMPANY IDENTIFICATION

Product Trade Name: SUPERCURE Core Build-Up

Synonyms: Light-Cure Composite Restorative Paste

Recommended Use: Dental Material

Part (Item) Number: 310140, 310141, 310142, 310143, 310146, 310147

Manufacturer: CENTRIX, INC. 770 River Road, Shelton, CT 06484 USA

Emergency Telephone Number: (203) 929 - 5582

Information Telephone Number: (800) 235 - 5862

### SECTION II - HAZARDS IDENTIFICATION

For potential health hazards: SEE Section 11 below.

No other significant hazards to man or the environment are reasonably foreseen.

### SECTION III - COMPOSITION / INFORMATION ON INGREDIENTS

Product Contents: Light-Cure Composite Restorative Paste

Hazardous Ingredients: None listed.

Exact % content withheld due to trade secret restrictions.

### SECTION IV - FIRST AID MEASURES

Skin: Wash with soap and water

Ingestion: Induce vomiting if large amount of material is swallowed

Inhalation: No harm anticipated from materials in quantities packaged.

Eyes: Flush with copious amounts of water

### SECTION V - FIRE-FIGHTING MEASURES

Suitable extinguishing media: Water contact acceptable

Forbidden extinguishing media: None

Special firefighting measures: None in quantities packaged

Unusual fire and explosion hazards: None

Special protection equipment: None

### SECTION VI - ACCIDENTAL RELEASE MEASURES

Personal precautions: Avoid contact with skin and eyes.

Wash hands with soap and water if contact occurs

Environmental precautions: Dispose of in accordance to all federal, state & local regulations

Reclaiming methods: Collect using mechanical means and solid absorbents

### SECTION VII - HANDLING AND STORAGE

Handling: Not applicable

Storage: Store at temperatures not exceeding 75°F (24°C)

Containers: Not applicable

Incompatibilities: Not applicable

Other precautions: Avoid exposure to moisture and light

### SECTION VIII - EXPOSURE CONTROLS / PERSONAL PROTECTION

General measures: Not required

Limits: Not determined

Respiratory protection: Not required

Eye protection: Safety glasses or goggles

Skin protection: Not required under normal handling

Hands protection: Avoid prolonged contact skin: wash with soap and water

Other equipment:

### SECTION IX - PHYSICAL AND CHEMICAL PROPERTIES

Appearance: Tooth shade paste Explosion limits: Not Applicable

Odor: Negligible Water solubility: Virtually insoluble

Boiling point: Not Applicable Evaporation rate: Not Applicable

Flash point: Not Applicable

Other Data: Not available or none.

### SECTION X - STABILITY AND REACTIVITY

Stability: Generally stable

Conditions to avoid: Avoid Exposure to elevated temperatures and light

Materials to avoid: Not Applicable

Hazardous decomposition products: Not Applicable

Hazardous polymerization products: May occur if exposed to light

### SECTION XI - TOXICOLOGICAL INFORMATION

Eye contact: May cause irritation if not removed promptly

Skin contact: Possibility of skin irritation if not washed off promptly, or upon prolonged or repeated exposure

Prolonged exposure: Possibility skin sensitization or irritation upon prolonged or repeated exposure

Ingestion: Small quantities are not considered harmful

Inhalation: None

### SECTION XII - ECOLOGICAL INFORMATION

No ecological damage or impact anticipated upon exposure to environment in quantities as packaged.

### SECTION XIII - DISPOSAL CONSIDERATIONS

Do not dispose into sink. Dispose in accordance with all federal, state and local regulations.

### SECTION XIV - TRANSPORT INFORMATION

Avoid elevated temperatures and high intensity light

### SECTION XV - REGULATORY INFORMATION

U.S. FDA Dental Device: Class II product

EC Medical Device Directive: Class IIa product

### SECTION XVI - OTHER INFORMATION

Product handling exclusively by dental specialists per instructions for use.

### CAUTION

The above information is based on presently available data and to our best knowledge for handling the product under normal conditions. Any use of this product in any way not indicated on this document or using it together with any other process/procedure will be exclusively under the user's responsibility.

Products covered by one or more of the following patents:

US 4,963,093; US 5,122,057; and US D353673.

Other U.S and International patents pending or granted.

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### EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex Street  
Sydney, NSW 2000, Australia