

# Pediturn Soft & Hard

Manual



## Illustrations

Illustration 1



Illustration 2



## Articles

Article no.	Description	Size (mm)
IM99836GT	Pediturn turnplate, soft, for floor use	Ø400
IM99700	Pediturn turnplate, hard, for floor use	Ø380

## General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, read the manual carefully.

The person described as "the user" in this user manual is the person who is lying or sitting on the product. The assistant or assistants are the people who manoeuvre the product.



**This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the user or assistant.**

The products comply with the standards applicable for Class 1 products in the European Council Directive MDD 93/42/EEC on medical devices.

**Lot no/Batch no:**

x	x	x	x	x	x	x	x	x	x
Production year					Running number				

At Etac we strive to improve our products all the time and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or combinations not predetermined by Etac, the Etac CE certification and Etac warranty will be voided. If in doubt, please contact Etac.

**Warranty:** Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

For further information on the Etac transfer range, see [www.etac.com](http://www.etac.com).

The product can be scrapped in accordance with national regulations.

## Symbols



Washing



Handwash



Wipe off



Tumbledry



Keep out of the sun



Iron



Do not dryclean



Do not bleach



Risk of sliding down



Never leave on the floor



Warning



Read the User Manual



Batch no./Lot no.



User mass limit = maximum rated load



The product can be scrapped in accordance with national regulations



Manufacturer



Class 1 - European Council Directive MDD 93/42/EEC on medical devices

## Intended use

Pediturn is a slim, lightweight turnplate for floor use in connection with manual handling in the standing or seated position, where the user must rotate but is not

capable of moving his/her legs without assistance.

## Intended environment

Acute care, Long-term care, Home care

## Practical handling

### Positioning

Place Pediturn on the floor. Position the users feet on the middle of Pediturn so that the users legs are perpendicular to the device.

### Use

When manually handling a user in the seated position, the users legs will now follow the user, thus avoiding strain on the users foot, knee and hip joints (Illustration 2).

When manually handling a user standing, it is crucial that the user can stand/bear his/her own weight. The assistant guides the user to the standing position and ensures that the user holds his/her balance. Ideally, the assistant places one foot on the middle of Pediturn to help the user turn into the correct position. Then the user can slowly sit down again (Illustration 1).

It will often be advantageous to use a Support Belt, Sling or SmartMove with Pediturn.

## Combinations

Article no.	Description	Size (mm)
IM401	Butterfly, glide board	W320 x L660
IM403	3B Board	W225 x L765
IM404	E-Board	W330 x L600 x H5
IM405	E-Board	W330 x L750 x H6
IM408	E-Board	W250 x L600 x H5
IM410	E-Board	W200 x L450 x H5
IM409	Dolphin toilet glide board	L690
IM426	SmartMove, handles	W210 x L600
IM428	Sling, 2x4 catching loops	W160 x L1900
IM428K	Sling, 2x4 catching loops	W160 x L1200
IM460PU	OneManSling	W80 x L2700
IM8017015	SupportBelt, 11 handles, X-large	W150 x L1700
IM8015013	SupportBelt, 8 handles, large	W130 x L1500
IM8013013	SupportBelt, 6 handles, medium	W130 x L1300
IM8010513	SupportBelt, 5 handles, small	W130 x L1050
IM8010509	SupportBelt, X-small	W90 x L1050
IM8009009	SupportBelt, XX-small	W90 x L900

# Notice

Special features
------------------

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.






Read these instructions carefully.

It is important that assistants receive instruction in manual handling. Etac offers advice and training for assistants. For further information, contact Etac.

Always use the correct manual handling techniques

Encourage the client to assist where possible.

To ensure that the client feels safe and that every manual handling is smooth, always plan the manual handling in advance.

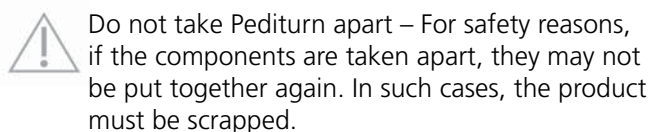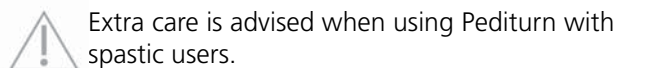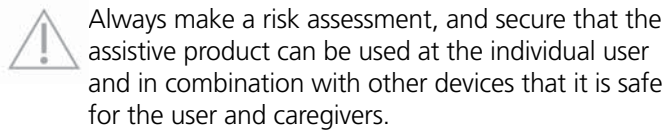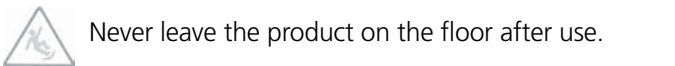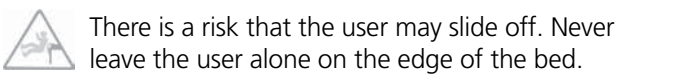

If in any doubt - please contact Etac.

# Materials


Top side: Toughtek  
Inner surface: 100% nylon.  
Backside: Toughtek  
Connector disc: synthetic.

Top side: ABS/TPU  
Inner surface: HDPE  
Backside: ABS/TPU  
Connection: Brass and steel

Cleaning

The product should be checked regularly, preferably each time it is used, and especially after wash.

Check that there is no damage to seams or fabric.

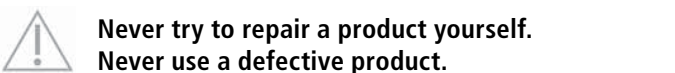

**Washing instructions:**

**Washing instructions:**  
Can be wiped with a damp cloth.



# Service Information

**Thank you for choosing an Etac product.** If the customer makes adjustments, repairs or combinations not predetermined by Etac, the Etac CE certification and Etac warranty will be voided.

**Warranty:** Two-year warranty on material and manufacturing defects, provided that the product is used correctly. Before taking a product in service always make a risk assesment according to national and/or local legislation.

## First inspection:

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description
- Check that Short Instruction is enclosed – updated User Manual/Instruction for use can be downloaded from [www.etac.com](http://www.etac.com) or contact your Etac Customer Service or the local distributor.
- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

## Periodic Inspection:

- Make sure that your Manual transfer system always is in a perfect condition.
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.
- Lot no/Batch no:

x	x	x	x	x	x	x	x	x	x
Production year		Running number							



### Visual inspection/Check the product:

Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.



### Stability test:

Try to bend the product verify that it feels firm, stable and solid.



### Mechanical load or stress/Test handles:

Draw hard in the handles in opposite directions and control the material, seams and stitching.  
Test buckles:  
Lock the buckle and draw in opposite directions, control the buckle and the stitching.



### Test low/high friction:

Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the product.  
Low friction – slides effortlessly  
High friction – no sliding/moving

## Check always:

<p>Manual transfer aids with handles, straps and buckles:</p>	<p>Manual transfer aids for sitting and lying transfer:</p>
<p>Manual transfer aids with high or low friction:</p>	<p>The product can be scrapped in accordance with national regulations</p> <p>Read the User Manual</p>

If in doubt, please contact Etac Customer Service or local distributor for more information and guidance.  
For more information about training and the use of these products, contact the Etac transfer experts – [www.etac.com](http://www.etac.com).



"Everyone should be able to live  
a free and independent life and  
pursue their dreams regardless  
of any physical circumstances."



Imported by **Novis Healthcare PTY LTD** ABN 45 102 735 491  
Unit 12/12 Mars Road Lane Cove NSW 2066 Australia  
t 1300 738 885 f 1300 738 886 e sales@novis.com.au  
[www.novis.com.au](http://www.novis.com.au)

bringing you the world in health