


# **BASIC OPERATION INSTRUCTIONS**

---

## **Front Riggings**

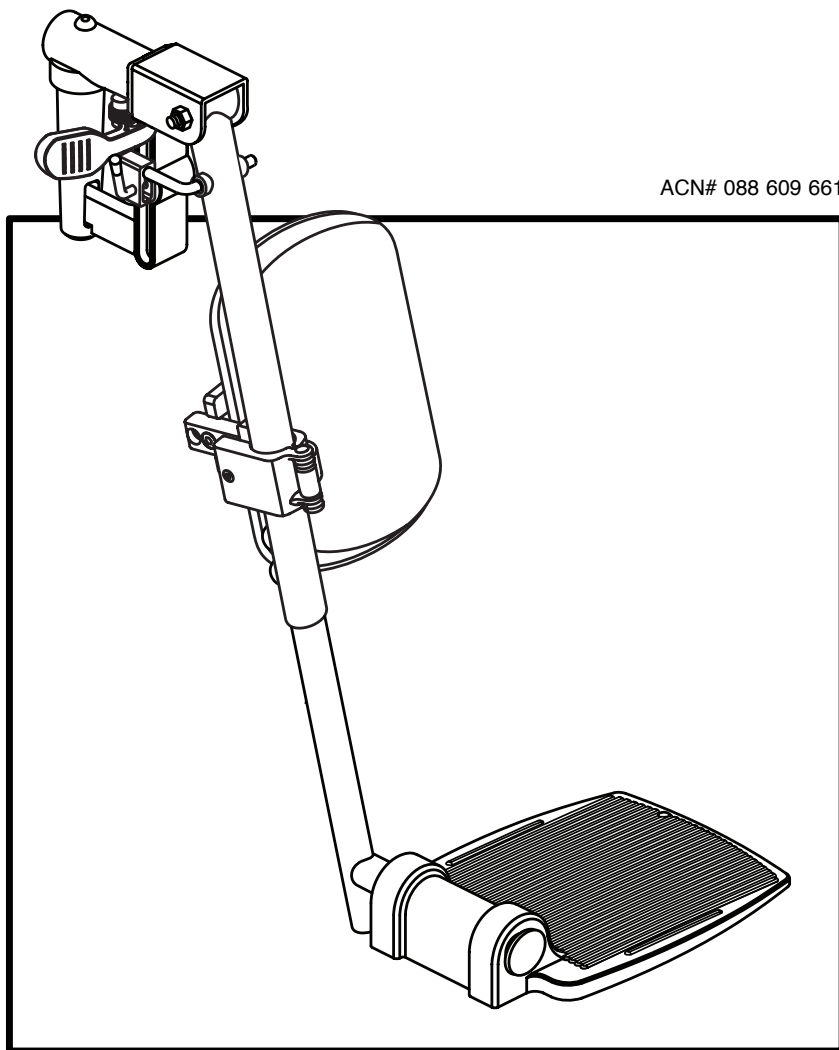

**Pride** **QUANTUM<sup>®</sup>**  
*Mobility Products Corp.* **REHAB**

---

A Division of Pride Mobility Products<sup>®</sup> Corp.

# SAFETY GUIDELINES



**WARNING! An authorized Pride Provider or a qualified technician must perform the initial setup of this product and must perform all of the instructions in this manual.**

The symbols below are used throughout this owner's manual and on the power chair to identify warnings and important information. It is very important for you to read them and understand them completely.



**WARNING! Indicates a potentially hazardous condition/ situation. Failure to follow designated procedures can cause either personal injury, component damage, or malfunction. On the product, this icon is represented as a black symbol on a yellow triangle with a black border.**



**MANDATORY! These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol on a blue dot with a white border.**



**PROHIBITED! These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a black symbol with a red circle and a red slash.**

***NOTE: These instructions are compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. The latest/current version of this manual is available on our website.***



Copyright © 2008

Pride Mobility Products Corporation

INFMANU3613/Rev D/October 2008

# LABEL INFORMATION

## PRODUCT SAFETY SYMBOLS

The symbols below are used on the product to identify warnings, mandatory actions, and prohibited actions. It is very important for you to read and understand them completely.



**Read and follow the information in the owner's manual.**



**Avoid exposure to rain, snow, ice, salt, or standing water whenever possible. Maintain and store in a clean and dry condition.**



**EMI-RFI - This product has been tested and passed at an immunity level of 20 V/m.**

### INTRODUCTION

WELCOME to Pride Mobility Products Corporation (Pride). The product you have purchased combines state-of-the-art components with **safety**, comfort, and styling in mind. We are confident that the design features will provide you with the conveniences you expect during your daily activities. Understanding how to **safely** operate and care for this product should bring you years of trouble free operations and service.

**Read and follow** all instructions, warnings, and notes in this manual before attempting to operate your product for the first time. You must also read all instructions, warnings, and notes contained in any supplemental instructional booklets for the controller, cushion system, and/or seating system that accompanied your power chair before initial operation. Your **safety** depends upon you, as well as your provider, caretaker, or healthcare professional in using good judgement.

This manual is to be used in addition to the power base and seating manuals that came with your power chair. If there is any information in this manual which you do not understand, or if you require additional assistance for setup or operation, please contact your authorized Pride Provider. **Failure to follow the instructions, warnings, and notes in this manual and those located on your Pride product can result in personal injury and/or product damage and will void Pride's product warranty.**

***NOTE: If you ever lose or misplace your product registration card or your copy of this manual, contact us and we will be glad to send you a new one immediately.***

## **ELEVATING LEG RESTS**

Elevating Leg Rests (ELRs) offer an infinite range of adjustment for the leg angle.

### **To rotate the ELRs:**

1. Push in release lever A.
2. Rotate the ELRs.

### **To adjust the angle on standard ELRs:**


1. Push down release lever B. See figure 6.
2. Move the leg rest to the desired angle.

### **To adjust the angle on international style #14 ELRs:**

1. Pull up on release lever B. See figure 7.
2. Move the leg rest to the desired angle.
3. Push down on release lever B.

### **To adjust the ELR length:**

1. Remove the two adjustment screws from the side of each leg rest extension.  
See figure 6 or 7.
2. Slide the leg rest up or down to the desired length.
3. Reinstall the two adjustment screws.

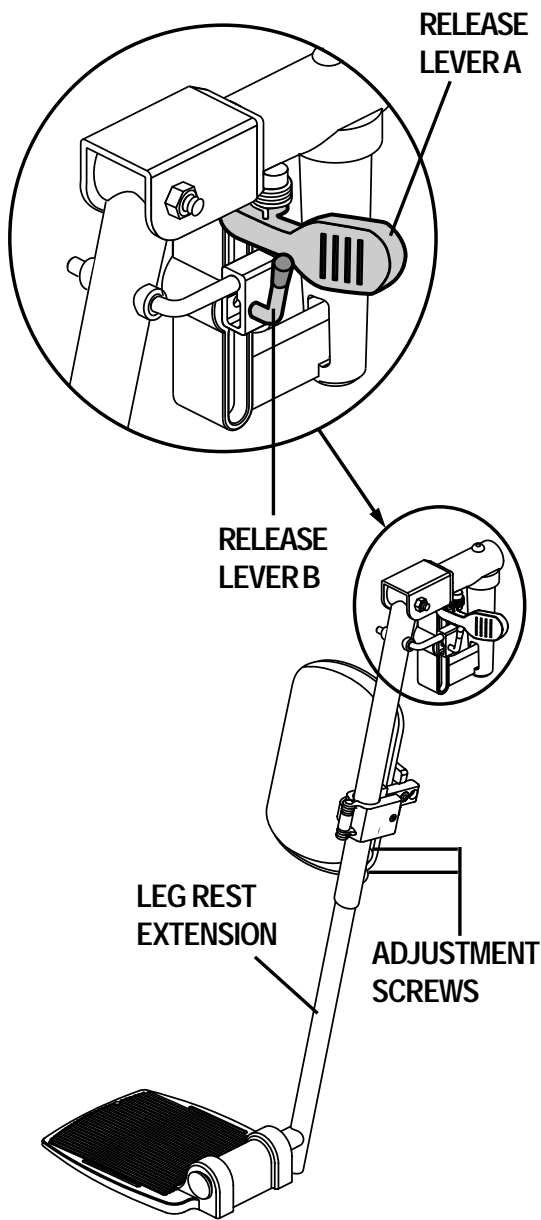

## **MULTI-AXIS FOOT PLATES**

Multi-axis foot plates can be installed on a standard elevating leg rest (ELR) or a swing-away footrest. The multi-axis foot plates have four adjustments: leg rest length (A), position (B), tilt (C), and angle (D). See figures 8 and 9.

### **To change leg rest length (A):**

1. Remove (loosen on swing-aways) the hardware.
2. Move the leg rest to the desired position.
3. Reinstall (tighten on swing-aways) the hardware.

### **To change foot plate position (B):**


1. Remove (loosen on swing-aways) the hardware.
2. Move the foot plate to the desired position.
3. Reinstall (tighten on swing-aways) the hardware.

### **To change foot plate tilt and angle on swing-aways (C):**

1. Loosen the hardware.
2. Tilt (rotate on swing-aways) the foot plate to the desired position.
3. Tighten the hardware.

### **To change foot plate angle on ELRs (D):**

1. Turn the setscrew(s) clockwise to decrease the angle.
2. Turn the setscrew(s) counterclockwise to increase the angle.

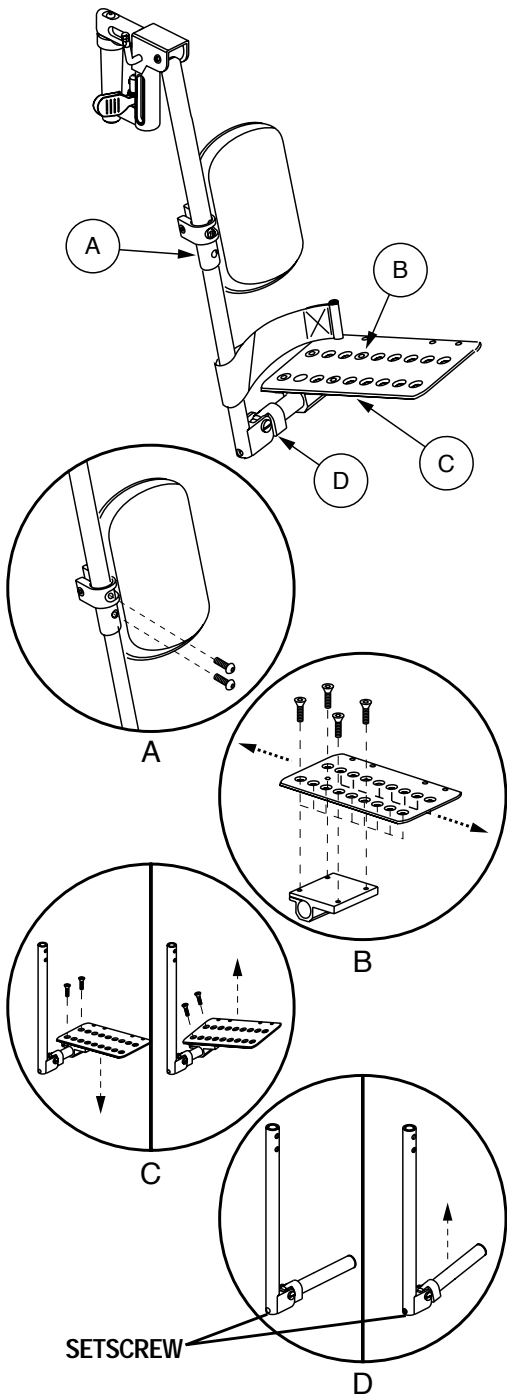

# **QUANTUM<sup>®</sup>** **REHAB**

---

A Division of Pride Mobility Products<sup>®</sup> Corp.

**Pride**  
Mobility Products Corp.