## ENGLISH

### FIGURE 1

- A. Ankle Cuff
- B. Ankle Cuff Buckle
- C. Ankle Clamp
- D. Nut and Bolt
- E. Shoe Inlay
- F. Connection Strap
- G. Connection Strap Keyhole
- H. Optional Foot Wrap

### INDICATIONS FOR USE

Drop foot

### PRECAUTIONS

- Not intended for use by patients with spasticity.
- Never wear the brace over an open wound.
- Users with sensitive skin or those at risk of skin ulcerations and infections, such as in the case of diabetes or peripheral vascular disease, are advised to consult with their practitioner before use of this product.
- Do not over-tighten brace as risk loss of blood flow.
- If the user experiences pain, swelling, redness, change or shows signs of insufficient blood flow (turning blue, white, or cold) while using the brace, advised to discontinue use immediately and consult practitioner.

### WARNING

- Over time the connection strap will lose elasticity, rendering the product less effective at providing dynamic support. The user is advised to replace the strap at regular intervals dependent on their level of activity, and support required.
- The brace should be applied against the tight shoe socks over the Foot Wrap, as this may result in diminished blood flow to/from the foot causing injury.
- For users with lower-leg width in the lowest range, the ankle clamp may not provide an optimal level of hold when donning the device.

### INITIAL APPLICATION AND ADJUSTMENT

#### Donning Ankle Cuff

- Measure ankle circumference with ankle cuff [Figure 2].
- Cut ankle cuff to size [Figure 3].
- Secure the ankle cuff [Figure 4].

#### Nut and Bolt [Figure 5]

- Ensure a secure and tight fit between nut and bolt.
- Bolt can be screwed with a coin if required.
- Nut should be checked periodically and tightened if required.

### WASHING & CARE

### LIST OF MATERIALS

Nylon, polyethylene, polypropylene, cotton, polyurethane, polyester, silicone (printing).

### WARRANTY

Ossur provides a limited 6-month warranty.

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