AUVON TENS and Muscle Stimulator

AS8012

Pain Relief, Anywhere, Anytime

Please read the product introduction carefully before
using the product

EC REP European Representative

MedNet EC-REP GmbH

Borkstrasse 10 • 48163 Muenster • Germany

Manufacturer

Shenzhen Astec Technology Co., Ltd

8E Xin Bao Yi Industrial Bld Hou ting Village,

Bei Ting Rd,Shajing,BaoAn, ShenZhen

Guangdong 518104 China.

AUVON TENS and Muscle Stimulator

AS8012

Pain Relief, Anywhere, Anytime

Please read the product introduction carefully before
using the product

EC REP European Representative

MedNet EC-REP GmbH

Borkstrasse 10 • 48163 Muenster • Germany

Manufacturer

Shenzhen Astec Technology Co., Ltd

8E Xin Bao Yi Industrial Bld Hou ting Village,

Bei Ting Rd,Shajing,BaoAn, ShenZhen

Guangdong 518104 China.

TABLE OF CONTENTS

1. INTRODUCTION.....	2
FOREWORD.....	2
HOW DOES TENS WORK.....	2
SYMBOLS.....	3
STANDARD PACKAGE.....	6
CAUTIONS.....	7
2. RECOMMENDED MASSAGE POINTS.....	11
3. ILLUSTRATIONS OF THE DEVICE AND ACCESSORIES.....	12
OPERATING INSTRUCTIONS.....	17
FEATURES.....	18
4. PRODUCT TECHNICAL PARAMETERS.....	19
5. PLACEMENT OF ELECTRODES.....	22
TENS PADS HOLDER.....	23
6. TROUBLE SHOOTING.....	25
7. MAINTENANCE AND STORAGE.....	28
ELECTROMAGNETIC COMPATIBILITY.....	29
CAUTION.....	29
8. DISPOSAL OF MATERIALS.....	34
9. CUSTOMER SERVICE.....	34

Thank you for purchasing AUVON Tens (Transcutaneous Electrical Nerve Stimulation) Stimulator. To enable you properly operate this product, please thoroughly read this product introduction first and keep the manual well after reading for future reference. Should users still have problems, please contact us via support@iauvon.com.

1. INTRODUCTION

UK

FOREWORD

AS8012 is a two output channels TENS machine and highly effective in relieving pain, different program suits to different parts of the body and pain. TENS stands for Transcutaneous Electrical Nerve Stimulation is recognized as a safe and highly effective method of relieving pain. TENS is now regularly recommended by doctors, physiotherapists and pharmacists throughout the world. Before using, please read all the instructions in this user manual carefully and keep it safe for future reference. Due to its portable appearance, the use of the device will be no longer bound by time or space. It is really a good partner for health care in daily life.

HOW DOES TENS WORK

Scientific theory suggests that electrical stimulation therapy may work in several ways.

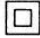
- The gentle electrical pulses move through the skin to nearby nerves to block or shut out the pain message from ever reaching the brain from the source of the pain.
- The gentle electrical pulses increase the production of the body's natural pain killer.
- Furthermore, it is thought that the electrical stimulation improves blood circulation as well. Muscles contract and relax with the flow of the electrical stimulation. With repeated contracting and relaxing, the blood flows in and out and the blood circulation is improved.

Functions (Intended for use) Intended uses include the following:

- Temporary relief of pain associated with sore
- Aching muscles in the shoulder, waist, back, neck, upper extremities (arm)
- Lower extremities (leg) due to strain from exercise

- Lower extremities (leg) due to normal household work activities

SYMBOLS

Symbol	Meaning
	Attention, consult accompanying documents
	BF Isolated Applied Part
	Class II equipment
	Disposal in accordance with Directive 2002/96/EC
	MANUFACTURER
	Date when the medical device was manufactured.
	AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY
	Pls read the user manual before use
	CE Mark, meets compliance of applicable directives and regulation in EU

Symbol	Meaning
IP22	<p>IIIP Classification: The first number 2: Protected against access to hazardous parts with a finger, and the jointed test finger of 12 mm Φ, 80 mm length, shall have adequate clearance from hazardous parts, and protected against solid foreign objects of 12.5 mm Φ and greater. The second number: Protected against vertically falling water drops when enclosure is tilted up to 15°. Vertically falling drops shall have no harmful effects when the enclosure is tilted at any angle up to 15° on either side of the vertical.</p>

SAFETY PRECAUTIONS

1. Never apply the pads to your skin with the power on, which will result in sudden shock. During massaging, if you want to move the pads to another body part, please shut down the device first, and then move the pads to the place that you want to massage.

2. Never use this product in concurrence with the following devices: pacemakers or any other embedded electronic medical devices, heart-lung machine and any other life keeping electronic medical devices, electrocardiograph and any other medical screening and monitoring devices. Simultaneous use of AUVON TENS and any of the above devices will cause malfunction and can be very dangerous to the users.

3. Never use this product in concurrence with high frequent surgical equipments, it may result in burns at the site of the stimulator electrodes and possible damage to stimulator.

4. Never use this product near microwave oven, or other high frequent equipments.

5. Please do not operate the products in close proximity (e.g. 1m) to shortwave therapy equipment, it may produce instability in stimulator output.

6. Please do not operate electrodes near the thorax which may increase the risk of cardiac fibrillation.

7. Please consult a physician regarding the use of the product under the following circumstances: patients with heart disease, baryodynia disease, acute conditions, malignant tumors, tuberculosis, any contagious disease, high fever, abnormal blood pressure, skin disease, lack of sensation on the skin, wounds on skin; pregnant women, women in menstrual cycle, or puerpera; the one who has wet body after shower or sweat; and patients who are receiving treatment. Use of the device under those circumstances may cause discomfort, accidents and may worsen your condition.

8. Please do not use the products near the heart, it may increase the risk of cardiac fibrillation.

9. Please do not use the products in bathroom or a moisture environment. Do not apply it while bathing.

10. Please do not use the products in the condition of air mixed with flammable anaesthetic gas or oxygen or nitrous oxide mixed with flammable anaesthetic gas.

11. Do not disassemble, repair or modify without authorization, you will void any warranty on the product.

12. This product cannot be repaired until the repairing technical personnel is designated by the manufacturer. The designated repairing technical personnel can require the manufacturer to supply the technical data (e.g. circuit diagram, PCB drawing, component list etc.).

13. Method for disposing the expired equipment and accessories: Do not mix the expired equipment and

accessories with the household rubbish, the rubbish should be disposed in accordance with local regulations (Example: put the rubbish into electronics recycling station)

14. Two pads should be used together, and peel off the protective film on the pads before using. Do not connect two pads in the process of using to avoid short cut.

15. Please do not apply pads on same position for over 30 minutes every time.

16. Children and persons with communication disabilities are forbidden to use the device.

17. Do not use other equipments when use the device.

18. Do the disposal of waste device regarding your local government instruction.

19. Keep the device out of pets, pests or children reach. They will damage the device to cause incorrect output performance with uncomfortable stimulation.

20. The device may be degraded by lint, dust, light (including sunlight) to cause not correct output performance with uncomfortable stimulation.

STANDARD PACKAGE

- AUVON Dual Channel TENS and Muscle stimulator
- Pin Pads
- Pads Holder
- Pouch
- Set of Lead Wires (½ Wire)
- USB Cable
- User Manual
- Happy Card

1. Pads can be used for 30-40 times at home, it depends on different situations. In hospital, they can only be used for not more than 10 times. When viscous force is not enough,

you can wipe the sticky side of pads with water by hands (do not wash them with water directly, oil and soap). When viscous force cannot regain, please exchange them.

2. All accessories, except for host machine, including USB cables, pads, output cables, charges can only be changed by special person. Please exchange them by retails.

CAUTIONS

The intensity of the stimulation is felt differently by each individual and also will vary with physical conditions. Therefore, adjust the strength level to the position you feel most comfortable. Do not assume that by increasing the intensity of the stimulation there will be a stronger effect. However, it is possible that excessive stimulation to the skin will result in irritation or red eruptions.

This product needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided, and this unit can be affected by portable and mobile RF communications equipment.

1. Do not apply the massage pads near the heart, on the head, above the neck, in the pubic region, over scarred areas, on the throat or over the mouth.
2. Avoid touching the pads when the unit is on.
3. Pregnant women must avoid using this massager.
4. Do not use this Stimulator if you a pacemaker or other life support equipment such as an artificial lung or respirator.
5. Should not be used while driving, operating machinery, or during any activity in which involuntary muscle contractions may put the user at undue risk or injury.
6. Caution should be used for patients with suspected or diagnosed epilepsy.

7. Caution should be used in presence of the following.
 - When there is a tendency to haemorrhage following acute trauma or fracture
 - Following recent surgical procedures when muscle contraction may disrupt the healing process
 - Over the menstruating or pregnant uterus
 - Over areas of the skin which lack normal sensation
8. Do not use if you have following medical conditions:
 - Patients with cardiac problems
 - Patients with carotid sinus. Tens may cause bradycardia
 - Directly over any wound
 - Over the uterus of a pregnant woman
 - Unless for labour pain
 - Over the eye
 - Patients with pacemakers
 - Over poorly enervated areas
 - Over temple
 - On or near the trigeminal nerve if you have a history of herpes zoster induced trigeminal neuralgia

(Postherpetic neuralgia)

Should ask the doctor about this device before using it if the patient:

- Suffer from acute diseases
- Have a tumour
- Have an infectious disease
- Suffer from fever
- Have blood pressure troubles
- Have skin diseases
- Recently had an accident
- Suffer from nausea or dizziness
- Experience the onset of a disease
- Notice any irregularities
- Have pains of unknown reasons

- Suffer from diabetes
- Suffer from convulsive disorders
- Have your period
- Have metal in patient's body

9. Keep out of the reach of children.

10. It is contraindicated for use on any muscle that is injured or diseased.

11. Powered muscle stimulators should be used only with the leads and electrodes recommended for use by the manufacturer.

12. Electrode placement and stimulation setting should be based on the guidance of the prescribing practitioner.

13. Some patients may experience skin irritation or hypersensitivity due to the electrical stimulation or the conductive medium. The irritation can usually be reduced by using an alternative conductive medium or alternative electrode placement.

14. Please consult a professional to replace the battery

15. No modification of this stimulator is allowed.

16. That stimulation should not be applied directly on the eyes, covering the mouth, (especially the carotid sinus), or from electrodes placed on the chest and the upper back.

17. **WARNING: IT CAN BE UNSAFE TO**

- Use of accessories, detachable parts, and materials not described in the instructions for use
- Modification of the equipment
- Use of the ME EQUIPMENT outside its carrying case when some part of the protection required by this standard is provided by that carrying case

18. Do not use this product with an electrocardiograph meter (ECG) or any other medical apparatus.

19. Do not use this product with any topically applied creams or ointments.

20. The device should not be used while the user is

connected to high-frequency surgical equipment, it may cause burn injuries on the skin under the gel electrodes, as well as problems with the device.

21. Do not use in the vicinity of shortwave or microwave therapy equipment, since this may affect the output of the device.

22. Do not use on a wet body.

23. Do not use in the bathroom or in any other area of high humidity as this may cause an uncomfortable intense stimulation.

24. Do not use while driving or operating any other mechanical equipment.

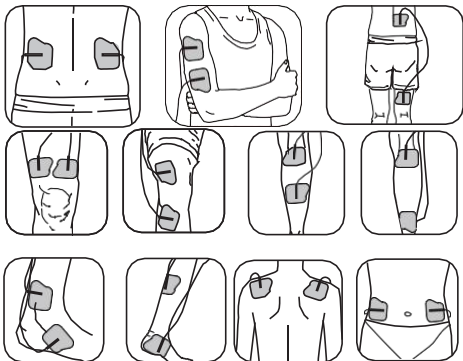
25. If the electrode was not intact, do not use the device. Degraded sensor and electrodes, or loosened electrodes may cause output performance with potential risk.

Additional precaution:

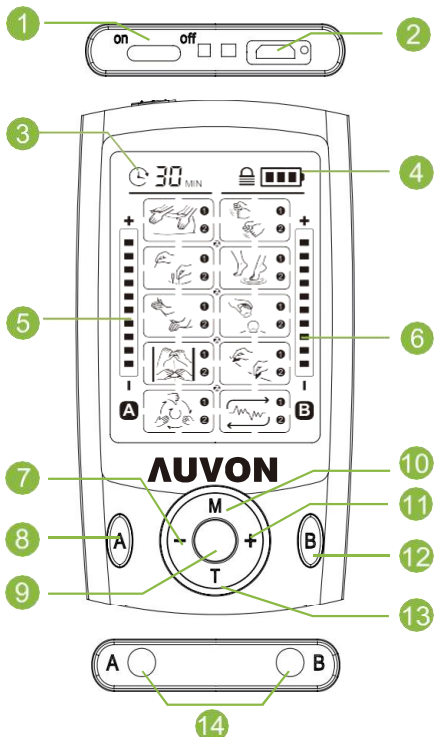
- Strangulation due to cables and hoses, particularly due to excessive length.
- Inhalation or swallowing of small parts: Keep the device out of the reach of children.
- Potential allergic reactions to accessible materials used in the ME EQUIPMENT.
- Electrode gel may cause a skin irritation, if this happens, please seek medical advice.
- Contact injuries: Do not use on around area of the body with skin disease, damage or inflammation.

2. RECOMMENDED MASSAGE POINTS

UK

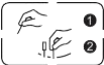
3. ILLUSTRATIONS OF THE DEVICE AND ACCESSORIES

The AUVON Dual Channel TENS Massager is with 10 Massage Modes, and totally 20 Sub-mode Choices.

- ① On / off switch
- ② USB port
- ③ Time
- ④ Battery indicator
- ⑤ Adjustable Intensity (A Channel)
- ⑥ Adjustable Intensity (B Channel)
- ⑦ Intensity Decrease
- ⑧ Channel A
- ⑨ Lock
- ⑩ Mode Selection
- ⑪ Intensity Increase
- ⑫ Channel B
- ⑬ Time Selection
- ⑭ Two output connector

Icon & Mode		Name	Description
	1	Tuina	Simulating Tuina is a hands-on body treatment which opens the flow of Qi through the meridians
	2	Tuina-ombination	
	3	Tapping	Simulating tapping is intended to produce a hard thumping sensation that massage your muscles
	4	Tapping-ombination	
	5	Acupuncture	Simulating acupuncture provides the same sensation without needles
	6	Acupuncture-Combination	
	7	Thai massage	Simulating Thai massage to relax your deep layers of muscles
	8	Thai massage-Combination	

Icon & Mode		Name	Description
	9	Palm kneading	Simulating the feeling of using fingers and palm to apply weight to the body
	10	Palm kneading-Combination	
	11	Cupping	Stimulating cupping, relatively constant stimulation contracts muscles
	12	Cupping-Combination	
	13	Lifting pinching manipulation	Simulating the thumb and index finger lift the skin
	14	Lifting pinching-Combination	
	15	Scrapeing	Stimulating the feeling of scrape over skin with a smooth edged instrument
	16	Scrapeing-Combination	

Icon & Mode		Name	Description
	17	Tapping-Kneading-Pushing	Auto combination of Tapping-Kneading-Pushing
	18	Auto-Combination 1	
	19	Auto-Combination 2	Auto combination of several modes for relaxation
	20	Auto-Combination 3	

Parameter Range:

VP: 0-60V (+/-20%)

Pulse width: 80us(±5%)

Frequency: 0-100HZ (±5%)

OPERATING PROCEDURES

UK

HOW TO USE

1. Clean your skin before use. Use the pads on clean and flat areas, no sweat, no oil.

2. Attach one pair or two pairs of the electrode pads to the massage area, such as shoulder and leg.

Note: Adding a few drops of water on the pads and drying the pads naturally will increase their longevity. Make sure both pads on the skin and not overlapping.

3. Connect a pair of electrode pads to one lead wire by plug them on; the other end of the wire is connected to the left output of the device. Similarly, the other pair of electrode pads is connected to the remaining wire end and the right output of the device.

4. Turn on the device by sliding the on/ off switch on the top of the unit.

5. Press the A button to choose Channel A. Press the M button to try different mode and press +/- to increase or decrease intensity. Press the B button to choose Channel B and select your favorite mode and intensity (The modes on Channel A can be different with that on Channel B).

Note: Remember the strength will default to the lowest setting by each mode selection.

6. Recommended application is 10-60 minutes. Increase 10 minutes by each setting by pressing the T on the device. The time remaining is automatically counted down until it turns off once the time finishes.

7. Press and hold the center button for 3 seconds to lock the screen, 3 seconds again to unlock the screen.

AFTER USE

1. Turn off the device on by sliding the on/ off switch on the top of the unit.

2. Remove lead wire pin from electrode connector by gently twisting and pulling it out.
3. Lift a corner of the electrode and slowly peel it away. Avoid touching the gel as much as possible. DO NOT peel the electrode off by the wire.
4. Remove the electrodes back to the storage bag and reseal tightly to prevent dry-out. Store at room or cool temperature and keep out of direct sunlight.

Note:

- Never connects this product with common headphone.
- Please do not touch USB port in the process of using. The USB port only use for connect the charger, do not connect other device.
- The charger must be in compliance with IEC/EN 60601-1, the use of unauthorized charger can degrade minimum safety.

Warning:

- The stimulator has no any output when it is being charged. you must take off the electrode pads from your body before the stimulator charged.

FEATURES

- 2 output channels with 20 different modes
- 20 intensity level setting
- 20 minutes default timer, can adjustable from 10-60 minutes
- Internal rechargeable lithium battery
- Battery indicator to know when need to recharge battery
- Large, easy to ready LCD display
- Key lock function protection

- Stainless steel backing, portable durable design
- 1 year guarantee
- AC adapter 100-240V (Not included), USB Charger (Not included) 5V
- Stimulator pads and pad wires are applied part

Charging the Battery

1. Turn off the controller.
2. Connect the controller and the charger (not included) with USB extension cord. The charging process will last approximately 1 hour.

Note: Only charge the controller when battery is completely drained the first 2 times. Unplug charger from power outlet when charging is complete. When massage strength decreases, it indicates that the device needs charging. Recharge it and then continue to use the device. Do not use the controller when charging.

4. PRODUCT TECHNICAL PARAMETERS

Product Specification

Name of product	Tens and Muscle Simulator
Model No.	AS8012
Product material	Shell material of controller with ABSPads with silica gel



Power/voltage	3.7V
Consumed current	40mA
Status adjustment	20 kinds of status
Strength level adjustment	20 sections
Timer	With 10-60 minutes stop timer
Weight of main frame	0.159 Kg (including controller, pads, USB extension cord and output cord)
Normal working ambient temperature	5~40°C
Normal working ambient humidity and Atmosphere pressure	≤80%RH; Atmosphere Pressure: 700 ~1060hPa
Store and transport ambient temperature	-20~55°C

Store and transport ambient humidity and Atmosphere pressure	$\leq 93\%RH$; Atmosphere Pressure: 700 ~1060hPa
Safety classifications	internal powered equipment, Type BF applied part, not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous
Charger (Not included)	Input: 100-240V 50/60Hz 0.25A Output: DC5V, 0.55A Class II, not applied part, not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide, continuous operation
Software version	Version: 2014


5. PLACEMENT OF ELECTRODES

UK

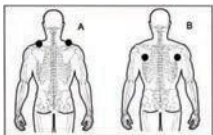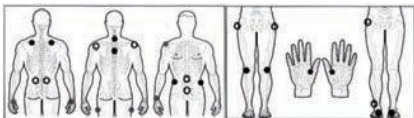
You may use TENS stimulate pads wherever you feel pain and fatigue to relax body. The TENS provides pain relief for a number of different pain conditions, and this product is designed for the temporary relief of pain associated with: Aching muscles in the back, waist, shoulders, arms and joints

Example of Placement for Shoulder

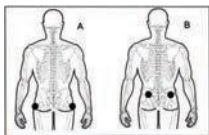Example of Placement for gluteus

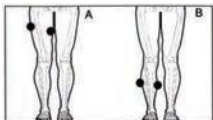Example of Placement for leg

TENS PADS HOLDER

UK

FEATURES

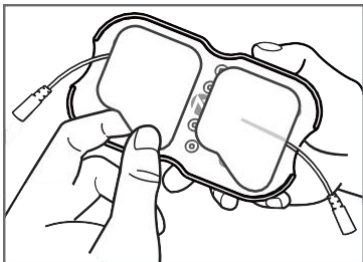
High quality
ABS material to
make sure your
use safely.


Wide
compatibility
for patches in
different sizes.

The shape of salient point designed specially
to lessen damage to the patches.

INSTRUCTION

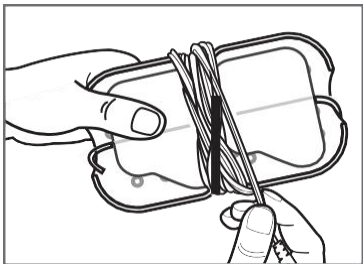- Step 1** Stick the electrode patches on the holder to avoid getting dust.

- Step 2** Wind the thread around the holder for easy use as the picture shown.

The shape of the patch in the picture is for reference only, please refer to the actual product received.

6. TROUBLE SHOOTING

UK

If your device is not operating properly, please check below for common problems and suggested solutions. If the recommended action does not solve the problem, please contact the seller.

Problem	Possible Cause	Solution
One pad feels stronger than the other	This is normal. Different area of your body will react differently	Nothing needs to be done. Make sure the pads are moist and making good contact.
The skin turns red or the skin feels irritated	The adhesive the surface of the pads is dirty or dry	Wash adhesive the surface of pads gently with your fingertips for about 3 seconds under slow running water
	The therapy time is too long or the intensity is set too high	Reduce the application time or reduce the intensity
	The electrode pad surface is worn out	Replace electrode pad

UK

Problem	Possible Cause	Solution
The intensity is not felt with a very weak intensity level	Pads are not attached to the body firmly	Attach both pads firmly to the skin
	The transparent films are still stuck to the pads	Peel off film on the adhesive surface of pads
	The pads stack together or overlap	Do not stack pads together or overlap pads
	The cord is not properly connected to the unit	Connect cord correctly into the jack
	The intensity of the setting is getting weak	Increase the intensity level
	The battery capacity is low	Charge the battery
No power source; no display on LCD.	The battery capacity is depleted	Charge the battery

Problem	Possible Cause	Solution
It is difficult to attach the pad to the skin	Have you remove the transparent film from the pad?	Peel off film on the adhesive surface of pads
	Was the pad applied immediately after washing?	Dry the pad
	Is the adhesive on the surface of the pad damaged?	Replace the pad
Power cuts off during use	The battery is weak	Charge the battery
	The cord is broken	Replace the cord
Adhesive on the surface of a pads not sticky	Pads get deteriorative	Contact the vendor for replacements
	Were the pads stored under high temperature, high humidity, or direct sunshine?	Replace the pad

7. MAINTENANCE AND STORAGE

1. Unplug the output cord from the output jack of the controller after using.
2. Cover both pads with the protective film before storage.
3. Never fold the massage pads.
4. To keep the controller clean, use a soft and dry cloth for dust or a soft damp cloth for any dirt and smudges. Do not use any cleaning solutions to clean the controller and its pads.
5. Never apply the pads on any surface other than your skin. If the pads become soiled or dirty, the adhesive power may decrease. In this case, moisten the surface of the pads with water and wipe away the dirty portion. This will allow a temporary restoration of the adhesive power. However, too much water will result in loss of the adhesive power.
6. Do not use or store the devices where there are magnetic fields or electric wave (near TV set or speakers).
7. Do not place the devices in areas of high temperature, high humidity, or under direct sunlight.
8. Keep the device out of reach of children.
9. All worn accessories should be dealt according to the regulations.
10. Do not immerse the device in water or any liquid. Do not drop the device or throw it from a height.
11. After using the device, please remove the electrode and replace the protective film. Always use the protective film when the electrode is not in use.
12. Cleaning
 - Machine washing and soaking is PROHIBITED.
 - Do not wash the belt in fabric softener as this can damage the belt.
 - Do not dry clean.

- Do not use bleach.
- Clean accessories with a slightly wet sponge with detergent and wipe with a cloth.
- Clean the unit with a cloth soaked in with disinfectant (Mercryl type) diluted in water at 1/10th.
- Clean the electrodes with a mix of water/ hypoallergenic detergent, then rinse thoroughly.
- Keep the device and electrode clean.

ELECTROMAGNETIC COMPATIBILITY

The AS8012 has been tested and found to comply with the electromagnetic compatibility (EMC) limits for medical devices to IEC 60601-1-2: 2014. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation.

CAUTION

Do not use this device simultaneously with devices having high EMI levels.

Guidance and manufacture's declaration – electromagnetic emission
AS8012 is intended for use in the electromagnetic environment specified below. The customer of the user of AS8012 should assure that it is used in such an environment.

Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	AS8012 use RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment
RF emissions CISPR 11	Class B	AS8012 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacture's declaration – electromagnetic immunity

AS8012 is intended for use in the electromagnetic environment specified below. The customer or the user of AS8012 should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±15kV air S	±8kV contact ±15 kV air	Floors should be wood, concrete or ceramic tile. If floor is covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines	±2kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
<p>Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11</p>	<p><5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 sec</p>	<p><5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 sec</p>	<p>Mains power quality should be that of a typical commercial or hospital environment. If the user of AS8012 requires continued operation during power mains interruptions, it is recommended that AS8012 be powered from an uninterruptible power supply or a battery.</p>

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Surge	± 1 kV line(s) to line(s)	± 1 kV differential	Main power quality should be that of a typical
IEC 61000-4-5		Mode	Commercial or hospital environment.
Power frequency (50Hz/60Hz) magnetic field IEC 61000-4-8	30A/m		Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE UT is the ac. mains voltage prior to application of the test level.			

8. DISPOSAL OF MATERIALS

A symbol of Waste Electrical and Electronic Equipment means you should find out about follow local regulations about disposing this kind of product.

- Do not dispose this product as you would other household waste. Dispose this device in accordance with the corresponding local regulations.
- Electrical and electronic devices contain hazardous substances that can have harmful effects on the environment and/or human health and should be recycled properly.

9. CUSTOMER SERVICE

- 24-Month Limited Warranty
- Lifetime Technical Support

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Shenzhen Astec Technology Co, Ltd