

# FDA EUA KN95 mask instructions for use

## Product Name

FOLDABLE PROTECTIVE EARLOOP MASK

## Product Brand

XIKII

## Product model

Non-medical KN95 mask



## User instructions

1. Open the KN95 mask and check whether the mask is complete and defective. If there is a defect, do not use it.
2. Open the mask by hand and pull the ear straps with both hands.
3. Hang the ear straps of the mask on your ears to adjust the wearing comfort of the mask.
4. Press the bridge of the nose so that the upper part of the mask is close to the bridge of the nose.
5. Press around the mask with your hands to check whether the mask is closed to your face after wearing the mask.

## Instruction and direction



## Product information

Material: non-woven, melt-blown, electrostatic cotton.

Implementation standards and filtering levels: GB2626-2006 KN95

Storage conditions: 10-25°C temperature, relative humidity<80%, keep cool and ventilated.

Validity period: 2 years, the production date is shown on the package.

Tips: The user should read the product instruction carefully before wearing it.

## Remarks

1. Please do not use and promptly replace the mask if the package and mask are damaged or stained.
2. Please read the instructions and directions carefully before use, improper usage may cause the product's protection performance to fail.
3. Prohibited to use after the expiration date.
4. It is not recommended to use the same mask for a long time in non-special circumstances. Please replace the mask at least in a week.