



## **Installation and Care Instructions:**

### ***Prior to installation:***

- Be aware of all warning labels on your Nomad™ that will be covered by the Protomad™, and ensure all Nomad™ operators are trained to conform to all Nomad™ warning labels and manufacturer's operating instructions.
- Inspect the Nomad™ unit for any cracks or other damage, and never operate any damaged or defective unit.
- Repair or replace any defect of the Nomad™ device case as per the manufacturer's recommendations.
- Find a clear working surface, table or countertop, for Protomad™ cover installation.
- Slide the clear reflector disk on the Nomad™ all the way in the forward position.
- Secure the Protomad™ cover retention clasp on one side of the Protomad™ if not already attached.

### ***Installation:***

- Keeping the Nomad™ over the cleared working surface, slide the cover over the Nomad™.
- Maneuver the cover into position until it is fully seated.
- In the fully seated position the cover retention clasp should be able to be clasped without stretching, and the Protomad™ control panel window should be flat against the Nomad™ control panel for best visibility.
- Securely fasten both sides of the forward cover retention clasp to hold the Protomad™ firmly in place on the Nomad™.
- Wipe the assembled unit and cover with your disinfectant of choice and, after allowing to dry, it is ready for use. Remember to always keep one hand firmly on the Nomad™ handle grip per manufacturer's instructions when handling and operating.

### ***Care:***

- Remove the Protomad™ from the Nomad™ and inspect the Nomad™ at regular intervals to ensure the case is not damaged or requires replacement per the manufacture's recommendation.
- Disinfect all surfaces of both the Nomad™ and Protomad™, and after allowing to dry, reinstall the cover as instructed above.

### ***Warranty:***

- The Protomad™ has a one-year, full replacement on return, warranty against damage with the exception of discoloration.