

Radiant Impressions Warranty

2 years from the receipt of your breast form.

Please read the care instructions attached carefully, as any warranty claim may be affected if you do not follow the guidelines supplied.

In the event that your breast form becomes defective within the warranty period, please contact your provider who will arrange with Trulife for a replacement product.

Please note that small tears in the honeycomb core are expected over time but in no way affect use, integrity, fit or shape and are not considered a defect.

Excluded from this warranty are defects in the Radiant Impressions product that have been modified or altered. Specifically, but without limitation, attaching or applying to our product any material, object or product that was not manufactured by us will void this warranty.

Serious incidents that occur in relation to these products should be reported to Trulife at: quality_department@trulife.com and the Competent Authority of your member state.



For more information on Radiant Impressions products, please contact your local provider.



Radiant Impressions

Customer Information and Warranty



 Trulife, Airton Road, Tallaght, Dublin 24, Ireland.
Tel: +353 1 4511755 Fax: +353 1 4275790

www.trulife.com



TL/BC/RI/1/123

A Beautiful Option

after mastectomy, lumpectomy or reconstruction

Thank you for choosing a Trulife Radiant Impressions Custom breast form.

Following breast surgery, it's important to get back to the things you love doing. A Radiant Impressions custom breast or overlay form can help you look great and feel your best. Your custom breast form can also be worn in chlorinated or salt water, allowing you to get back to the activities you enjoy.

We recommend a review of your breast form fitting every two years or sooner if you notice a difference in your shape, size or if weight gain/ loss creates an imbalance.



Hand crafted with care

Each Radiant Impressions custom breast form is carefully designed and hand painted by experienced engineers and artisans that understand how to match your individual contour, shape and profile.



Routine Cleaning

Wash your breast form as often as required by following the steps below:

1. Apply a pea size amount of liquid soap to a wet cloth and use this to wash the breast form, front and back. Wipe with a dry cloth.
2. Bathe the breast form as required by submerging in warm water and soap. Rinse the breast form thoroughly until the water runs clear.
3. The honeycomb core of your breast form will retain water. To extract this water, squeeze out the product after rinsing. Then place the back of the breast form onto an absorbent towel and press down repeatedly on the form.
4. Leave to dry, face downwards (with the back exposed), overnight, or dry with a hairdryer on a medium heat setting.
5. To further enhance the silky soft feel you may wish to apply body powder evenly over the front of the breast form.

Note: Radiant Impressions are 100% natural silicone products and may release minuscule amounts of natural oils over time. This is perfectly natural and will not adversely affect the product or the user in any way.

Please dispose of your breast form in accordance with local and national laws and regulations.

Product care guidelines

To ensure long lasting satisfaction and performance, please follow these guidelines:

- Do not pinch, stretch or wring the breast form as this can cause small separations in the silicone honeycomb core.
- Avoid applying perfume, deodorant, cream, moisturizer, tanning products or any lotions to the breast form.
- Only to be worn when the surgical site is completely healed. Please consult with your physician before wearing a silicone prosthesis. Do not wear if skin is broken or irritated. If any irritation occurs, discontinue use immediately and contact your physician.
- Your Radiant Impressions breast form may be permanently stained by ink and fabric dye. Take care to protect your breast form.

Fitting in a bra pocket

Your Radiant Impressions custom breast form can be worn directly against your skin or in your bra pocket.

