
PECTUS PRO PECTUS CARINATUM ORTHOSIS Fitting Instructions for Asymmetric Pectus Carinatum

The Pectus Pro Brace has been designed to allow lateral adjustment of the front panel, and back panel, for patients presenting **Asymmetric Pectus Carinatum**.

The following instructions will guide you through the process of adjusting the brace's settings to maximise its lateral range.

First, identify the different components of the Pectus Pro Brace.

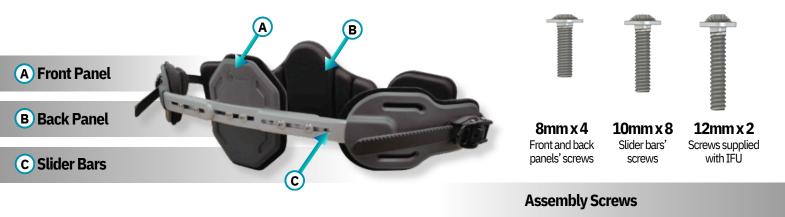

The Pectus Pro brace is supplied in a **symmetric configuration** with the Front and Back panels adjusted at the centre. For patients with an **asymmetric presentation**, with one side of the chest pushing out further than the other, the Front panel can be adjusted laterally by following the below incremental configurations.

