


PECTUS PRO
Pectus Carinatum Orthosis
Installation Guide

XST-021/01/24

Fitting Instructions

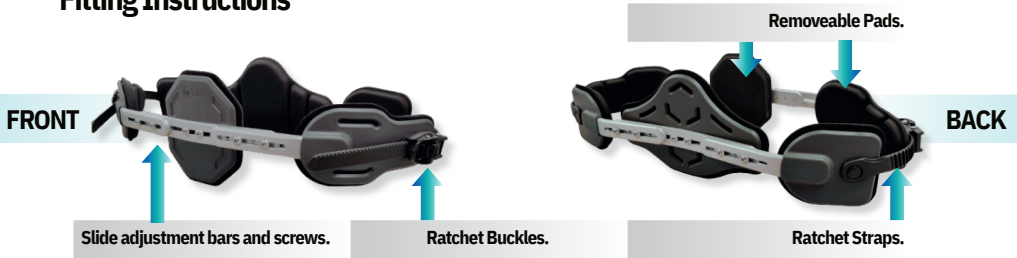

1. Loosen the sternal and posterior adjustment screws to increase or decrease the circumference, and/or to adjust the front panel for asymmetric Pectus Carinatum.
2. Slide the components to the desired circumference/position and re-tighten using the included hex key.
3. The metal parts can be contoured with the use of bending irons. Care should be taken not to make sharp bends in slotted areas.
4. Fit the orthosis and tighten to the desired tension using the ratchet buckles. If needed, the included optional shoulder straps can be used to prevent slippage of the orthosis. Feed the straps through the front and back panel slots, fasten using the hook and loop fasteners, and check fit.



5. To remove the orthosis, push down on the release button in the centre of the buckle and remove the strap. It is recommended to only remove the orthosis by loosening one side.

Intended Use

- The Pectus Pro is an orthotic spinal stabilization brace for Pectus Carinatum (Pigeon chest), and other chest anomalies, that provides direct anterior-posterior corrective compression forces.
- This device should only be used as directed by a qualified medical practitioner.

Indications

- Abnormal chest shape, protrusion of both the sternum (breastbone) and the cartilage portion of the ribs, and bowed chest (Pigeon chest).

Contraindications

- Severe chest deformations.

Maintenance

- It is recommended that a snug fitting under shirt be worn between the skin and the orthosis.
- The brace should be regularly clean by wiping with a damp cloth and handwash the removable pads and straps.



Important

- Discontinue use immediately if you notice any excessive redness, abrasions, or irritation, and contact your medical practitioner.

Disposal

- Please dispose of your Pectus Pro orthosis in accordance with local and national laws and regulations. The device does not contain any hazardous materials.

Warranty

- 90 days



Trulife

3225 Woburn St. Suite 160
Bellingham, WA 98226
USA

Phone (+1) 360 697 5656

Email supportop@trulife.com



MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany

