

Reynard Neutral detergent wipes.

Unparalleled equipment compatibility.

An alcohol and fragrance-free solution to effectively cleanse hard surfaces and equipment with no risk of tainting. Widely used in the health sector for cleaning medical equipment, mattress covers, operating theatres and other areas where harsh chemicals need to be avoided.

Points to remember:

- Use one wipe per piece of equipment or surface
- Wipe from clean to dirty
- Clean from top to bottom
- Always reseal the pack after use

Step 1: Remove plastic tamper-proof seal strip from under the tub lid and discard.



Step 2: Remove lid by easing off area by area in a circular motion.



Step 3: Open the sealed bag containing the wipes (do not remove the cloths from the bag).



Step 4: Take hold of the wipe in the centre of the roll and pull through the dispensing hole in the tub lid.



Step 5: Reattach the lid to the tub.



Step 6: Pull the required number of wipes through the dispensing hole.



Step 7: Pull sharply in a downward motion to detach the wipe(s) from the roll, ensuring that the top of the next wipe is presented



outside the dispensing hole for next use.

Step 8: Always reseal the lid to keep the wipes moist.



Please note: we recommend following hospital patient cleaning protocols, including the use of P.P.E. where appropriate.

Product code	RHS206
Wipes per canister	280
Canisters per carton	2



Reynard is reassurance.

