



**MALTA DYNAMICS**

**3-PLY DISPOSABLE  
EVERYDAY MASK**

FM1001



**PRODUCT MANUAL**



# ONE - TIME THREE - FACE MASK

TAEP0552

Non-woven fabric, nose piece, with elastic ear loops.

Product Specification - 17.5cm×9.5cm±5%

This product is applicable for the health care of the user in the general environment.

## INSTRUCTIONS

- 1.Clean hands and check the packaging and the validity before use
- 2.The side of the mask with color goes outward and the side with metal soft strip goes upward. When taking off the mask, hands shall not contact with the inner part
- 3.Fix the elastic ear loops around ears
- 4.Put the metal soft strip on the bridge of nose with fingers, and unfold the wrinkles to cover the mouth, nose and the whole jaw
- 5.Press the metal soft strip on the bridge of nose and face to make sure that the edge of the mask completely fits the face and does not affect breathing
- 6.Avoid touching the mask when wearing it
- 7.When taking off the mask, hands shall only touch the elastic ear loops
- 8.Discard the used masks in the garbage, and clean hands

Contraindication - None

## ATTENTIONS

- 1.Mask is not sterile
- 2.Mask is forbidden to use if the packing is damaged or the product is out of date
- 3.This product is disposable and shall not be reused
- 4.Please refer to the instructions before use
- 5.Please distinguish between the upper and lower sides and the inner and outer parts of the mask. The side with metal strip shall be the upper side. The blue side shall be the outer part and the white one shall be the inner part. Hands shall not contact with the inner part of the mask
- 6.The product shall be used as soon as possible after unpacking
- 7.The used masks shall only be discarded in the special garbage bags for medical equipment wastes

8. The masks shall be stored in a well ventilated room without corrosive substances where the relative humidity shall be no more than 80%
9. In case of discomfort during wearing, please remove mask

## PACKING INSTRUCTIONS

50 masks / box; 75 boxes /carton.

## STORAGE AND TRANSPORTATION

1. The packed products shall be stored in a room which is dry with good ventilation conditions without direct sunlight or corrosive gas;
2. During transportation, loading and unloading process of this product, it is forbidden to be put in the sun for a long time or in rain or snow without covering

Execution Label - GB/T 32610-2016;EN 149:2001+A1:2009

Date of Production - See packing

Storage Period - Two years

Manufacturer - Bengbu Han Yu Technology Limited

Address of Manufacturer - Technology and Innovation Industrial Park,  
Huaishang district, Bengbu City, Anhui Province, China

Document Number - PM-TAEP0552-20200220

Revision Date - April 2020





# MALTA DYNAMICS

800-494-1840  
MaltaDynamics.com  
210 13th Street  
Malta, OH 43758 USA