M5072A HeartStart Infant/Child SMART Pads
Cartridge

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ENGLISH: About this edition

The information in this guide applies to the HeartStart model M5072A infant/child SMART Pads cartridge for use with HeartStart HSI defibrillators. These include the HeartStart OnSite Defibrillator M5066A, the HeartStart Defibrillator M5067A, and the HeartStart Home Defibrillator M5068A. This information is subject to change. Please contact Philips at www.philips.com/productdocs for information on revisions.

Edition History

Edition 9

Publication date: August 2011

Publication #: 012664-0010

Printed in the U.S.A.

Copyright

© 2011 Philips Electronics North America Corp. No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system or translated into any human or computer language in any form by any means without the consent of the copyright holder.

Unauthorized copying of this publication may not only infringe copyright but also reduce the ability of Philips Medical Systems to provide accurate and up-to-date information to users and operators alike.

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician.

Authorized EU Representative

Philips Medizin Systeme Böblingen GmbH
Hewlett-Packard Strasse 2
71034 Böblingen, Germany
(+49) 7031-463-2254

Device Manufacturer

Philips Medical Systems, Seattle, WA 98121-1825, USA

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Philips Medical Systems, Seattle WA 98121-1825,
 USA

ENGLISH

What do I need to know about the infant/child pads cartridge?

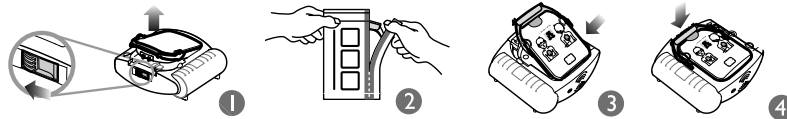
- The infant/child pads cartridge is designed for use with the HeartStart HSI Defibrillators. When used with the infant/child pads cartridge, the defibrillator delivers a lower-energy shock, and the optional CPR Coaching is tailored for use on a child.
- To keep the pads from drying out, keep the infant/child pads cartridge in its unopened foil pouch. Store it near your defibrillator, until you need to use the pads. It is recommended that you store the HeartStart Defibrillator with an adult pads cartridge installed.
- Be sure to note the expiration date of the pads cartridge when you receive it.
- CPR for infants and children is different than for adults. It is important to get training in infant/child CPR.
- If you have concerns about your child's health, be sure to talk to your child's doctor.

How do I know when to use the infant/child pads cartridge?

- Use the infant/child pads on an infant or a child less than 8 years old or weighing less than 55 pounds (25 kg) who may be in sudden cardiac arrest (SCA).
- If you are not sure of the child's exact age and weight, or if infant/child pads are not available, do not delay treatment.
- Use adult pads but place one pad on the chest and one in the center of the back, as shown in Emergency Instructions, below.
- A victim of sudden cardiac arrest does not respond when shaken, and is not breathing normally. If in doubt, apply the pads.

How do I install the infant/child pads cartridge?


- To install the infant/child pads cartridge, remove the adult pads cartridge from the defibrillator. Slide the latch at the top of the defibrillator to release the cartridge ❶. Lift out the adult pads cartridge.
- Open the foil pouch and take out the infant/child pads cartridge ❷.
- With the PULL handle side up, put the lower end of the cartridge into the pads compartment ❸.
- Press down the top end until the cartridge clicks into place with the PULL handle pressed down ❹.

- After using the infant/child pads, remove the cartridge. Install an adult pads cartridge before storing the defibrillator.

EMERGENCY INSTRUCTIONS

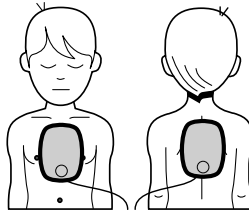- Infants and children more often become unresponsive due to breathing or other issues than to heart problems. When responding to cardiac arrest in an infant or child, provide infant/child CPR while a bystander calls EMS and brings the HeartStart. If no bystander is available, provide 1-2 minutes of CPR before calling EMS and retrieving the HeartStart. If you witnessed the child's collapse, call EMS immediately and then get the HeartStart.
- Insert the infant/child pads cartridge and turn on the HeartStart Defibrillator. It will give you voice instructions for each step.
- Pads placement for infants and for children under 8 years old or 55 pounds (25kg) is different from adult pads placement. Place one pad in the center of the chest, between the nipples. Place the other pad in the center of the back.

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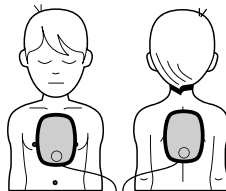
Philips Medical Systems is part
of Royal Philips Electronics

PHILIPS

Philips Medical Systems

United States
Philips Medical Systems
2301 Fifth Avenue, Suite 200
Seattle, WA 98121-1825, USA
(800) 263-3342

Canada
Philips Healthcare, a Division of Philips Electronics Ltd.
281 Hillmount Road
Markham, Ontario
L6C 2S3, Canada
(800) 291-6743

Europe, Middle East, and Africa
Philips Medizin Systeme Boeblingen GmbH
Cardiac and Monitoring Systems
Hewlett-Packard Strasse 2
71034 Boeblingen, Germany
+49 7031 463 2254

Latin America
Philips Medical Systems Ltda.
Rua Verbo Divino - 1400 - 7o Andar
04719-002 -Sao Paulo, Brazil
0800 7017789

Asia Pacific
Philips Electronics Hong Kong Ltd.
6/F, Core Building I
I Science Park East Avenue
Hong Kong Science Park
Shatin, New Territories, Hong Kong
(852) 2821 5888

0 1 2 6 6 4 - 0 0 1 0

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© 2011 Philips Electronics North America Corp. Kein Teil dieser Publikation darf ohne Zustimmung der Copyright-Inhaber in irgendeiner Form oder auf irgendeine Weise reproduziert, übertragen, kopiert, gespeichert oder in eine menschliche oder Computersprache übersetzt werden.
Niets uit deze uitgave mag in enige vorm of op enige manier worden verveelvoudigd, verzonden, getranscribeerd, opgeslagen in een geautomatiseerd gegevensbestand of vertaald in menselijke of computertaal zonder toestemming van de houder van het copyright.
© 2011 Philips Electronics North America Corp. Non sono consentite la riproduzione, trasmissione, trascrizione, memorizzazione in un sistema di recupero dati o la traduzione in altre lingue o linguaggi software della presente pubblicazione, in qualsiasi forma o con qualsiasi mezzo senza il consenso del detentore dei diritti di copyright.