

WASHING INSTRUCTIONS

How to clean your Sacci-products

- The bags can be machine washed at a maximum of 60 degrees, 1time/quarter, 8 washes.
- Centrifugation is not recommended, let the product drip dry.
- For increased longevity, regular wiping and disinfection is recommended instead of machine washing.
- The washing instructions only apply to bags, not inserts/middle inserts.
- Medical bags from Sacci have a guarantee of 2 years.

If you have any questions please contact us at:
support@sacci.se or 0243-932 50



We got your back.



Sacci is an innovative developer and production partner of high quality carrying systems. We've made carrying easy since 1914.

SACCI SINCE 1914