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MASQUE DE PROTECTION UNS1 - M3LH011

Composition de la matière principale : 50% Polyester - 50% Polypropylène

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USER GUIDE

PROTECTIVE MASK UNS1 - M3LH011

Composition of the main material: 50% Polyester - 50% Polypropylène

Washing method: Machine wash at 60 ° C with conventional detergent. Drum drying. This mask meets AFNOR recommendations of 5 washes as they can be washed and reused up to 20 times in accordance with the tests carried out in our laboratories. Iron at a temperature of 120 ° C.

Performances: Test protocol described in the DGA document of March 25, 2020 Category 2 mask "UNS 1" = Individual mask for use by professionals in contact with the public. This mask is intended for personnel assigned to positions or missions involving regular contact with the public. Filtration efficiency of particles of 3 µm:> 90 %. Breathability: air permeability test> 96 L.m².S-1 for a vacuum of 100 Pa.

Usage precautions: The wearing time of the mask is limited to 4 hours. Test to wear for 4 hours carried out by us. Before wear a mask, wash your hands with soap and water or a hydroalcoholic solution; Apply the mask to cover the nose and mouth. Make sure to adjust it as well as possible on your face; check there is no air jet in the eyes during a strong exhalation; When wearing the mask, avoid touching or moving it. Whenever you touch a used mask, wash your hands with soap and water or a hydroalcoholic solution; If you need to drink or eat, change your mask; When it gets wet, replace it with a new mask; To remove the mask, Remove it from the back (do not touch the front of the mask); Throw it away immediately in a closed bin (if it is no longer reusable); Wash your hands with soap and water or with an alcoholic solution.

Mask maintenance: After use, the mask must be isolated either to be thrown away or to be washed. It is recommended to isolate the mask in a disposable plastic bag or in a reusable washing bag. In the case of the reusable bag, mask and bag can be washed together, in this case, remove the mask from the bag. Unused masks to stoker in a clean, dry and out of light place.

Usage context: This mask is strictly adapted to the health crisis linked to COVID 19.

Field of use : This mask is intended for the professional environment and is exclusively reserved for Non Sanitary Uses (UNS). It is intended to prevent droplet projections. The use of this mask is part of the strict application of measures related to containment, measures of work organization as well as barrier gestures. This collective mask is intended to protect an entire group wearing these masks in the context of their professional activity. This mask is worn by people with occasional professional contact with other colleagues (business, service, office), it is also worn in the presence of other individuals wearing a mask from another category. It is not intended for use by caregivers in contact with patients

Label worn inside the mask