nouri auto

User Manual
Manual del usuario
Mode d'emploi
Benutzerhandbuch
Manuale d'uso

TensCare™

Contents

Introduction	3
Kit Contents	3
Pump Parts	3
Cleaning and Sterilising	4
Installation Guide	6
Pump Controls and Display	7
Pumping with Nouri Auto	7
Storing Breast Milk	9
Preparing Breast Milk	9
Power Supply	10
Feeding Breast Milk	10
Charging your Nouri Auto	10
When to start using Nouri	11
Troubleshooting	11
Warranty	12


Introduction

Nouri Auto is an automatic breast pump, designed to make breast pumping quick and easy. The pump offers a variety of suction levels and modes alongside its soft breast shields. Nouri Auto includes an integrated light, allowing you to pump day and night in complete comfort.

Kit Contents

- 1 x Electric pump
- 2 x Bottle (150ml)
- 1 x Feeding teat and cap
- 1 x Diaphragm
- 1 x Tube
- 1 x Breast Shield
- 1 x Shield cover
- 1 x Shield connector
- 1 x Bottle cap
- 1 x USB cable
- 1 x Bottle dock
- 2 x Bottle valve
- 1 x Diaphragm cap

Pump Parts

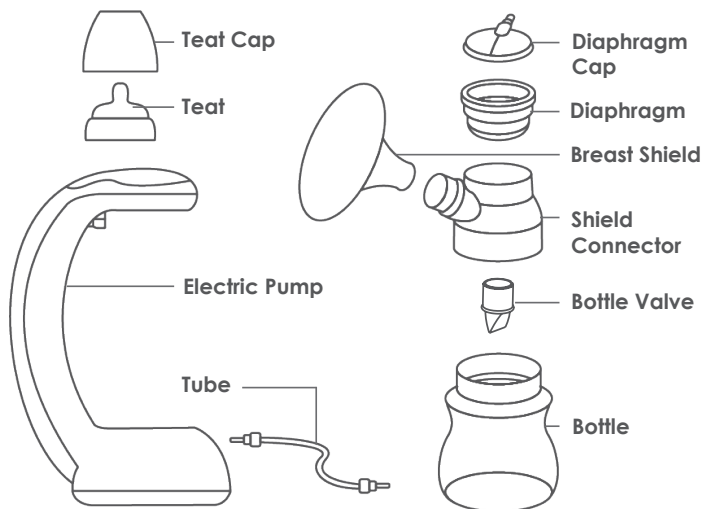Cleaning and Sterilising

Before using for the first time

Clean and sanitise the below parts before using your Nouri Auto for the first time. Use the following cleaning and sanitising instructions.

- Breast shield
- Shield connector
- Bottle Valve and Valve membrane
- Bottle

After each use:

Clean all parts that come in contact with your breast and breast milk.

- Separate all parts, remembering to remove the clear membrane from the bottle valve
- Rinse the parts in cool water to remove breast milk residue.

Washing in sink:

- Soak all parts in warm soapy water for 5 minutes and then clean with a clean dish-cloth or soft brush.
- Rinse all parts that come in contact with your breast and breast milk with clear water.
- Allow parts to dry.

Washing in dishwasher:

- Wash all separated parts on the top rack of the dishwasher.
- Allow all parts to dry.

Sanitise daily:

- Wash hands thoroughly.
- Separate all parts that come in contact with your breast and breast milk, remembering to remove the clear membrane from the bottle valve
- Immerse parts in boiling water for 10 minutes using a household pot.
- Allow water to cool before removing parts from the water.
- Place parts on a clean surface and/or towel and allow to dry.

Tubing and Electric Pump Unit

- Turn off the breast pump
- Unplug the pump from any power source.

- Separate the tube from the pump and bottle.
- Wipe the electric pump unit and tube with a clean, damp cloth.

Caution:

Only use drinking tap water or bottled water for cleaning. Separate and wash all parts that come in contact with the breast and milk immediately after use to avoid milk residue drying and to prevent bacteria growth.

Do not store wet or damp parts.

During sanitising with boiling water, prevent the parts from touching the side of the pot. This can cause the parts to deform or be damaged. Nouri is for single person use. To avoid contamination, please do not share the pump.

Note:

Wash the Nouri Auto parts that come in contact with the breast and breast milk after every use.


Sanitise the Nouri Auto parts that come in contact with the breast and breast milk once a day.

Be careful not to damage parts during cleaning.


Installation Guide

1. Wash your hands thoroughly with soap and water.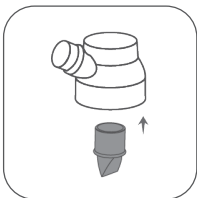

2. Insert the bottle valve into the shield connector.

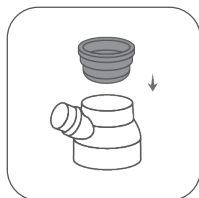3. Insert the diaphragm into the top of the shield connector.

4. Attach the diaphragm cap on top of the diaphragm with the tube insert positioned roughly 135° from the shield connector thread.


5. Turn the diaphragm cover 45° clockwise.

6. Screw the bottle into the shield connector assembly.


7. Attach the breast shield to the shield connector.

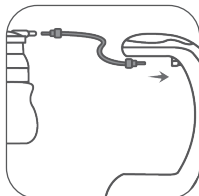8. Connect the bottle to the electric pump by inserting the tube firmly onto the diaphragm cap and pump unit or insert the diaphragm cap directly into the pump.


Pump Controls and Display

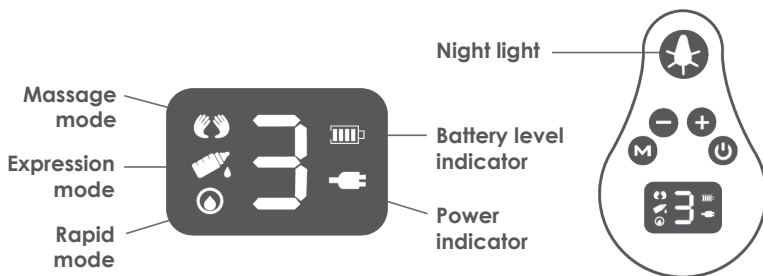Pumping with Nouri Auto


1. Wash your hands thoroughly with soap and water and make sure that your breast is clean.

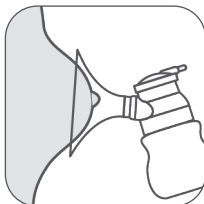2. Place the breast pump onto your breast. Make sure that you centre your nipple.

3. Turn on the Nouri Auto. Note pumping will start automatically so make sure the breast is positioned correctly before switching on the pump.

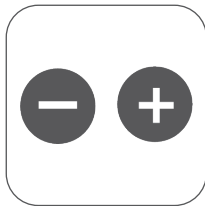
4. When turning on, massage mode will begin. Use the increase and decrease buttons to find a comfortable suction level.

5. After 2 minutes, expression mode will automatically begin. Again, select a comfortable suction level using the increase and decrease buttons.

6. For a faster expression, switch to rapid mode by pressing and holding the mode switch button for 2 seconds.

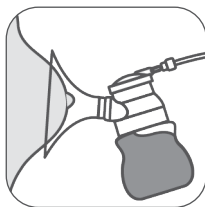7. Stop pumping when the milk stops flowing or when the bottle is full.

8. Remove the electric pump unit (and tube) from the bottle assembly.

9. Unscrew the bottle from the shield connector. Use the bottle cap to seal the bottle for storage or the feeding teat for feeding.

Note:

- The pump will run for 30 minutes before automatically turning off. A single expression should not last longer than 20 minutes.
- To return to expression mode from rapid mode, press and hold the mode switch button for 2 seconds. To switch between massage mode and expression mode, press the mode switch button. Massage mode cannot be accessed from rapid mode.
- Night light has 2 brightness settings. Press the light button to cycle between OFF, low brightness and high brightness.

Cautions

- As shown in the illustration, your nipple should be positioned centrally in the breast shield. Incorrect positioning may cause irritation or frictional pain.
- Volume of feeding bottle is 150ml. If milk volume is estimated to be more, please use a second bottle.
- If you feel pain or discomfort, stop using the pump and refer to the Troubleshooting section of this manual.

Storing Breast Milk

Storage location	Temperature	Max. storage duration
Room	16-29 °C (60-85 °F)	4 hours
Refrigerator	4 °C (39 °F)	4 days
Freezer	<-4 °C (25 °F)	0-6 months

The above only applies for infants one month of older. Please consult a doctor for more details.

Warning:

- Do not refreeze thawed milk.
- Do not mix fresh breast milk with thawed milk.

Note:

- Fresh breast milk should be stored in refrigerator or freezer.
- Be sure to date the milk for storage.
- When freezing, maximum milk volume should be $\frac{3}{4}$ of storage bottle. Milk can expand during freezing.

Preparing Breast Milk

- If the milk is refrigerated, put the milk container in warm water to heat the milk. Shake the container to warm the milk through.
- If the milk is frozen, defrost in the refrigerator 24 hours prior to using. Once defrosted, warm it as refrigerated milk.

Using milk heater:

Turn on milk heater and set it to 40°C. Put milk container into heater. Layers of liquid may form when using frozen milk. This is normal. Please shake bottle well before feeding.

Warning:

- Do not heat thawed milk again.
- Do not use microwave oven to defrost breast milk.
- Do not heat milk with boiling water.
- If hot water is used to heat milk, please try milk temperature before feeding.

Feeding Breast Milk

It is recommended that breastfeeding is well established prior to bottle feeding your baby.

- Always inspect the bottle, teat and other parts before and after each use. If the teat appears cracked or torn, stop using immediately.
- To prevent possible choking hazard, test the strength of teat by pulling on bulb portion of the teat.
- Do not attempt to enlarge the teat hole.
- Infants must not be bottle fed without adult supervision.
- The teat should not be used as a pacifier.
- Clean and disinfect the bottle before first use and after every use.

Charging your Nouri Auto

To charge the Nouri Auto, connect the USB charging cable to the pump and a power socket. The charging indicator and battery level will show on the display.

When fully charged, the battery indicator will stop flashing and show a full battery.

The Nouri Auto can be used during charging if needed.

The battery has a charge time of 2 hours and will last for 70 minutes before needing to be recharged.

Power Supply

Input: AC100-240V (50/60Hz) 0.2A

Output: 5V/1A

Built-in lithium ion battery: 3.7V/850mAh

Warning:

- Do NOT use a USB charging cable other than the cable provided.

When to start using Nouri

1. If you start breastfeeding successfully, we suggest to start using Nouri after regular milk secretion and feeding is formed. (Breast pumps are usually used 2-4 weeks after the baby is born)
2. Nouri Pump should be used in the following circumstances:
 - If you cannot start breastfeeding, regular pumping helps establish and sustain milk secretion.
 - When you feed your baby in hospital.
 - When you feel your breast is swelling or painful, you can pump milk between feeding to release discomfort.
 - If your nipple is painful or cracked and therefore unable to feed your baby, you can pump milk until the nipple recovers.
 - If you have to leave your baby for a period of time and therefore not breastfeed regularly, you can pump milk to keep functional milk secretion.

Troubleshooting

Problem	Possible causes	Solution
Pump does not work	<ul style="list-style-type: none">• Empty battery• Battery installed incorrectly• Power cable is not connected properly.	<ul style="list-style-type: none">• Recharge battery• Make sure battery is installed correctly.• Check power cable is firmly connected with the pump and power socket.
Low/no suction force	<ul style="list-style-type: none">• Incorrectly positioned on the breast• Tubes are not connected correctly.• Incorrect installation of valve plate/valve plate damaged	<ul style="list-style-type: none">• Check shield is placed on the centre of the breast.• Make sure tubes are firmly connected with the diaphragm cover and pump unit.• Check valve plate is installed correct and there is no deformation.
Pain during pumping	<ul style="list-style-type: none">• Incorrectly positioned on the breast• Too high suction force	<ul style="list-style-type: none">• Check shield is placed on the centre of the breast.• Use lower suction force• Using massage mode before express milk can help to alleviate pain.

Warranty

This product is warranted to be free from manufacturing defects for one year from the date of purchase. This warranty is void if the product is modified or altered, is subject to misuse or abuse; damaged in transit; lack of responsible care; is dropped; if damage occurs by reason of failure to follow the written instruction booklet enclosed; or if product repairs are carried out without authority from TensCare Ltd.

We will repair, or at our option replace free of charge, any parts necessary to correct material or workmanship, or replace the entire unit and return to you during the period of the warranty. Otherwise, we will quote for any repair which will be carried out on acceptance of our quotation. The benefits conferred by this warranty are in addition to all other rights and remedies in respect of the product, which the consumer has under the Consumer Protection Act 1987.

Our goods come with guarantees that cannot be excluded under the UK consumer Law. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality.

Before you send your unit for service please be sure to read the user manual and follow all the instructions.

Should repair be needed within the warranty period, please return the device with the following details:

1. Name:
2. Address:
3. Telephone:
4. E-mail:
5. Date of Purchase:
6. Proof of Purchase:

Returns should be sent to the following address:

TensCare Ltd
PainAway House,
9 Blenheim Road,
Longmead Business Park,
Epsom, Surrey
KT19 9BE, UK

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TensCare Limited
TensCare Ltd. 9 Blenheim Rd,