

# PegAssist™ Insole

## Instructions

- 

Place a transparent barrier or dressing over the wound area.
- 

With the barrier in place, circle the wound or ulcer on the patient foot with the supplied marker.
- 

Remove the black insole from the shoe.
- 

Insert the PegAssist™ Insole into the shoe with the **peg side facing down**.
- 

Have the patient step onto the insole which will transfer the marking onto the top surface of the insole. (Remove barrier)
- 

Remove the insole from the shoe. Remove pegs beneath the marking to create the off-loading area.

**Important:** Remove the pegs slowly to avoid tearing the top layer of PORON®.
- 

Remove the adhesive backing from the stabilizer board that is included in the kit.
- 

Align and attach the adhesive side of the stabilizer board to the **peg side** of the insole.
- 

Insert the PegAssist™ Insole back into the shoe with the stabilizer board facing down, blue side up. The shoe is now ready to be worn.

### Caution:

Use of this product is limited to single-patient use, under order of a physician or licensed medical professional. If you experience additional pain, swelling or skin irritation, discontinue use and contact your physician immediately.