



DNTLworks
Portable Dental Equipment

DNTLworks Equipment Corporation
7300 South Tucson Way
Centennial, CO 80112 U.S.A.
Customer Service
1-800-847-0694
1-303-693-6189
Fax: 1-303-693-6189
E-Mail: info@dntlworks.com



Caution

The air regulator is factory set at 65psi. Adjusting this knob may damage compressor system and void DNTLworks Equipment Corporation unit warranty.

Operation Instructions

Plug in power cord. Fill water containers. Turn on power switch. Attach handpieces. Depress foot control.



Adjust handpiece drive air pressure. Check water flow to handpieces by adjusting handpiece water adjustment knobs. Check air/water syringe for water, air and mist.

Check Vacuum System

To activate vacuum system, remove HVE or saliva ejector from holder. Make sure vacuum solids trap lid is secure. When using one valve, close the other for maximum performance.

If you leave the vacuum system running, but are temporarily not using a valve, be certain to leave the HVE valve open. Do not leave the waste purge switch on. When waste container is full, suction will shut off.

Waste Purge

Turn off vacuum switch. Secure waste purge tubing to unit and hold other end in a container. Turn on the waste purge switch to empty vacuum container. Turn off waste purge switch.

All authorized personal who operate, maintain, or service your ProCart III unit should carefully read the manual before attempting to operate, perform maintenance on or service the unit. Your ProCart III unit should be operated and maintained by trained personal only. Should questions or problems arise, contact our Customer Service Department.

