




Mini Trolley Products

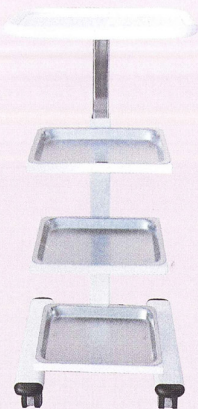

Front

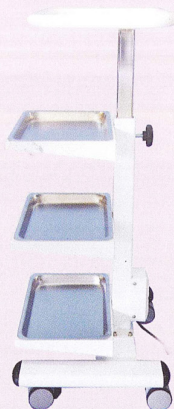

Side

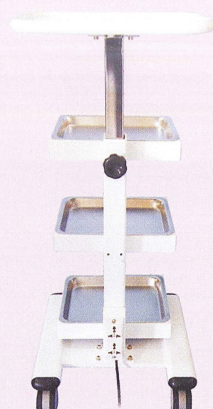

Back

Format of Mini Trolley

Size: (Lowest: 40L x 39W x 57.5H cm)
(Highest: 40L x 39W x 97.5H cm)

Input Power: 110v~220v
15 A
50~60Hz

Weight: 15KG (without additional parts)

ALL THE PICTURES, WORDS, AND ALL CONTENTS OF ASSEMBLY INSTRUCTIONS AND PRECAUTIONS CARD ALL BELONGS TO HONG HOW ENTERPRISE CO.

HONG HOW ENTERPRISE CO.

Address: 10F.-1, No.18, Sec. 1, Liuchuan E. Rd.,
South Dist., Taichung City 402, Taiwan (R.O.C.)

TEL: +886-4-23761285 +886-4-23767007

FAX: +886-4-23726117

EMAIL: mokuan@ms34.hient.net

WEB: www.honghowdental.com

Hong How
Enterprise CO.

Hong How
Enterprise CO.

Mini Trolley

(4 layers trolley)


Assembly instructions and precautions



Warning: Assembly instructions and precautions card should not use on counterfeit goods or others brand's goods. Offenders will be severely punished through legal ways.

1 · Part of product


When you receive your **Mini Trolley** (4 layers), please check if there are those items as following:

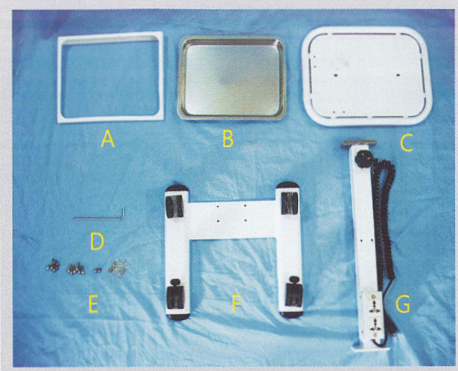

- A. Hollow shelf x3
- B. Trays x3
- C. Upper shelf x1
- D. Hex Wrench x1
- E. Gaset x14
 - Screw (short(6x6m/m) x2)
 - (medium(6x10m/m) x6)
 - (long(6x20m/m) x6)
- F. H shape base x1
(include 2 normal wheels and 2 brakes wheel)
- G. Major handle x1

2 · Assemble and install products

When you receive all parts of **Mini Trolley**(4 layers), please assemble step by step as following for your personal Mini Trolley:

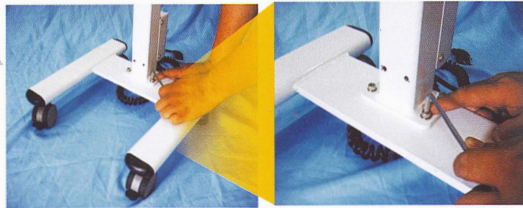


1. The H shape base should be installed together with Major handle and lock with long screws.



2. 3 Hollow shelves should be installed on the Major handle and lock with medium screws.



3. Please take out the Adjustable rod from Major handle. And lock Adjustable rod and Upper shelf with long screw tightly.

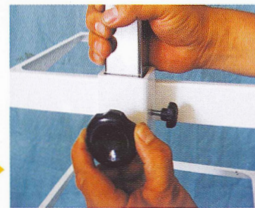

4. put the Adjustable rod into the Major Handle and lock it with screw tightly.

5. Finish.



3 · Use of product

When you finish install your personal **Mini Trolley**, please read the information as following , it could let you use Mini Trolley faster and safely.


1. Chemicals should avoid dropping on Mini trolley. If drop it carelessly, please wipe it immediately to avoid the damage.
2. Please note the bumpy surface of floor when moving the Mini Trolley, it may cause the things falling down.
3. The limited weight of load should not over 45kg. it may cause the the damage and deformation of Major handle and Adjustable rod.
4. Please DO NOT SIT on the Mini Trolley to avoid any danger.
5. When adjusting the height of Adjustable rod, first please hold the Upper shelf or Adjustable rod to avoid descending rapidly and damage.

4 · Purchase assist products


After you use the **Mini Trolley**(4 layers), if you need more assisting tools. You could refer to our assist products. It will give you easier and more convenient during the tremens.



Handpieces holder

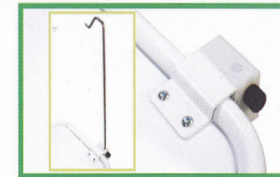

Irrigation Bag rod holder

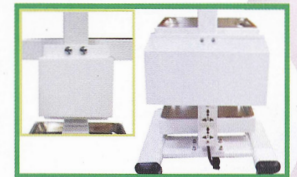

Charger and pedal holder box

Large:90*300*140mm
Small:60*130*110mm



Install ways :

When Handpieces holder put on the Upper shelf, please lock it from the back side of Upper shelf with screws.



Install ways :

When bag rod holder rod put on the Upper shelf, please lock it from the back side of Upper shelf with screws.



Install ways :

Please put the holder box on the top of plugger in the Major handle, then please lock it with short screw tightly.