
















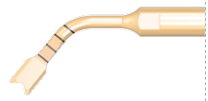












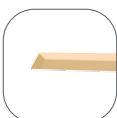




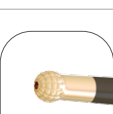




PIEZO TIP REFERENCE GUIDE

* Some of the following tips may not be included in the Piezo Tip Kit
 * The tip combination on each basic kit for the models of SUS10 and SUS20 can be varied according to the specific needs by countries

KIT AFFILIATED		USE	BOOST	POWER	PUMP
OT Osteotomy		 SP-SAW Principal micro-saw / Bone block grafting / Ridge expansion	3	3	5
		 SP-SAW L Angled micro-saw / Bone block grafting	3	3	5
		 SP-SAW R Angled micro-saw / Bone block grafting	3	3	5
		 SP-RS1 Ridge split	3	3	5
		 SP-SAW RD Rounded micro-saw, Bone block grafting	3	3	5
		 SP-SAW BS Powerful bone surgery, Bone block grafting	3	3	5
OP Osteoplasty		 SP-100 Scraper / Bone remodeling and harvesting of bone chips	3	3	1
		 SP-320 Principal osteoplasty insert	3	3	1
		 SP-321 Principal osteoplasty insert	3	3	1
EX Extraction		 SP-610 Principal extraction scalpel	0	3	5

PIEZO TIP REFERENCE GUIDE

* Some of the following tips may not be included in the Piezo Tip Kit
 * The tip combination on each basic kit for the models of SUS10 and SUS20 can be varied according to the specific needs by countries

KIT AFFILIATED		USE	BOOST	POWER	PUMP
SL Sinus Lift		 SP-BT2 Osteoplasty and osteotomy insert	3	3	3
		 SP-BT3 Osteoplasty and osteotomy insert	3	3	3
		 SP-400 Sinus membrane separator	0	3	1
		 SP-401D Sinus membrane separator	0	3	1
		 SP-ST1 Sinus bony window osteotomy insert	3	3	3
IM Implant Preparation		 SP-016 Implant site preparation insert	3	3	5
		 SP-020is Implant site preparation insert	3	3	5
		 SP-028i Implant site preparation insert	3	3	5
		 SP-028i 4H Implant site preparation insert with 4 water spray halls	3	3	5



- The tips are not sterilized before packing. Please sterilize it before usage.
- Do not allow the Tip to touch a patient's tongue, lips or cheeks. It could burn the patient.