




AVTEC DENTAL


PROFESSIONAL HANDPIECE REPAIR

Post this Side in your Sterilization Room.

How to use *MK-dent* Lubricant

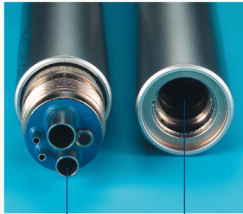

Lube here

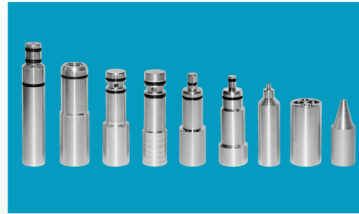

Lubrication Tools



1

Spray

Using the correct adapter, spray MK-dent spray into the drive line or the hole. Spray for approx. 2 seconds.
Count 1..... 2.....



Once a day the spindle of the handpiece has to be lubricated

2

Run 5 to 10 seconds

Run the handpiece to expel out excess lubricant. This should be done at the control unit in the operatory or at a purge station in the sterilization room.
Count 1..... 2 3..... 4..... 5.....



3

Wipe with alcohol to remove excess lubricant

Wipe down the exterior of the handpiece with a 2x2 lightly dampened with alcohol. Use pure rubbing alcohol only, many over-the-counter "alcohol wipes" have other solutions added to them, this can cause spotting of the exterior and severe damage to the internal parts of the handpiece.



4

Bag and Autoclave

Sterilize handpiece as per manufacturer's instructions. Always allow the handpiece to go through the "Dry Cycle" on your sterilizer. Note: We recommend using sterilization bags with one side paper and the other side plastic.



5

Let Cool

Remove the handpiece immediately from the sterilizer to prevent any risk of contact corrosion and let it cool to room temperature. Running "Hot" handpieces will severely reduce the life of the turbine.
Don't Re-Lubricate - With MK-dent Spray You Lubricate Only Once!



Caution: *Do not place handpieces in disinfectant baths!
*Never submerge in water!
*Do not place handpieces in ultrasonic cleaners!