Merula®

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IMPORTANT INFORMATION

Please only use the Merula Cup XL after having read and understood the instructions. Keep these instructions safe. These instructions are not intended to be a substitute for medical advice. Always seek medical advice in case of uncertainties prior, during or after the use of the Merula Cup XL. The Merula Cup XL is a hygiene product and cannot be returned or exchanged. All sales are final.

We reserve the right to make modifications to the instructions without notice. The most current version of the Merula Cup XL instructions can always be downloaded on our homepage www.merula.eu.

Inserting & removing

Wash your hands thoroughly before inserting or removing the Merula Cup XL. There are various ways of folding your cup. We recommend the **punchdown fold**. Press the upper rim of the Merula Cup XL on one side to the bottom of the cup and fold it. This way, it is narrow and pointy and can be easily inserted.

Be in a comfortable position and relax when inserting the cup. This could be sitting on the toilet, while squatting or while standing with one leg bent on the rim of the bathtub.

You can use some water or a water-based lubricant (e.g. *Merula Lube*) for easier insertion. Insert the folded cup and position it so that the cervix is within the cup (see image page 2). Make sure that the upper rim of the cup opens up completely. The body of the cup can remain squished. Check with one finger, whether your cervix is within the cup and surrounded by the upper rim.

In order to remove the cup, put one finger into the ladder and break the seal with a second finger by pushing on the body of the cup. Once the vacuum is released, the cup can be removed and then cleaned. The menstrual cup should be emptied and cleaned at the latest after 8 hours. You can certainly change your menstrual cup before then, if you need to.

Assistance:

The right position of your cup is individual. It may take a few cycles until you have found the perfect position of the Merula Cup XL, even if you have already had experience with other menstrual cups.

If your Merula Cup XL is leaking or causes an uncomfortable feeling, it is not positioned correctly. Remove it immediately and position it again so that the cervix is placed within the cup (see image on page 2).

Should you have difficulties reaching the body of the cup during the removal, you can make gentle zigzag movements of the ladder, moving it downwards. It can also be helpful to push the cup down slightly with pelvic floor movements.

Cleaning

Prior to the first use and after every period, the Merula Cup XL should be boiled for 5 minutes in water (if required with a couple of drops of vinegar). You can either do this in a pot on the stove or in a container in the microwave (e.g. with help of the *Merula Cupscup*).

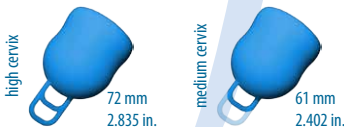
The menstrual cup should be thoroughly cleaned after every use (at the latest every 8 hours). Empty the content of the Merula Cup XL into the toilet, get rid of blood residue by either rubbing it off or by washing the cup with cold water and then disinfect the cup by either boiling it or by using *Merula Spray*, *Merula Wipes* or any other adequate cleaning products with disinfecting effect that can be used for intimate products made from silicone.

The *Merula Wipes* and the *Merula Spray* are particularly practical when you are on the go. You can also thoroughly rinse and clean your cup with water from a bottle you have brought along. Make sure that there is never more than 8 hours between two thorough cleanings.

Always keep your Merula Cup XL in an airy container, e.g. the provided pouch when not in use. You can use your cup for several years if optimally cared for. Slight discolorations over time are normal.

Adjusting

You can adjust the stem of the Merula Cup XL to the perfect length for you. Try out the cup several times in order to discover which length is right for you.

The cup and the stem should be completely within your body, but you should be able to reach it with one finger. Remove one or two of the marked segments with nail scissors (not while in use within your body!). We strongly recommend not completely removing the stem. Thoroughly clean the Merula Cup XL after you have shortened it.

If you want to find out more about the use of the Merula Cup XL, please visit our website at www.merula.eu or contact us.

What is so special about the Merula Cup XL?

- ★ The Merula Cup XL has a large capacity due to its spherical shape (up to 50 ml / 1.691 fl oz \approx 2.5 super plus tampons), and it is, therefore, the ideal companion during extra heavy flows.
- ★ Only the upper rim must open up, as the vacuum is created here. This is why the body of the Merula Cup XL may remain squished and it fits well in a strong and in a weak pelvic floor.
- ★ The soft stem can be easily shortened to the desired length.
- ★ The Merula Cup XL is best suited for a medium or high cervix. If your cervix is low, or you feel your vagina is particularly 'tight', the Merula Cup XL may be too large. Then we recommend the Merula Cup, 'one size' with a capacity of 38 ml / 1.285 fl oz.

Safety instructions & disposal

- ★ Always wash your hands thoroughly before and after removing your cup.
- ★ Only wear your cup during your period and not during your puerperium.
- ★ Always release the vacuum before removing your cup.
- ★ Clean your cup after 8 hours at the latest and boil it at least once during your cycle, prior to the beginning of the next period.
- ★ Always keep your cup out of reach of children and pets.
- ★ Remove your cup should you feel any pain.
- ★ Remove your cup prior to sexual intercourse because there is a risk of injury. The Merula Cup XL is not intended for contraception nor does it prevent sexually transmittable diseases.
- ★ Seek medical advice prior to using menstrual cups if you have an IUD (intrauterine pessarine) (coil/chain/ball/...). Should the thread of your IUD appear longer, it is possible that it has shifted.
- ★ Please consult a physician if the pain persists after removal of your cup.
- ★ If your cup starts to show signs of material fatigue, please dispose of it in the general waste bin and buy a new one. Please consider exchanging your cup after 3-5 years.
- ★ **Menstrual toxic shock syndrome (TSS)**

TSS is a rare, but serious condition that can occur in every type of person. There are hardly any known cases of TSS that can be connected to menstrual cups, however: it can lead to death in rare cases.

The following symptoms that develop quickly and are initially similar to the flu, may be first signs of the TSS disease: Sudden onset of high fever ($>39^{\circ}\text{C}$ / $>102^{\circ}\text{F}$), emesis, diarrhea, skin rashes similar to sunburns, sore throat, vertigo and/or syncopes.

Should you suffer from one or more symptoms during your period, remove the cup and seek immediate medical advice. Mention TSS and that you are menstruating. This way, the condition can be detected early and effectively treated. Seek medical advice if you have TSS prior to the continued use of cups or tampons.

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