

Simple operation | Efficient production |
Bionic aesthetics

The all-porcelain aesthetic of choice



■ Everyone with a healthy and beautiful smile

[Color and application]

*The color of printed matter is for reference only, and the color of a real object shall prevail.

Color	Picture	Product coding	Color	Picture	Product coding
A Shade		ACZStain-A	Blue 1		ACZStain1-Blue
B Shade		ACZStain-B	Blue 2		ACZStain2-Blue
C Shade		ACZStain-C	Purple grey		ACZStain-Purplegrey
D Shade		ACZStain-D	Black		ACZStain-Black
Orange 1		ACZStain1-Orange	White		ACZStain-White
Orange 2		ACZStain2-Orange	White C		ACZStainC-White
Yellow		ACZStain-Yellow	Terracotta		ACZStain-Terracotta
Pink		ACZStainR-Pink	Brown		ACZStain-Brown

[Staining operation]

Standardized staining operation:

1. The crown part is painted with transparent glaze as a whole.
2. The cervical is painted with the main colors such as A shade.
3. Use blue to create transparency on incisal 1/3.
4. Sintered according to the suggested sintering curve.

Personalized staining operation:

1. The crown part is painted with transparent glaze as a whole.
2. The cervical is painted with the main colors (such as A shade), terracotta and a little brown.
3. Use orange 2 on the upper 1/3 of the incisal edge to build mamelon finger structure.
4. Blue 1, blue 2 or a small amount of black shall be used for 1/3 of the incisal to enhance the transparency.
5. Use orange 1 at the incisal edge to create the halo effect.
6. Sintered according to the suggested sintering curve.



[Final effect]



Aidite[®]