

Aidite Masking Liquid Instruction

[Product Application]

Function: brushed in inner side of zirconia restoration to block out dark abutment, metal posts or implant.

[Application]

Discolored tooth, devital tooth, tetracycline pigmentation teeth, metal posts, implant superstructure.

[Material applied in]

All zirconia products

Note: advised to be used in areas with thickness above 0.8mm

[Preparation]

Tools: 0 size brush, plastic container, oven

Products:Aidite indicator,masking liquid

[Instruction]

Brushing

Consumables:Indicator

Function: make masking liquid show color, used to indicate and mark when brushing. Indicator has no impact on final coloring effect.

Steps:

Mix masking liquid and indicator(mix method:squeeze suitable volume of masking liquid in container , then use tip of 0 size brush to dip a little indicator to mix with masking liquid and stir evenly.)
Use 0 size brush to dip masking liquid and brush one time in crown innerside.



ATTENTION !

Note:

- Not too much liquid, use brush tip to dip liquid then scrape in the bottle opening a little before brushing.
- Do not brush too much, especially margin or thin area to prevent over permeation of masking liquid and therefore cause opacious surface.
- The product must be used up within 90 days after being opened. The liquid should not be exposed to the air for a long time. After use, the bottle mouth should be sealed completely.

Note:

White zirconia operation:

First dip crown following standard dipping instruction, then use oven to dry(dry temperature 150°C, drying time 20 minutes), after drying, follow above steps to brush masking liquid.