
VITAMIN D

Vitamin D refers to a group of fat-soluble secosteroids responsible for increasing intestinal absorption of calcium, iron, phosphorus, magnesium, phosphate and zinc.

The Vitamin D test is an immunoassay based on the principle of competitive binding. During testing, the mixture migrates upward on the membrane chromatographically by capillary action. The membrane is pre-coated with 25 (OH) D antigens on the test line region of the strip.

CONTENT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

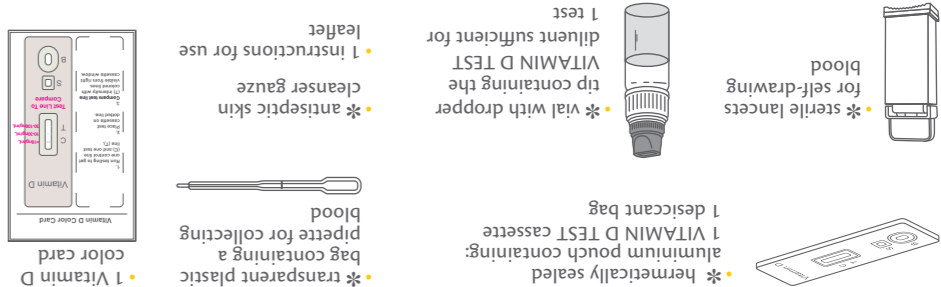LIMITATIONS

- 1) The VITAMIN D TEST provides only a semi-quantitative analytical result. A secondary analytical method must be used to confirm the result.
- 2) It is possible that technical or procedural errors, as well as other interfering substances in the whole blood specimen may cause erroneous results.
- 3) As with all diagnostic tests, all results must be considered with other clinical information available to the physician.
- 4) Other clinically available tests are required if questionable results are obtained.

PRECAUTIONS

1. For self-testing in vitro diagnostic use only.
2. Do not eat, drink or smoke in the area where the specimens or kits are handled.
3. Store in a dry place at 2-30°C (36-86°F), avoiding areas of excess moisture. If the foil packaging is damaged or has been opened, please do not use. The test is stable through the expiration date printed on the sealed pouch.
4. This test kit is intended to be used as a preliminary test only and repeatedly abnormal results should be discussed with doctor or medical professional.
5. Follow the indicated time strictly, using only the specified quantities of blood and diluent.
6. Use the test only once. Do not dismantle and touch the test window of the test cassette.
7. The kit must not be frozen or used after the expiration date printed on the package.
8. Keep out of the reach of children.
9. The test is not recommended for people who take anti-coagulant medications (blood thinners) or people suffering from haemophilia.
11. After using, please dispose of all components according to your local waste disposal laws. Ask your pharmacist for advice.
12. If you test another person, handle the blood samples as if containing a blood-borne infectious agent. Wear disposable gloves during use.

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BIBLIOGRAPHY / BIBLIOGRAPHIE / BIBLIOGRAPHIE / BIBLIOGRAPHIE

1. Holick MF (March 2006). "High prevalence of vitamin D inadequacy and implications for health." *Mayo Clinic Proceedings*. 81 (3): 353–73.
2. Eriksen EF, Glerup H (2002). Vitamin D deficiency and aging: implications for general health and osteoporosis. *Biogerontology*. 3 (1-2): 73-7.
3. Grant WB, Holick MF (June 2005). Benefits and requirements of vitamin D for optimal health: a review. *Alternative Medicine Review*.10 (2): 94-111.
4. Moyad MA. Vitamin D: a rapid review. *DermatolNurs*. 2009, 21:25-30.

antiseptic cleanser gauze: 70% Alcohol
Gaze zur antiseptischen Reinigung der Haut: 70% Alcohol
compresse pour un nettoyage antiseptique : 70% Alcool
garza detergente antisettica: 70% Alcol

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LEGAL MANUFACTURER
Legal manufacturer
Hersteller
Fabricant
Fabbricante

STERILE R Sterilised using irradiation
Sterilisation durch Bestrahlung
Méthode de stérilisation utilisant l'irradiation
Sterilizzato mediante radiazioni

LOT Lot number
Chargenbezeichnung
Code du lot
Lotto Prodotto

TEMPERATURE LIMITS
Temperature limits
Temperaturbegrenzung
Limites de température
Limiti di temperatura

DO NOT REUSE
Do not reuse
Nicht wiederverwenden
Ne pas réutiliser
Non riutilizzare

AUTHORISED REPRESENTATIVE
Authorised Representative in the European Community
Bevollmächtigter in der Europäischen Gemeinschaft
Mandatario dans la Communauté européenne
Mandatario nella Comunità Europea

Test your health

PRIMA LAB SA
Via Antonio Monti 7
CH-6828 Balerna - SWITZERLAND

QARAD EC-REP BV
Pas 257
2440 Geel - Belgium

+41 61 61 605 1030
support@primallabsa.ch
primallabsa.ch

100066

100066_IFU_11_5_0_03/2022

Registered trademarks.

ENGLISH

DIRECTIONS FOR USE

- 1) Wash hands with soap and warm water, rinse with clean water and allow to dry. If this is not possible, use the provided gauze as an alternative.
2) Prepare the necessary material as follows: open the aluminium pouch, take out only the test cassette and throw away the desiccant bag.
3) Carefully rotate the protective cap of the sterile lancet 360° without pulling it.
4) Carefully massage the finger chosen for the puncture...
5) Holding the hand down, massage the finger until a large drop of blood forms.
6) Take the pipette without pressing the bulb.
7) Deposit the blood collected into the Specimen well (S) of the cassette...
8) Wait for the blood to be totally dispensed in the well.
9) Wait for the colored line(s) to appear. Read results at 10 minutes.

RESULTS INTERPRETATION

Please refer to the illustration and compare the T line intensity with "Vitamin D color card" provided with the kit. As control system, a colored line will always appear in the control line region indicating that proper volume of specimen has been added and membrane wicking has occurred.

DEFICIENT 0-10 (ng/ml) 0-25 (nmol/l)
INSUFFICIENT 10-30 (ng/ml) 25-75 (nmol/l)
SUFFICIENT 30-100 (ng/ml) 75-250 (nmol/l)
EXCESS >100 (ng/ml) >250 (nmol/l)
INVALID

FA.Q. - QUESTIONS AND ANSWERS

HOW DOES THE VITAMIN D TEST WORK? In medicine, a 25-hydroxy vitamin D is the main storage form of vitamin D in the body. Therefore, the overall status of vitamin D can be determined by detecting the content of 25-hydroxy vitamin D.
WHEN SHOULD THE TEST BE USED? The clinical application of vitamin D is mainly for diagnosis, treatment and monitoring of rickets (children), osteomalacia, postmenopausal osteoporosis and renal osteopathy.
CAN THE RESULT BE INCORRECT? The results are accurate as far as the instructions are carefully respected.

HOW TO INTERPRET THE TEST IF THE COLOR AND THE INTENSITY OF THE LINES ARE DIFFERENT? Please refer to the illustration and compare the T line intensity with "Vitamin D Color card" provided with the kit.
WHAT DO I HAVE TO DO IF THE RESULT IS DEFICIENT OR INSUFFICIENT? If the result is deficient or insufficient, it means that the vitamin D level in blood is less than 30ng/mL and that you should consult a physician to show the test result.
WHAT DO I HAVE TO DO IF THE RESULT IS SUFFICIENT? If the result is sufficient, it means that the vitamin D level is higher than or equal to 30ng/mL and is within the normal range.

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