Test your health

PRIMA LAB SA
Via Antonio Monti 7
CH-6828 Balerna - SWITZERLAND
+41 91 605 1030
support@primallabs.ch
primallabs.ch

EC REP Qarad EC-REP BV
Pas 257
2440 Geel - Belgium

800040_IFU_11_5.0 04/2022 **REF** 800040

Registered trademarks.

INSTRUCTIONS FOR USE GEBRAUCHSANWEISUNG NOTICE D'UTILISATION ISTRUZIONI PER L'USO

BOWEL FOB TEST

Self-test for the detection of occult blood in the faeces
Selbsttest zur Bestimmung von okkultem Blut im Stuhl
Auto-test pour la détermination du sang occulte dans les selles
Self-test per la determinazione del sangue occulto nelle feci

ENGLISH FOB (FECAL OCCULT BLOOD)

Faecal occult blood is the presence in the faeces (stool) of blood that is not clearly visible to the naked eye. Occult blood may be the consequence of gastro-intestinal disorders such as ulcers, polyps, colitis, diverticulitis, rhagades, haemorrhoids or cancer. The signs of these lesions are often silent in the first phases, so searching for occult blood in the stool is an important screening test, which allows to identify the presence of pathologies in the gastro-intestinal system at an early stage.

PRINCIPLES OF THE TEST

BOWEL FOB TEST is an immunochromatographic test that detects the smallest amounts of blood in stool samples by using highly specific monoclonal antibodies. These properties mean that the BOWEL FOB TEST does not require particular dietary restrictions before being carried out.

BIBLIOGRAPHIE


1. Van Rossum Leo G. et al.: "Random Comparison of Guaiac and Immunochemical Fecal Occult Blood Tests for Colorectal Cancer in a Screening Population" Gastroenterology, Volume 135, Issue 1, 82 – 90.
2. Faivre J, Dancourt V, Denis B, Dorval E, Piette C, Perrin P, Bidan JM, Jard C, Jung S, Levillain R, Viguier J, Bretagne J.F.: "Comparison between a guaiac and three immunochemical faecal occult blood tests in screening for colorectal cancer". European Journal of Cancer. 2012; 48(16):2969-76.
3. Benton S.C., Seaman H.E., Halloran S.P.: "Faecal Occult Blood Testing for Colorectal Cancer Screening: the past or the Future". Current Gastroenterology Reports. 2015; 17(2):428
4. European Commission. European guidelines for quality assurance in colorectal cancer screening and diagnosis. First edition 2010.

CONTENT

* The number of the testing devices of the kit may vary. For the exact number of tests contained, please refer to the "content" section on the external box.

- * hermetically sealed aluminium pouch containing:
1 BOWEL FOB TEST cassette
1 desiccant bag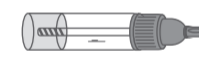

- * vial with red screw cap, with collection stick and dripper, containing enough BOWEL FOB TEST diluent for 1 test

- * small sheet for the collection of the faeces sample

- * 1 instructions for use leaflet

* Material required but not supplied: cotton wool, a device to measure time (i.e. timer, watch).
* Do not open the sealed aluminium pouch until just before performing the test. Take care to open it as marked.
* The desiccant bag must not be used. Dispose of it with household waste without opening it.

PRECAUTIONS

1. Read these instructions for use carefully before performing the test. The Test is reliable only if all the instructions are followed correctly.
2. Keep the Test out of the reach of children.
3. Do not use the Test after the expiry date or if the package has been damaged.
4. Store the Test components at a temperature between +4°C and +30°C. Do not freeze.
5. Use the test once only.
6. The test is for external use only. DO NOT SWALLOW.
7. In vitro diagnostic device for individual use.
8. After using, please dispose of all components according to your local waste disposal laws. Ask your pharmacist for advice.

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