

Set Up Guide

GET READY TO BREATHE EASIER

AIRWHEEL SMART RESPIRATOR FRESH AIR MASK – SET UP GUIDE

INSTALLING THE FILTER – Insert a coin in the Release Slot and twist to open the **Mask front shell**. Use tweezers provided to insert a new filter, light or dark side either in or out. Once the filter is in place, snap the **Mask front shell** and **Mask rear shell** back together.

INSTALLING THE FRESH AIR ELECTRIC MODULE – Insert the **Fresh Air Electric Module** into the **Mask front shell**. Rotate the **Fresh air electric module** $\frac{1}{4}$ turn clockwise to slide into place.

TURNING ON THE FRESH AIR ELECTRIC MODULE - Press the **Power switch** 1 time to power on (high) then press again for low wind speed.

CHARGING THE FRESH AIR ELECTRIC MODULE – Plug the USB cable provided into a USB charging port. The **Work indicator** light will flash blue slowly, a full charge takes 60 minutes.

- The **Fresh air electric module** uses a lithium battery for a fast charging speed and does not need to be fully drained before recharging.
- A full charge lasts 4 hours (monitored by your APP). If the mask is required to work longer, make sure to charge the **Fresh air electric module** during work breaks or consider purchasing an extra module.

INSTALLING STRAPS – Each kit contains 2 sets of silicone straps. Attach the straps to the **mask band buckles** choosing the appropriate adjustment holes for your personal wearing comfort and style, behind the ears or head.

- Silicone strap excess ends can be stretched and hooked to secure onto an alternate band buckle.
- The plastic hood provided can join the straps behind your head for a secure fit.

CLEANING YOUR MASK – Remove the **Fresh air electric module** and **filter** if choosing the boiling method as these cannot be immersed in water. Disassemble the remaining pieces and put in a pot of boiling water for 5 minutes to kill 99.9% of bacteria.

- Once cleaning is complete – attach the vented or non-vented face shield to the rear shell (the non-vented face shield is recommended for Covid-19 protection).
- Replace the filter and snap the **Mask front shell** to the **Mask rear shell**.
- Install the **Fresh air electric module** and attach the **Straps**.

INSTALLING THE AIRWHEEL APP ON YOUR MOBILE DEVICE – Download the APP using your camera over the QR code, or download from the APP store – **Airwheel Free Intelligent Life**.

- Open the APP – Choose “Intelligent Mask”, click the magnifying glass in the APP to find the serial number for your mask and select, (turn on the Fresh Air Electric Module manually before connecting the mask to the APP for the first time – only ON masks will be listed to connect).
- Label your mask in the APP – go to settings and select “change device name”. This will let you easily identify your mask if surrounded by multiple wearers.
- The timer in the APP counts how long the filter has been in use. When the timer reaches 8 hours, you will get notification showing “recommend replace filter element”; when the timer reaches 14 hours, the notification is “please replace filter element”. Filters should be replaced more or less often based on the usage environment.
- Once a new filter is inserted, go to settings in the APP, select resume to default, then select confirm – this will reset the smartAPP’s timer to “0” and restart the new filter life to 8 and 14 hours.

NOW THAT SET UP IS COMPLETE

TAKE A BREATH OF FRESH AIR