SADT2 - SAD Light

Contents

English Manual	2
Manuel d'utilisation en français	15
Gebrauchsanleitung auf Deutsch	27
Manual de Instrucciones en Español	38
Manuale D'istruzioni In Italiano	50
Instrukcja obsługi. Polski	62

English Manual

About the SAD Light

Instructions for Use

Congratulations on the purchase of your new “SAD Light”. The product will offer many years of reliable usage if handled correctly. Please keep your receipt, these instructions and the original packaging safe in case of future requirement. Please read the instructions carefully to gain the most benefit from your new “SAD Light”, although the usage guidelines in this document do not replace the advice of your qualified medical advisor.

Intended Use:

Medical intended use of this appliance is to treat the symptoms of SAD (seasonal affective disorder) or winter depression. “SAD Light” is intended to boost people’s energy levels, to adjust circadian rhythms and make people feel more awake. “SAD Light” is safe and medically proven to provide relief from the symptoms of SAD.

How does the “SAD Light” work?

In autumn and winter, the seasons with the least sunlight because of shorter days, the lack of sunlight and a problem with certain brain chemicals stops the hypothalamus working properly. The lack of light is thought to affect the production of the hormone melatonin (that makes you sleepy).

“SAD Light” uses bright lights to simulate the sunlight during the darker autumn and winter, whilst eliminating the dangerous UV rays contained in sunlight. The additional light helps adjust the levels of melatonin that can help the symptoms of SAD.

Seasonal Affective Disorder:

Seasonal Affective Disorder (SAD) as known as winter depression or winter blues. It may cause people to:

- Feel low, restless and fatigued
- Have trouble concentrating
- Lack energy
- Need more sleep
- Less active than normal

Symptoms occur during the same season each year.

Most people with SAD start to have symptoms during winter, and feel better in the summer.

Contradiction

The following conditions may be in contradiction for use of this appliance:

- suffer(ed) from severe depression
- recent eye surgery or a diagnosed eye condition whereby doctor has advised you to avoid bright light
- on specific medication (i.e. certain antidepressants, psychotropic drugs or malaria tablets);
- have high blood pressure with vitreous hemorrhage;
- skin sensitive to light, such as systemic lupus erythematosus

Always consult your doctor before your start using this appliance if you have any of the above conditions.

Possible Side effects:

- | | |
|---------------------|-----------|
| • Headaches | Eyestrain |
| • Sleeping Problems | Nausea |

Measures to avoid, minimize or alleviate these side effects:

- Increase the treatment distance
- Avoid using the appliance during the late evening or before sleep
- Stop using the appliance for a few days to make the side effects disappear and then try again.

Safety and performance standards

The “SAD Light” has been designed and manufactured to conform to the highest standards of safety and performance including electromagnetic compatibility (EMC).

The “SAD Light” conforms to the applicable requirements of the following:

This appliance meets the requirement of Medical Device Directive 93/42/EEC for medical appliances.

Emissions Test Standard

RF emissions CISPR 11

RF emissions CISPR 11

Harmonic emissions IEC 61000-3-2

Voltage fluctuations/ flicker emissions IEC 61000-3-3

Compliance

Group 1

Class B


Class A

Comply

Guidance and manufacturer's declaration - electromagnetic Immunity							
Radiated RF IEC61000-4-3 (Test specification for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Modulation (W)	Distance (m)	Immunity Test Level (V/m)
	385	380 - 390	TETRA 400	Pulse modulation 18Hz	1.8	0.3	27
	450	430 - 470	GMRS 460 FRS 460	FM +5kHz deviation 1kHz sine	2	0.3	28
	710 745 780	704 - 787	LTE Band 13, 17	Pulse modulation 217Hz	0.2	0.3	9
	810 870 930	800 - 960	GSM 800/900 TETRA 800, iDEN 820 CDMA 850 LTE Band 5	Pulse modulation 18Hz	2	0.3	28
	1720 1845 1970	1700 - 1990	GSM 1800 CDMA 1900 GSM 1900 DECT; LTE Band 1 3,4,25 UMTS	Pulse modulation 217Hz	2	0.3	28
	2450	2400 - 2570	Bluetooth WLAN 802.11 b/g/n RFID2450 LTE Band 7	Pulse modulation 217Hz	2	0.3	28
	5240 5240 5785	5100 - 5800	WLAN 802.11 a/n	Pulse modulation 217Hz	0.2	0.3	9

Immunity Test Standard	IEC 60601-1-2 Test Level	Compliance Level
Electrostatic discharge (ESD) IEC 61000-4-2	± 8kV contact ± 2kV, ± 4kV, ± 8kV, ± 15kV air	± 8kV contact ± 2kV, ± 4kV, ± 8kV, ± 15kV air
Electrical fast transient/burst IEC 61000-4-4	± 2kV, for power supply lines ± 1kV signal input/ output 100 kHz repetition frequency	± 2kV, for power supply lines Not applicable 100 kHz repetition frequency
Surge IEC 61000-4-5	± 0.5kV, ± 1kV differential lines ± 0.5kV, ± 1kV, ± 1kV common mode	± 0.5kV, ± 1kV differential lines Not Applicable
Voltage dips, interruptions and variations on power supply input lines IEC 61000-4-11	0 % UT; 0.5 cycle. At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°. 0 % UT; 1 cycle and 70 % UT; 25/30 cycles; Single phase: at 0°. 0 % UT; 250/300 cycle	0 % UT; 0.5 cycle. At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°. 0 % UT; 1 cycle and 70 % UT; 25/30 cycles; Single phase: at 0°. 0 % UT; 250/300 cycle
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz
Conducted RF IEC 61000-4-6	3 V 0,15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	3 V 0,15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz
Radiated RF IEC 61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz

The following symbols appear in this manual, on the “SAD Light” or on its accessories:

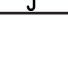	Warning instruction indicating a risk of injury or damage to health		Read operating instructions before use
	Safety note indicating possible damage to the appliance/ acces-		Caution: Consult accompanying documents
	Specifies serial number of the “Daylight”		Class II Equipment
O / I	OFF/ ON		Date of Manufacture
EC/REP	Authorized Representative in the European Community		Manufacturer
IP21	Ingress Protection Rating		Keep Dry

Notes

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.


- Do not use this appliance in wet surroundings (e.g. in the bathroom or near a shower). Do not immerse appliance in water or any other liquid.
- Before use, ensure that all packaging materials are removed and that there is no visible damage to the appliance or accessories.
- When in doubt, do not use the appliance and contact your dealer or the customer service address provided.
- The adaptor contains a transformer. Do not cut off the adaptor to replace it with another plug, as this can cause a hazardous situation.
- This appliance is only intended for use on the human body.
- Close supervision is necessary when this appliance is used by, or near children or invalids.
- Children should be supervised to ensure that they do not play with the appliance.
- There is no need for any special training for this appliance.
- Keep the cord away from hot objects and open flames.
- Connect the appliance only to the mains voltage listed on the type plate.
- Always remove the mains plug and let the appliance cool off before you touch it.
- Do not touch the appliance with wet hands while it is plugged in; do not allow any water to be sprayed onto it.
- The appliance must be operated only when it is completely dry.
- Protect the appliance from strong impacts.
- Avoid pulling the cord when removing the plug from the socket. Instead, hold the plug.
- If the adaptor, cord or the appliance is damaged, contact customer service or the dealer, since special tools are needed to make repairs.
- Never try to repair the appliance yourself.
- Disconnection from the power supply network is only guaranteed when the mains part is unplugged.
- This appliance is not suitable for use in the presence of flammable anesthetic mixtures with air or with oxygen or nitrous oxide.
- This appliance requires no calibration, preventive inspection and maintenance.
- This appliance is not repairable and contains no user serviceable parts.

- The operator shall not touch the appliance and the patient simultaneously.
- The appliance is protected against unauthorized use.
- Do not modify this appliance without authorization of the manufacturer.
- If this appliance is modified, appropriate inspection and testing must be conducted to ensure continued safe use of the equipment.
- This appliance is not intended for commercial or clinical use, but only for individual, private household use.
- If you are taking medication such as pain relievers, antihypertensives or antidepressants, consult your doctor before using this appliance.
- People with retinal diseases as well as diabetics need to be examined by an ophthalmologist before using this appliance.
- Please do not use if you have an eye disease such as cataracts, glaucoma, diseases of the optic nerve in general and inflammation of the vitreous humour.
- If you have health concerns of any kind, contact your general practitioner.
- Use of this appliance adjacent to or stacked with other device should be avoided because it could result in improper operation. If such use is necessary, this appliance and the other device should be observed to verify that they are operating normally. If the appliance does not light on or flickering, it means that the appliance is not operating normally.
- Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.
- Do not change any cable without authorization of the manufacturer. Use of cables other than those specified or provided by the manufacturer of this appliance could result in increased electromagnetic emissions or decreased electromagnetic immunity of this appliance and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm to any part of this appliance, including power cable. Otherwise, degradation of the performance of this equipment could result.
- If the appliance does not light on or flickering due to the electromagnetic disturbances, do not use it and try to place the appliance in other location.
- For maintaining the basic safety and essential performance with regard to electromagnetic disturbances for the expected service life, please precaution to take if the use location is near (e.g. less than 1,5 km from) from AM, FM or TV broadcast antennas.

Maintenance and Repairs

- Avoid exposing the screen to direct sunlight, spotlight or any other heat source.
- Do not use alcohol or ammonia base liquid to clean the screen. The cleaning information provided in the section “Cleaning and Storage”.
- Please do not attempt to repair the appliance yourself. This could result in serious injuries. Failure to observe this regulation shall void the warranty.
- If this appliance is damaged in any way or it is not working, it should not be operated. Please try to fix the problem according to FAQ table.
- If the problem is still not resolved, please contact customer service or the dealer, since special tools are needed to make repairs. Never try to repair the appliance yourself.

Unit Description

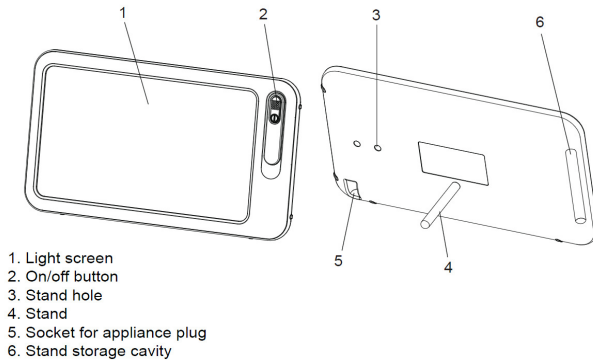Installation

1. Remove the stand from the stand storage cavity.
 2. Insert the end of stand into the hole on the back of appliance.
 3. Place the appliance on a solid surface approximately 20 - 30cm from your eyes, the light must bathe your face from the side.
 4. Connect the appliance only to the mains voltage listed on the type plate.
- Push the plug completely into the mains socket.
 - Ensure that there is a socket near the place where the appliance will stand.
 - Lay the mains cable in such a way that no one can trip over it.

Operation

Before use, please check the appliance for damage or faults. If you notice any damage or faults on the appliance, do not use it and contact customer services or your supplier.

1. Turning on

Press the ON/OFF button once of the appliance to turn on with higher intensity.

2. Choosing the light intensity

To choose a lower intensity, press the ON/OFF button again.

3. Taking the light

You can read, write, make telephone calls or work while you use the appliance. When using the appliance, try to maintain the recommended distance of 20 - 30 cm between your eyes and the appliance.

Usage guidelines are as follows:

Distance	Duration of higher intensity	Duration of lower intensity
30cm	Approx. 2 hours	/
20cm	Approx. 1 hour	Approx. 2 hours

In effect, the closer you are to the source of light, the less time you need to use it.

- Do not look directly into the light continuously! Some eye pain or mild headaches may be experienced during initial sessions but these should pass. If in doubt just lower your treatment times until comfortable again.
- Do not leave the appliance unattended in the duration.

4. Enjoying light over longer periods

During the darkest period of the year, repeat the treatment for at least 7 consecutive days, or longer, depending on your individual requirements.

5. Switching off

Press the ON/OFF button of the appliance to switch off.

Cleaning and Storage

- Before you clean the appliance, always switch it off,
- unplug it and let it cool off.
- Do not touch the appliance with wet hands.
- Clean using a damp cloth, to which you can apply a little detergent if necessary.
- Do not let water run into the appliance or spill water onto the appliance.
- Cleaning can be done once per month.

When this appliance is not being used, it should be stored in a dry location out of reach of children. Do not keep the appliance connected to a power source. Follow the storage instructions provided in the section “Technical Specification”.

Disposal

Please note that this appliance is marked with the WEEE symbol on the left. This means that this appliance must not be disposed of together with ordinary household waste, as electrical and electronic waste must be disposed of separately.

FAQ

Problem	Possible Causes	Solutions
Device does not light up	On/ Off button switched off	Switch on the on/off button
	No Power	Connect the main plug correctly
	No Power	The main cable is defective. Contact customer services or your retailer
	LEDs have reached the end of its service life	If you need to have the appliance repaired, contact customer service or an authorized dealer.
	LEDs faulty	

Technical Specification

Model	LND018915V2
Dimension (WHD)	260 x 150 x 20mm
Weight	500g
Lighting elements	LEDs
Power	DC12V, 1A
Light intensity	10,000 lux in 10 cm
Mains connection	12V DC, 1.0A, only in adaptor model: HP24A-120100-AdV or HP24A-120100-AdB
Operating Environment	0 to +40 °C, and 30% to 90% relative humidity Atmospheric pressure: 700-106
Storage and Transport Environment	-10 to +60 °C, and 30% to 90% relative humidity Atmospheric pressure: 700-1060hPa
Product Classification	Class II equipment No applied part Protection against ingress of water: IP21 Not category AP / APG equipment Mode of operation: Continuous operation
Maximum length of power cord	2000 mm
Location of emission aperture	Screen
Treatment area	Around 400 cm ²
Risk Group of light source equipment	Exempt Group
Product Service Life	5 years

Subject to technical modification.

Maximum output of optical radiation for the “SAD Light”:		
Radiance Output	Risk group classified by IEC 60601-2-57:2011	Max. value
EUVA: Eye UV-A	Exempt Group	1.0×10 ⁻⁴ W·m ⁻²
ES: Actinic UV skin & eye	Exempt Group	1.0×10 ⁻⁴ W·m ⁻²
EIR: Infrared radiation hazard exposure limits for the eye	Exempt Group	1.0×10 ¹ W·m ⁻²
EH: Thermal hazard for the skin	Exempt Group	2.0×10 ¹ W·m ⁻²
LB: Blue light	Exempt Group	5.0 ×10 ¹ W·m ⁻² ·sr ⁻¹
LR: Retinal thermal	Exempt Group	5.0 ×10 ¹ W·m ⁻² ·sr ⁻¹

Contact information

Model Number: SADV1

Brand: Duronic

Details:

Shine-mart Ltd, trademark owners of Duronic products.

1 Spilsby Road

Romford, RM3 8SB,

United Kingdom

The “SAD Light ” is manufactured by:

Zero-Plus International Limited

Room 1004, 10/F., Join-In Hang Sing Centre, 71-75 Container Port Road, Kwai Chung, New Territories, Hong Kong

European Authorised Representative:

Globalmind Consumer Electronics GmbH Ernst-Mantius-Str.11,

21029 Hamburg,

Germany

If this appliance is damaged in any way or it is not working and you cannot find the answer to your question in FAQ. Please contact customer service or the retail-er for assistance.

Warranty

1 Year Guarantee from Shine-Mart Ltd, Trademark owners of Duronic Products.

NOTE: THESE STATEMENTS DO NOT AFFECT YOUR STATUTORY RIGHTS AS A CONSUMER

This product has been manufactured under the strictest of quality control procedures, and using the highest quality of materials, to ensure excellent performance and reliability. It will give very good and long lasting service, provided it is properly used and maintained. The product is guaranteed for 1 years from the date of original purchase. If any defect arises due to a faulty materials or poor workmanship, the faulty product must be returned to the original place of purchase. Refund or replacement is at the discretion of that company. Duronic Products are offered with a 1 year guarantee under the following conditions:

1. The product must be returned to the retailer with original proof of purchase.
2. The product must be installed and used in accordance with the instructions contained in this operating manual.
3. It must be used only for domestic purposes.
4. It does not cover wear and tear, malicious damage, misuse, neglect, inexperienced repairs or consumable parts.
5. Shine-Mart Ltd has no responsibilities for accidental or consequential loss or damage.
6. Shine-Mart Ltd are not liable to carry out any type of servicing work, under the guarantee
7. Valid in the UK only

This product is fitted with CE:1363 plug and fuse
Imported by Shine-Mart Ltd, RM3 8SB

Information on waste Disposal for Consumers of Electrical & Electronic Equipment.

This mark on a product and/or accompanying documents indicates that when it is to be disposed of, it must be treated as Waste Electrical & Electronic Equipment, (WEEE).

Any WEEE marked products must not be mixed with general household waste, but kept separate for the treatment, recovery and recycling of the materials used.

For proper treatment, recovery and recycling; please take all WEEE marked waste to your Local Authority Civic waste site, where it will be accepted free of charge.

If customers dispose of Waste Electrical & Electronic Equipment correctly, they will be helping to save valuable resources and preventing any potential negative effects upon human health and the environment, of any hazardous materials that the waste may contain.

Follow Us

duronic.com

[duronic](https://www.youtube.com/duronic)

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1 Rok Gwarancji firmy Shine-Mart LTD, marki handlowej Duronic Products.
5. Shine-Mart Ltd. nie ponosi odpowiedzialności za przypadkowe albo celowe zgubienie produktu lub jego uszkodzenie.
6. Firma Shine-Mart Ltd nie jest zobowiązana do przeprowadzania jakichkolwiek prac serwisowych w ramach gwarancji.