## SAD Light - SADV1

**Contents**

<b>English Manual</b>	<b>4</b>
<b>Manuel d'utilisation en français</b>	<b>19</b>
<b>Gebrauchsanleitung auf Deutsch</b>	<b>37</b>
<b>Manual de Instrucciones en Español</b>	<b>46</b>
<b>Manuale D'istruzioni In Italiano</b>	<b>64</b>
<b>Instrukcja obsługi. Polski</b>	<b>72</b>

## Introduction

### Instructions for Use

Congratulations on the purchase of your new “Daylight”. The product will offer many years of reliable usage if handled correctly. Please keep your receipt, these instructions and the original packaging safe in case of future requirement.

Please read the instructions carefully to gain the most benefit from your new “Daylight”, although the usage guidelines in this document do not replace the advice of your qualified medical advisor.

### Intended Use:

Medical intended use of this appliance is to treat the symptoms of SAD (seasonal affective disorder) or winter depression. “Daylight” is intended to boost people’s energy levels, to adjust circadian rhythms and make people feel more awake. “Daylight” is safe and medically proven to provide relief from the symptoms of SAD.

### English Manual

### How does the “Daylight” work?

In autumn and winter, the seasons with the least sunlight because of shorter days, the lack of sunlight and a problem with certain brain chemicals stops the hypothalamus working properly. The lack of light is thought to affect the production of the hormone melatonin (that makes you sleepy).

“Daylight” uses bright lights to simulate the sunlight during the darker autumn and winter, whilst eliminating the dangerous UV rays contained in sunlight. The additional light helps your brain to reduce to production of hormone melatonin, adjust the levels of melatonin that can help the symptoms of SAD or winter depression.

### Seasonal Affective Disorder:

Seasonal Affective Disorder (SAD) as known as winter depression or winter blues.

It may cause people to:

- Feel low, listless or fatigued
  - Have trouble concentrating
  - Lacking in energy
- Less active than normal  
Need for more sleep

Symptoms occur during the same season each year. Most people with SAD start to have symptoms during winter and feel better in summer.

### EMC information

**Warning:** Use of this appliance adjacent to or stacked with other device should be avoided because it could result in improper operation. If such use is necessary, this appliance and the other device should be observed to verify that they are operating normally. If the appliance does not light on or flickering, it means that the appliance is not operating normally.

**Warning:** Do not change any cable without authorization of the manufacturer. Use of cables other than those specified or provided by the manufacturer of this appliance could result in increased electromagnetic emissions or decreased electromagnetic immunity of this appliance and result in improper operation.

**Warning:** Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

**Warning:** Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm to any part of this appliance, including power cable. Otherwise, degradation of the performance of this equipment could result.

If the appliance does not light on or flickering due to the electromagnetic disturbances, do not use it and try to place the appliance in other location.

For maintaining the basic safety and essential performance with regard to electromagnetic disturbances for the expected service life, please precaution to take if the use location is near (e.g. less than 1,5 km from) from AM, FM or TV broadcast antennas.

### Maintenance and Repairs

- Avoid exposing the screen to direct sunlight, spotlight or any other heat source.
- Do not use alcohol or ammonia base liquid to clean the screen.
- The cleaning information provided in the section "Cleaning".
- Please do not attempt to repair the appliance yourself. This could result in serious injuries. Failure to observe this regulation shall void the warranty.
- If this appliance is damaged in any way or it is not working, it should not be operated. Please try to fix the problem according to FAQ table. If the problem is still not resolved, please contact customer service or the dealer, since special tools are needed to make repairs. Never try to repair the appliance yourself.

### Safety and performance standards

The "Daylight" has been designed and manufactured to conform to the highest standards of safety and performance including electromagnetic compatibility (EMC).

The "Daylight" conforms to the applicable requirements of the following:

**CE 0123** This appliance meets the requirement of Medical Device Directive 93/42/EEC for medical appliances.

The appliance is complied with the following emission and immunity standard, and test level:

Emissions Test Standard	Compliance
RF emissions CISPR 11	Group 1, Class B
Harmonic emissions IEC 61000-3-2	Class A
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Comply

Guidance and manufacturer's declaration - electromagnetic Immunity							
Radiated RF IEC61000-4-3 (Test specification for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Modulation (W)	Distance (M)	Immunity Test Level (V/M)
	385	380 - 390	TETRA 400	Pulse modulation 18Hz	1.8	0.3	27
	450	430 - 470	GMRS 460 FRS 460	FM + kHz deviation 1 kHz sine	2	0.3	28
	710 745 780	704 - 787	LTE Band 13,17	Pulse modulation 217 Hz	0.2	0.3	9
	810 870 930	800 - 960	GSM 800 / 900 TETRA 800 iDEN 820 CDMA 850 LTE Band 5	Pulse modulation 18Hz	2	0.3	28
	1720 1845 1970	1700 - 1990	GSM 1800 CDMA 1900 GSM 1900 DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0.3	28
	2450	2400 - 2570	Bluetooth WLAN, 802.11 b/g/n, RFID 2450 LTE Band 7	Pulse modulation 217 Hz	2	0.3	28
	5240 5500 5785	5100 - 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9


Immunity Test Standard	IEC 60601-1-2 Test Level	Compliance Level
Electrostatic discharge (ESD) IEC 61000-4-2	+8kV contact +2kV, +4kV, +8kV, +15kV air	+8 kV contact +2kV, +4kV, +8kV, +15kV air
Electrical fast transient/burst IEC 61000-4-4	+2kV, for power supply lines +1kV signal input/ output 100 kHz repetition frequency	+2kV, for power supply lines Not applicable 100 kHz repetition frequency
Surge IEC 61000-4-5	+0.5kV, +1kV differential lines +0.5kV, +1kV, +1kV common mode	+0.5kV, +1kV differential lines Not Applicable
Voltage dips, interruptions and variations on power supply input lines IEC 61000-4-11	0 % UT; 0.5 cycle. At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°. 0 % UT; 1 cycle and 70 % UT; 25/30 cycles; Single phase: at 0°. 0 % UT; 250/300 cycle	0 % UT; 0.5 cycle. At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°. 0 % UT; 1 cycle and 70 % UT; 25/30 cycles; Single phase: at 0°. 0 % UT; 250/300 cycle
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m 50Hz/ 60Hz	30 A/m 50Hz/ 60Hz
Conducted RF IEC 61000-4-6	3 V 0,15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz	3 V 0,15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz
Radiated RF IEC 61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz

**Note:** UT is the a.c. mains voltage prior to application of the test level.

The following symbols appear in this manual, on the “Daylight” or on its accessories:

	Warning instruction indicating a risk of injury or damage to health		Read operating instructions before use
	Safety note indicating possible damage to the appliance/ accessories		Caution: Consult accompanying documents
	Specifies serial number of the “Daylight”		Class II Equipment
	OFF/ ON		Date of Manufacture
	Authorized Representative in the European Community		Manufacturer
	Ingress Protection Rating		Keep Dry

## Unit Description

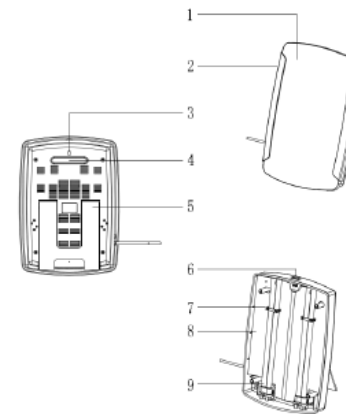1. Screen
2. Brackets
3. Mounting hole
4. Carrying handle
5. Fold-out stand
6. On/ Off Switch
7. Retaining Clips
8. Reflector
9. Tubes, 2 x 36W

### Setup

1. Remove the appliance from the packaging.
2. Fold the stand out and place the appliance on a solid surface, such as a table.
3. Place the appliance approximately 20-60 cm from your eyes, the light must bathe your face from the side.

### Mains Connection

Connect the appliance only to the mains voltage listed on the type plate. Push the plug completely into the mains socket.

### Note

- Push the plug completely into the mains socket.
- Ensure that there is a socket near the place where the appliance will stand.
- Lay the mains cable in such a way that no one can trip over it.

### Fastening of Daylight on the wall

There is a mounting hole on the back side of the appliance. At this mounting hole the appliance can be fixed on the wall with screw. Depending on the type of wall, an appropriate screw must be used, may be with pegs. E.g. for concrete wall, M4, 25mm long Stainless Steel Self-Tapping Screw which is screwed into a anchor.

The screw and the anchor are not included in the scope of delivery.

## Operation

Before use, please check the appliance for damage or faults. If you notice any damage or faults on the appliance, do not use it and contact customer services or your supplier.

### 1. Turning On

Press the ON/OFF button once of the appliance to turn on.

### 2. Taking the light

You can read, write, make telephone calls or work while you use the appliance. When using the appliance, try to maintain the recommended distance between your eyes and the appliance.

Usage guidelines are as follows:

Distance	Duration
60 cm	Approx. 2 Hours
40 cm	Approx. 60 Minutes
20 cm	Approx. 30 Minutes

In effect, the closer you are to the source of light, the less time you need to use it.

### Note

- Do not look directly into the light continuously.
- Some eye pain or mild headaches may be experienced during initial sessions but these should pass. If in doubt just lower your treatment times until you feel comfortable again.
- Do not leave the appliance unattended in the duration.
- Do not touch the appliance in the duration. Let the appliance cool down for at least 15 minutes before repositioning it.
- Do not block vent holes in the duration.

### 3. Enjoying light over longer periods

During the darkest period of the year, repeat the treatment for at least 7 consecutive days, or longer, depending on your individual requirements.

### 4. Switching off

Press the ON/OFF button of the appliance to switch off the light. Pull the mains plug from the socket. Let the appliance cool down for at least 15 minutes before storing it away.

## Cleaning

### Warning

Before you clean the appliance, always switch it off, unplug it and let it cool off.

- Do not touch the appliance with wet hands.
- Clean using a damp cloth, to which you can apply a little detergent if necessary.
- Do not let water run into the appliance or spill water onto the appliance.
- Cleaning can be done once per month.

### Storage


When this appliance is not being used, it should be stored in a dry location out of reach of children. Do not keep the appliance connected to a power source. Follow the storage instructions provided in the section "Technical Specification".

### Replacing the lamp tubes

In the event that the lights are faulty, or when they come to end of their life, the two light tubes can be replaced. NOTE: Do not replace the lamp tubes which are not provided by the supplier. To do so, proceed as follows:

- Always unplug the appliance and let it cool off.
- For safety reasons, do not remove any components or screws other than those specified below.


1. Slacken the retaining screws on the back using a screwdriver, lay the apparatus on its back and remove the screen.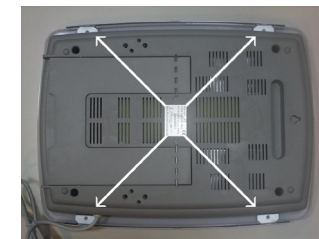

### Note

The four brackets on the front of the screen will become loose when the screws are slackened. Take care not to lose them, or the screen cannot be refitted correctly.

2. Cut the cable ties and bend the tube's two transparent retaining clips by pulling them gently upwards.

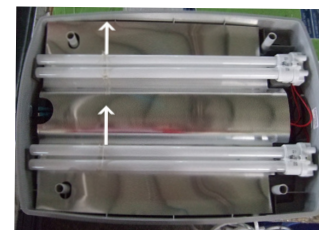### Note

Cable ties are used for transport protection only.

## Technical Specification

Dimension (WHD)	340 x 480 x 80 mm
Weight	2.0kg
Lighting Elements	2 x 36W tubes (2G 11), Daylight (full spectrum)
Nominal data for replacement Lighting elements	FSL – YDW36-H.RR 36W/865 2G11 4pin / Philips - MASTER PL-L 36W/865/4P Service Life: 8000 hours
Fuse	T3.15AH250V
Power	72 watts
Light Intensity	10,000 lux in 15cm
Mains connection	220 V ~240V, 50 Hz
Operating Environment	50 – 77 oF/ 10 – 25 oC, and relative humidity from 30% to 90%. Pressure altitude: <2000m Atmospheric pressure: 700-1060hPa
Storage and Transport Environment	41 - 122oF/ 5 – 50 oC, and relative humidity of max. 90%. Pressure altitude: <2000m Atmospheric pressure: 700-1060hPa
Product Classification	1. Class II equipment 2. No applied part 3. Protection against ingress of water: IP20 4. Not category AP / APG equipment 5. Mode of operation: Continuous operation.
Maximum length of power cord	1800 mm
Location of emission aperture	Screen
Treatment area	Around 1600 cm <sup>2</sup>
Risk Group of light source equipment	Exempt Group classified by IEC 60601-2-57:2011
Product Service Life	

## Maximum output of optical radiation for the “Daylight”:

Radiance Output	Risk group classified by IEC 60601-2-57:2011	Max. value
EUVA: Eye UV-A	Exempt Group	8×10 <sup>-3</sup> W·m <sup>-2</sup>
ES: Actinic UV skin & eye	Exempt Group	5×10 <sup>-5</sup> W·m <sup>-2</sup>
EIR: Infrared radiation hazard exposure limits for the eye	Exempt Group	2×10 <sup>-3</sup> W·m <sup>-2</sup>
EH: Thermal hazard for the skin	Exempt Group	10 W·m <sup>-2</sup>
LB: Blue light	Exempt Group	2×10 <sup>1</sup> W·m <sup>-2</sup> ·sr <sup>-1</sup>
R: Retinal thermal	Exempt Group	2×10 <sup>2</sup> W·m <sup>-2</sup> ·sr <sup>-1</sup>

## Contact Information

**Model Number:** SADV1

**Brand:** Duronic

**Details:**

Shine-mart Ltd, trademark owners of Duronic products.

1 Spilsby Road  
Romford, RM3 8SB,  
United Kingdom

**The “Daylight” is manufactured by:**

Zero-Plus International Limited  
Room 1004, 10/F., Join-In Hang Sing Centre,  
71-75 Container Port Road, Kwai Chung,  
New Territories, Hong Kong

**European Authorised Representative:**

Globalmind Consumer Electronics GmbH  
Ernst-Mantius-Str.11,  
21029 Hamburg,  
Germany

**Important**

If this appliance is damaged in any way or it is not working and you cannot find the answer to your question in FAQ. Please contact customer service or the retailer for assistance.

# Warranty

1 Year Guarantee from Shine-Mart Ltd, Trademark owners of Durotic Products.  
 NOTE: THESE STATEMENTS DO NOT AFFECT YOUR STATUTORY RIGHTS AS A CONSUMER

This product has been manufactured under the strictest or quality control procedures, and using the highest quality of materials, to ensure excellent performance and reliability. It will give very good and long lasting service, provided it is properly used and maintained. The product is guaranteed for 1 years from the date of original purchase. If any defect arises due to a faulty materials or poor workmanship, the faulty product must be returned to the original place of purchase. Refund or replacement is at the discretion of that company. Durotic Products are offered with a 1 year guarantee under the following conditions:

1. The product must be returned to the retailer with original proof of purchase.
2. The product must be installed and used in accordance with the instructions contained in this operating manual.
3. It must be used only for domestic purposes.
4. It does not cover wear and tear, malicious damage, misuse, neglect, inexpert repairs or consumable parts.
5. Shine-Mart Ltd has no responsibilities for accidental or consequential loss or damage.
6. Shine-Mart Ltd are not liable to carry out any type of servicing work, under the guarantee
7. Valid in the UK only

This product is fitted with CE:1363 plug and fuse  
 Imported by Shine-Mart Ltd, RM3 8SB

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Information on waste Disposal for Consumers of Electrical & Electronic Equipment.  
 This mark on a product and/or accompanying documents indicates that when it is to be disposed of, it must be treated as Waste Electrical & Electronic Equipment, (WEEE).  
 Any WEEE marked products must not be mixed with general household waste, but kept separate for the treatment, recovery and recycling of the materials used.  
 For proper treatment, recovery and recycling; please take all WEEE marked waste to your Local Authority Civic waste site, where it will be accepted free of charge.  
 If customers dispose of Waste Electrical & Electronic Equipment correctly, they will be helping to save valuable resources and preventing any potential negative effects upon human health and the environment, of any hazardous materials that the waste may contain.

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1 Rok Gwarancji firmy Shine-Mart LTD, marki handlowej Duronic Products.
5. Shine-Mart Ltd. nie ponosi odpowiedzialności za przypadkowe albo celowe zgubienie produktu lub jego uszkodzenie.
6. Firma Shine-Mart Ltd nie jest zobowiązana do przeprowadzania jakichkolwiek prac serwisowych w ramach gwarancji.