

VERSA GRIP

The EasyShoe Versa Grip lineup consists of hybrid urethane horseshoes available in nail-on and glue-on application options. The Versa Grip offers versatility for endurance riding, trail riding, dressage, show jumping, carriage driving, western disciplines, and more. The wide web design also makes the Versa Grip ideal for therapy cases. Holds pour-in packing very well.



Key Features of the Versa Grip Lineup

Shock absorbing urethane. Each Versa Grip shoe is designed with urethane that significantly reduces the concussion on hard surfaces. The shock absorption promotes a healthier hoof and healthier joints.

Natural hoof mechanism. The flexibility of Versa Grip allows the hoof to naturally expand and contract, both laterally and vertically. Additionally, the bar in the heel area supports continuous blood circulation to the hoof with each footfall. The open sole design assures complete breathability of the sole and frog.

Durability. The Versa Grip features an aggressive, long-wearing, thicker profile with a deeper tread pattern molded from a durable urethane. The Versa Grip lineup of horseshoes can often last for multiple resets.

Hoof protection across all terrains. All shoes in the Versa Grip lineup include a flexible frog protection to prevent stones and debris from bruising the hoof. Additionally, the aggressive tread pattern provides sufficient traction for all terrains and environments.

Modification options. Each Versa Grip horseshoe can be adjusted to meet the needs of each horse. Material can be easily rasped away to shape the shoe, adjust the length, or adjust breakover. Versa Grip accessories can assist in meeting the needs of each horse.

In addition to these aspects, each Versa Grip model has unique features that sets the shoe apart from the others.

Application videos and educational resources can be found at easycareinc.com/application

EasyShoe Versa Grip Size Chart

All horseshoes in the Versa Grip lineup are sized using the following size chart. Please see specific product for available sizes.

Front Sizes			
Size	Width (mm)	Length (mm)	Thickness (mm)
102F	96-102	113	16
106F	100-106	117	16
110F	104-110	122	16.2
114F	108-114	126	16.2
118F	112-118	131	16.2
122F	116-122	135	16.2
126F	120-126	139	16.2
130F	124-130	143	16.2
134F	128-134	148	16.2
138F	132-138	153	16.7
142F	136-142	157	16.7
146F	140-146	161	16.7
150F	144-150	166	17.2
154F	148-154	170	17.2
158F	152-158	175	17.2
162F	156-162	179	17.8
166F	160-166	183	17.8
170F	164-170	188	17.8

Hind Sizes			
Size	Width (mm)	Length (mm)	Thickness (mm)
102H	96-102	118	16
106H	100-106	122	16
110H	104-110	127	16
114H	108-114	131	16.2
118H	112-118	136	16.2
122H	116-122	140	16.2
126H	120-126	145	16.2
130H	124-130	149	16.2
134H	128-134	154	16.2
138H	132-138	159	16.7
142H	136-142	163	16.7
146H	140-146	168	16.7
150H	144-150	173	17.2
154H	148-154	177	17.2
158H	152-158	182	17.2
162H	156-162	186	17.8
166H	160-166	191	17.8
170H	164-170	195	17.8

EasyShoe Versa Grip Weights

Front (grams)						
Size	Versa Grip	Versa Grip with Studs	Versa Grip Gold	Versa Grip Light	Versa Grip Glue	Versa Grip Octo
102F	209	225	189	127	158	n/a
106F	222	238	204	137	166	n/a
110F	239	256	220	147	178	n/a
114F	258	276	238	161	187	188
118F	280	296	257	174	199	200
122F	299	315	273	185	216	220
126F	322	334	291	198	229	230
130F	338	351	307	209	253	243
134F	365	380	335	223	264	270
138F	392	414	360	244	286	287
142F	413	432	383	268	305	306
146F	447	469	408	274	325	TBD
150F	483	499	439	297	346	TBD
154F	513	528	463	316	n/a	n/a
158F	537	552	485	333	n/a	n/a
162F	579	592	520	356	n/a	n/a
166F	604	622	543	377	n/a	n/a
170F	625	649	569	395	n/a	n/a

Hind (grams)						
Size	Versa Grip	Versa Grip with Studs	Versa Grip Gold	Versa Grip Light	Versa Grip Glue	Versa Grip Octo
102H	210	226	194	133	161	n/a
106H	226	243	208	144	175	n/a
110H	243	260	225	154	186	n/a
114H	265	280	244	168	194	193
118H	285	304	263	181	208	209
122H	304	322	278	194	224	229
126H	325	342	299	206	244	241
130H	339	355	313	215	248	252
134H	367	386	339	233	276	275
138H	399	415	366	254	295	297
142H	436	445	387	264	315	317
146H	455	471	415	282	329	TBD
150H	491	507	446	310	360	TBD
154H	510	535	473	333	n/a	n/a
158H	547	558	493	345	n/a	n/a
162H	582	499	530	374	n/a	n/a
166H	611	626	554	390	n/a	n/a
170H	641	657	578	403	n/a	n/a

EasyShoe & Accessory Information

EasyShoe	Primary Application	Alternative Application	Original Tab Replacement	Other Accessories
Versa Grip	Nail	*Glue	n/a	All Tabs, Versa Grip Rocker, Versa Grip Wedge, Versa Grip Plug, Quick Studs
Versa Grip With Studs	Nail	*Glue	n/a	All Tabs, Versa Grip Rocker, Versa Grip Wedge, Versa Grip Plug, Quick Studs
Versa Grip Gold	Nail	*Glue	n/a	All Tabs, Versa Grip Rocker, Versa Grip Wedge, Versa Grip Plug, Quick Studs
Versa Grip Light	Nail	*Glue	n/a	All Tabs, Versa Grip Rocker, Versa Grip Wedge, Versa Grip Plug, Quick Studs
Versa Grip Octo	Super Glue	**Nail	11 Finger Tab	All Tabs, Versa Grip Rocker, Versa Grip Wedge, Versa Grip Plug, Quick Studs
Versa Grip Glue	Acrylic Glue	**Nail	EasyTab	All Tabs, Versa Grip Rocker, Versa Grip Wedge, Versa Grip Plug, Quick Studs

* Direct glue or weld on tabs.

** Tabs must be removed.

EasyShoe Versa Grip

SB-ESVG-F (FRONT) and SB-ESVG-H (HIND)

The Versa Grip features an aggressive, long-wearing, thicker profile with a deeper tread pattern molded from a durable urethane. The nailing slots and clear material make nail placement straight forward and are extremely beneficial for achieving an accurate installation. Pre-angled, integrated clips, 12 nail options, and debris guard. Available in front and hind patterns, sizes 102-170. Sold in pairs.

Optional accessories: 11 Finger Tabs, Sticky Finger Tabs, EasyTabs, Quick Studs, Versa Grip Rocker, Versa Grip Wedge, Versa Grip Plug



EasyShoe Versa Grip With Studs

SB-ESVGS-F (FRONT) and SB-ESVGS-H (HIND)

The Versa Grip is also available with threaded inserts for optional 3/8 threaded studs. The Versa Grip With Studs has the same durability and flexibility as the Versa Grip. Pre-angled, integrated clips, 12 nail options and debris guard.

Available in both front and hind patterns, sizes 102-170. Sold in pairs.

Optional accessories: 11 Finger Tabs, Sticky Finger Tabs, EasyTabs, Quick Studs, Versa Grip Rocker, Versa Grip Wedge, Versa Grip Plug



EasyShoe Versa Grip Gold

SB-ESVGG-F (FRONT) and SB-ESVGG-H (HIND)

The Versa Grip Gold features independent nailing plates for increased shoe flexibility, secure nailing and weight reduction. Pre-angled, integrated clips, 12 nail options and debris guard.

Available in both front and hind patterns, sizes 102-170. Sold in pairs.

Optional accessories: 11 Finger Tabs, Sticky Finger Tabs, EasyTabs, Quick Studs, Versa Grip Rocker, Versa Grip Wedge, Versa Grip Plug



EasyShoe Versa Grip Light

SB-ESVGL-F (FRONT) and SB-ESVGL-H (HIND)

The Versa Grip Light features an aggressive tread pattern with no metal for the lightest weight, flexible and most cost efficient option. The Versa Grip Light can be nailed on or glued on. Weld on tabs to glue with Super Glue or direct glue using Bond Glue. The Versa Grip Light can act as a Versa Grip Glue or Versa Grip Octo by welding on the corresponding tabs. Create a shoe for larger hooves by welding on tabs. 12 nail locations and debris guard. Can be reset.

Available in both front and hind patterns, sizes 102-170. Sold in pairs.

Optional accessories: Quick Studs, 11 Finger Tabs, Sticky Finger Tabs, EasyTabs, Versa Grip Rocker, Versa Grip Wedge, Versa Grip Plug



EasyShoe Versa Grip Octo

SB-ESVGO-C-F (FRONT) and SB-ESVGO-C-H (HIND)

The EasyShoe Versa Grip Octo is a glue-on urethane horseshoe with integrated Finger Tabs that make the gluing process quick and easy. Simply stand the horse in the horseshoe and apply the tabs using Super Glue. Super Glue will hold strong for a full trim cycle and dries in about 10 minutes. No welding required. Can be reset if removed carefully. New Finger Tabs can also be welded for a reset.

Available in both front and hind patterns, sizes 102-150. Sold in pairs.

Application tools needed: Super Glue, plastic wrap

Optional accessories: Quick Studs, 11 Finger Tabs, Sticky Finger Tabs, EasyTabs, Versa Grip Rocker, Versa Grip Wedge, Versa Grip Plug



EasyShoe Versa Grip Glue

SB-ESVGLU-F (FRONT) and SB-ESVGLU-H (HIND)

The EasyShoe Versa Grip Glue is a glue-on urethane horseshoe with integrated tabs that make the gluing process quick and easy. Simply stand the horse in the shoe and inject acrylic adhesive through the application ports. Features an aggressive tread pattern with no metal for a light weight, flexible shoe that allows the hoof to flex naturally with each footfall. Available in both front and hind patterns, sizes 102-150. Sold in pairs.

Application tools needed: Bond Glue or acrylic glue, glue dispensing gun, mixing tips, plastic wrap

Optional accessories: Quick Studs, 11 Finger Tabs, Sticky Finger Tabs, EasyTabs, Versa Grip Rocker, Versa Grip Wedge, Versa Grip Plug





EasyShoe Versa Grip Rocker

SB-ROCKER

The EasyShoe Versa Grip Rocker attaches to the bottom of all Versa shoes with the "Grip" tread pattern using eight included screws. The Rocker gives hoof care professionals the ability to add mechanics to a shoe without removing the shoe. The Rocker can be added, manipulated, or removed at any time during the shoeing cycle without removing the Versa Grip shoe.

Available in front patterns, sizes 114-150. Sold in pairs. Shoe not included.

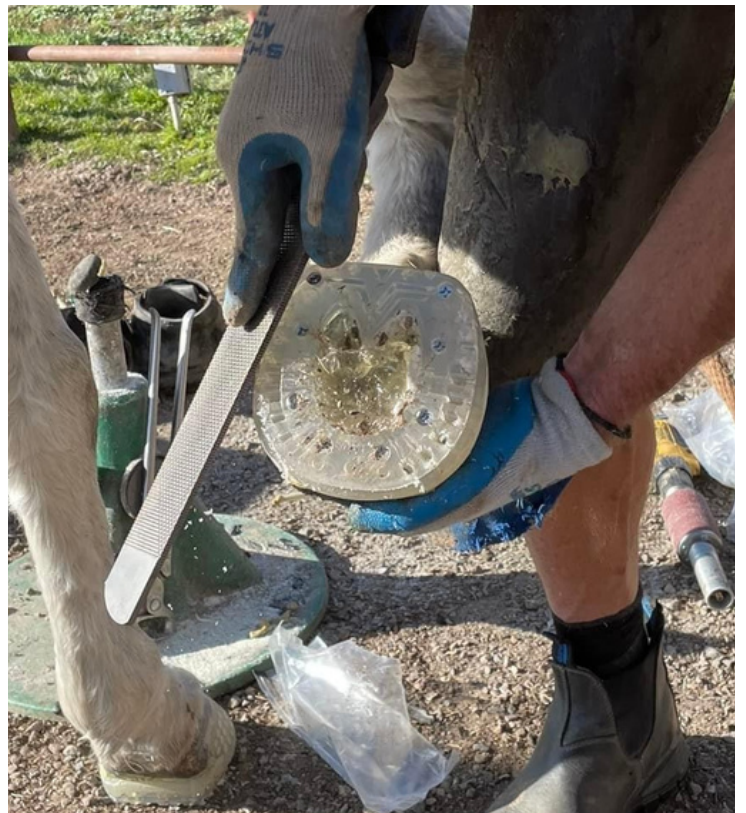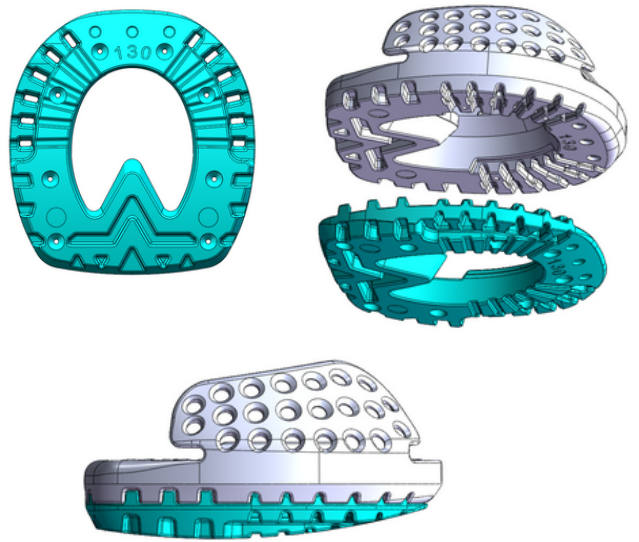


EasyShoe Versa Grip Wedge

SB-WED2.5-F (FRONT) and SB-WED2.5-H (HIND)

The EasyShoe Versa Grip Wedge inserts inside all Versa shoes with the "Grip" tread pattern using three included screws. The Wedge insert adds 2.5 degrees of heel lift and can be stacked to add 5 or 7.5 degrees of total lift.

Available in front and hind patterns, sizes 114-150. Sold in pairs. Shoe not included.

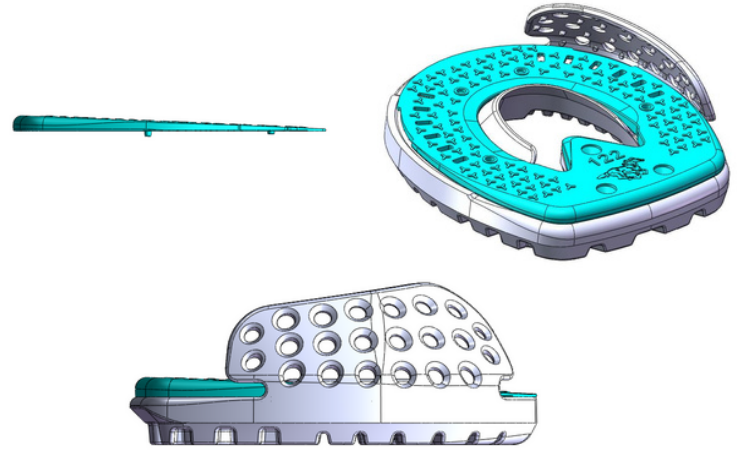

EasyShoe EasyTab

SB-EST

The EasyShoe EasyTab is a flexible cuff that can be welded to all Versa Grip horseshoes. Enjoy easy gluing with holes designed specifically for glue tips. Once welded to the horseshoe, simply apply the tabs to the hoof wall using acrylic glue such as EasyShoe Bond Glue. The EasyTab is ideal for the Versa Grip line or resetting the Versa Grip Glue. Sold in pairs.

EasyTabs are compatible with the Versa Grip, Versa Grip With Studs, Versa Grip Gold, Versa Grip Light, and Versa Grip Glue.



EasyTabs and Stealth Tabs Size Chart

Tab Size	Versa Grip Size
Small	102-114
Medium	118-134
Large	138-154
X-Large	158 +

EasyShoe Stealth Tab

SB-ES-ST

The EasyShoe Stealth Tab is a flexible cuff that can be welded to all Versa Grip horseshoes. Once welded to the horseshoe, simply apply the tabs to the hoof wall using Super Glue. The Stealth Tab is ideal for the Versa Grip line or resetting the Versa Grip Stealth. Sold in pairs.

Stealth Tabs are compatible with the Versa Grip, Versa Grip With Studs, Versa Grip Gold, Versa Grip Light, and Versa Grip Glue.



EasyShoe Versa Grip Plug

SB-ES-VGP-F (FRONT) and SB-ES-VGP-H (HIND)

The EasyShoe Versa Grip Plug attaches to the bottom of all Versa Grip horseshoes using Super Glue. With the Versa Grip Plug, no snow or mud will build up under the hoof. Apply the Plug at any time during the shoeing cycle. Available in both front and hind patterns, sizes 102-170. Sold in pairs.

The Versa Grip Plug is compatible with the Versa Grip, Versa Grip With Studs, Versa Grip Gold, Versa Grip Light, Versa Grip Glue, and Versa Grip Octo.




11 Finger Tabs

SB-ES-11R (Rubber) and SB-ES-11 (Urethane)

Weld on 11 Finger Tabs to any Versa Grip horseshoe. Tabs can be applied to hoof wall using EasyShoe Quick Set Super Glue. The Super Glue is easy to apply and holds strong for a full trim cycle. Glue-on tabs available in Rubber (Black) or Urethane (Clear).

11 Finger Tabs are compatible with the Versa Grip, Versa Grip With Studs, Versa Grip Gold, Versa Grip Light, and Versa Grip Glue.

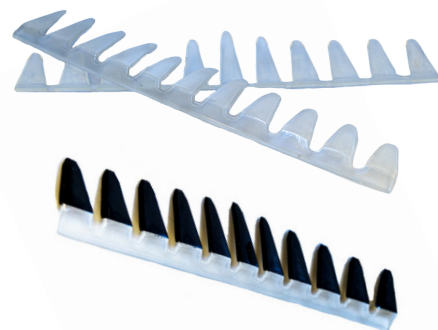

Sticky Finger Tabs

SB-ES-SF

Weld on Sticky Finger Tabs to any Versa Grip horseshoe. Tabs can be applied to hoof wall using EasyShoe Quick Set Super Glue. The Super Glue is easy to apply and holds strong for a full trim cycle. Glue-on tabs available in Rubber (Black) or Urethane (Clear).

11 Finger Tabs are compatible with the Versa Grip, Versa Grip With Studs, Versa Grip Gold, Versa Grip Light, and Versa Grip Glue.



Sticky Finger Tabs Size Chart

Tab Size	Versa Grip Size
Small	102-114
Medium	114-134
Large	134-162
X-Large	162+

EasyShoe Quick Set Super Glue

SB-QS-2 and SB-QS-8

EasyShoe Quick Set Super Glue is easy to apply and holds strong for a full trim cycle. Super Glue cures in about 10 minutes. Glue dispensing gun and mixing tips are not required. Available in 2 oz. and 8 oz. bottles.

Super Glue can be used with the Versa Grip Octo, 11 Finger Tabs, Sticky Finger Tabs, Stealth Tabs, and the Versa Grip Plug.



EasyShoe Bond Glue

SB-ES-BF (Fast Set) and SB-ES-BS (Slow Set)

EasyShoe Bond Glue for use with EasyShoe styles such as the EasyShoe Versa Grip Glue, EasyShoe Performance, EasyShoe Sport, and direct glue method. Also requires the EasyShoe Bond Glue Dispensing Gun (420 ml) and mixing tips. Available in Fast Set or Slow Set. Sold as single tube, 420 ml.

Bond Glue can be used with the Versa Grip Glue, EasyShoe EasyTabs, and direct glue method.



Dispensing Gun for EasyShoe Bond Fast or Slow Set. 420 ml.

SB-SHGUN

Dispensing Gun for EasyShoe Bond Glue, 420 ml. Glue sold separately.



EasyShoe Bond Glue 420 ml Mixing Tip

SB-ES-GTIP

EasyShoe Bond Glue 420 ml Mixing Tips. Sold Individually.



Heavy Duty Plastic Wrap - 150 gauge, 3" x 600

SB-WRAP

Heavy Duty Plastic Wrap. Quickly and easily wrap hoof so glue can set. Remove when glue is set. 150 gauge, 3" x 600'



Tab and Glue Information

EasyShoe Tab	EasyShoe Bond Glue	Vettec Adhere	EasyShoe Quick Set Super Glue
EasyTab	Yes	Yes	No
11 Finger Tab (Clear Urethane)	No	No	Yes
11 Finger Tab (Black Rubber)	No	No	Yes
Sticky Finger Tab	No	No	Yes
Stealth Tab	No	No	Yes



EasyShoe Bond Glue with EasyTabs.



EasyShoe Quick Set Super Glue with 11 Finger Tabs.