

EasyShoe Sport Frequently Asked Questions

Q Can my horse wear the shoes for the entire trim cycle?

A Yes. The shoe is designed to stay on the hoof for the entire trimming cycle.

Q Can the EasyShoe Sport be nailed on?


A No. The Sport is a Glue-On product shoe only.

Q Can I create a beveled toe with this shoe?

A No. The shoe is already designed for supreme traction. The shoe is also very thin and thus no space to add quick studs.

How to Properly Measure Your Horse's Hoof

We cannot over stress the importance of resizing! You wouldn't go for a jog in ill-fitting sneakers, so why would you expect your horse to do the same?

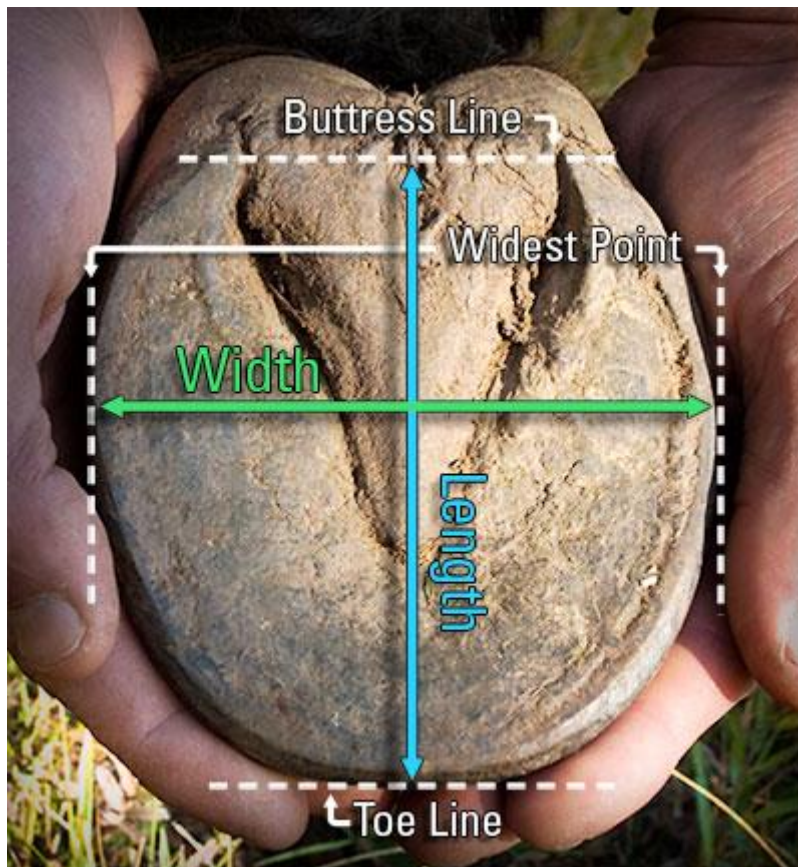

1. After a fresh trim, measure the width of the hoof across the bottom at the widest point.
 2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.
 3. Compare your measurement with the appropriate size chart. Please note that each boot style has its own unique size chart.
 4. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the hoof measurements fall closely to the maximum length or width, we recommend choosing the next size up. If the width and length measurements are different by more than one size, your selected boot style is not recommended. * Be sure to check your measurements against an alternative boot in the EasyCare line—another style may accommodate your horse's hoof shape better.
- *Note: If you are fitting an Easyboot Glue-On or Glove, width and length must fall into the same size category.