

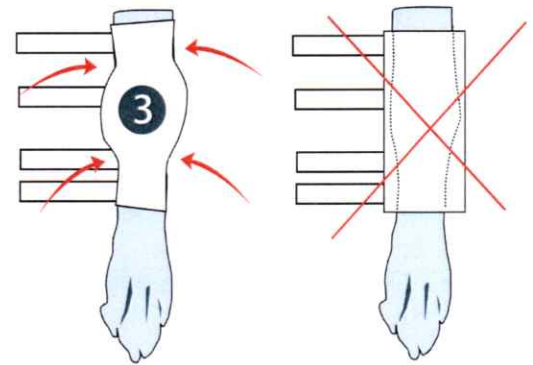
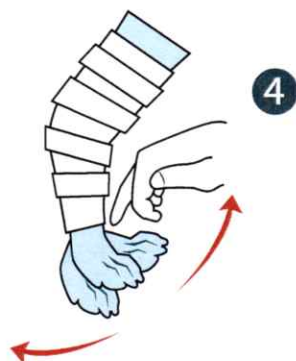


1. The orthosis is easiest to apply while the animal lies down and the tarsal joint is in a natural / stretched position.
2. Place the limb on the open orthosis with the wrap just above the back of the main footpad*1.
3. Tighten so that the orthosis is shaped around the leg. Fasten the orthosis with the elastic straps*2,3.

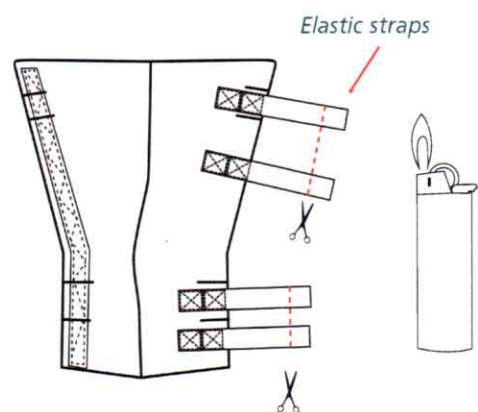

**2 If the wrap is too long or too wide, cut them with scissors.*

**3 If the elastic straps are too long, cut them with scissors. If you have cut the ends, burn them gently with a lighter to prevent fraying.*

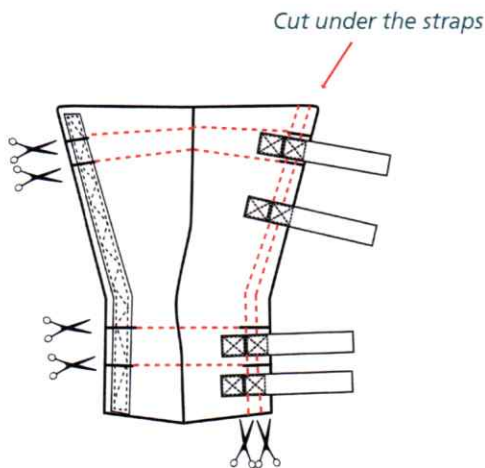

4. Apply the orthosis. Note! The Paw must have free movement and the orthosis must not cause pressure or chafing on top of the paw.

Important information

- Examine the leg 2-4 times / day for swelling of the toes, as well as heat gain and chafing.
- Contact your vet if exuding wounds, odours or other problems occur. It is always easier to treat complications that are noticed early, so let them know if there are skin irritations, preferably before it has become a wound or drainage of fluid.
- The orthosis always affects the gait. Does it seem uncomfortable for the patient? If the animal licks, bites or is not using the leg, please contact your vet. Be aware of changing behavior, for example, if the mood is not as it used to, or if the animal does not want to eat. Contact your veterinary clinic if you have questions!

Care instructions

- Remove fur and debris from the Velcro for best adhesion.
- Clean the orthosis with mild detergent by hand, maximum 30 °C.

Contact us for questions regarding the orthosis
and for any problems that may occur.

Tel. +46 (0)31-21 41 28 Epost: info@scandiop.se