

# LUXATO HEARING

Hear Today. Hear Tomorrow.

## Quick Guide

### 1. Charging

We recommend fully charging your hearing amplifier 3 hours before your first use.

The LED light on the Program Button will glow blue once connected properly and charging. Once fully charged, the light will glow green.

Fig. 1




### 2. Operating

Use the volume control knob to turn the hearing amplifier on and off. To turn the hearing amplifier on, rotate to any volume number will turn on the device. Rotate to zero will turn off the device.

Once on, you can press the Program Button once to cycle between the 4 preset amplification programs.

Fig. 2

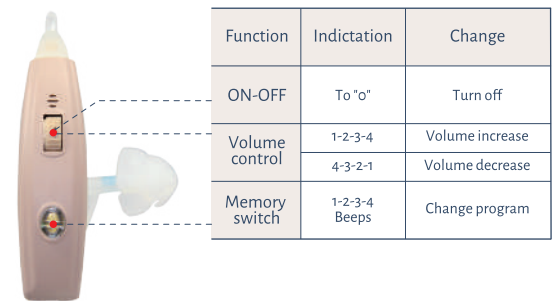

### 3. Fitting

The hearing amplifier intended for your right ear has a red 2B marking on the tube.

The hearing amplifier intended for your left ear has a blue 2B marking on the tube.

We recommend trying other sized ear domes if you are not initially satisfied with your hearing amplifier.

Fig. 3

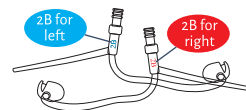



Fig. 4



### 4. Wearing

Place the hearing amplifier over your ear and insert the ear tube into the ear canal. The ear tube has a "retention lock" which you can position into your ear lobe to help secure the device. You may remove this lock from the ear tube by clipping it with scissors if it is not needed.

Fig. 5

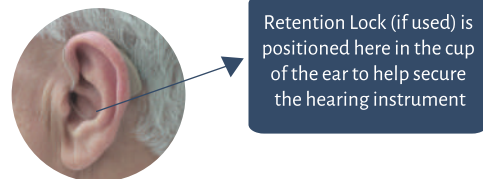

### 5. Maintenance

To clean, use the cleaning wire to clean the inside of the tubes and the brush to clean domes.

Fig. 6

