

LUX 14 Quick user guide

1. Charging

Place the hearing amplifier into the charging station.



Plug the small end of charging cable into the charging port.



Plug the big end of charging cable into the adapter.




Charging indicator

*Solid **BLUE** light when charging.



** Solid **GREEN** light when fully charged.

3. Operation

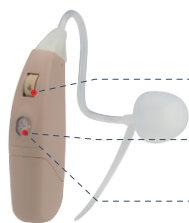

Function	Indication	change
Volume control	1-2-3-4	Volume increase
	4-3-2-1	Volume decrease
ON-OFF	Green light	Turned on
	Blue light	Turned off
Memory switch	1-2-3-4 beeps	Program switched

2. Fitting


The hearing amplifier intended for your right ear has a **red 2B** marking on the tube.

The hearing amplifier intended for your left ear has a **blue 2B** marking on the tube.



We recommend trying other sized ear domes if you are not initially satisfied with your hearing amplifier.

Round top dome(Power upgrade)
Effectively reduce the feedback/whistling sound.

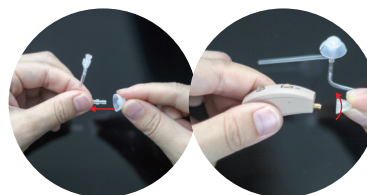

Magic dome
Effectively reduce occlusion effect and whistling sound.



4. Wearing

Place the hearing amplifier over your ear and insert the ear tube into the ear canal. The ear tube has a "retention lock" which you can position into your ear lobe to help secure the device. You may remove this lock from the tube by clipping it with scissors if it is not needed.



Retention Lock (if used) is positioned here in the cup of the ear to help secure the hearing instrument

5. Maintenance

To clean, use the cleaning wire to clean the inside of the tubes and the brush to clean domes.



Use the included microfiber cleaning cloth to daily clean the charging port of your devices.



6. FULL MANUAL & ONLINE SUPPORT

visit www.luxatohearing.com to view help documents, explainer videos and FAQs.

Scan the QR code for visiting our Youtube channel: Luxato hearing.

