Operations and Safety Manual

Before using Skiin Underwear, please read this manual.

GS1-ID# 62834195

Myant Inc.
Suite #500, 200 Ronson Drive,
Etobicoke, ON, M9W 5Z9, Canada
+1 (416) 423-7906
info@Myant.ca

LB00042
Rev. 2.3

Warranty Information

The Skiin Underwear and its accessories, when supplied and delivered new, in the original device to the original purchaser, are covered under a one-year warranty that covers damage due to defective materials and or failure of the equipment to operate in accordance with the information in this *Operation and Safety Manual for Skiin Underwear*.

Version Information

Skiin Underwear
Operation and Safety Manual for Skiin Underwear
Revision 2.3
Original Instructions PP002

For the latest styles of Skiin Underwear, visit www.skiin.com

Table of Contents

1. Introduction	4	Step 2: - Charge the Skiin Pod	10
2. Essential Performance	4	9. Skiin Underwear Components	10
3. Indication for Use as a Medical Device	4	9.1 Skiin Pod.....	10
4. Intended Use as a Medical Device	5	9.2 Skiin Charging Kit.....	12
5. Safety Information	5	9.3 Skiin Underwear	13
6. Contents	5	10. Getting Help	14
7. Concept of Operation	6	11. Traveling with Skiin Garments	15
8. Getting Started	6	12. Limited Warranty	15
Step 1 - Try on the Garment:		13. Device Specifications	16
a) Underwear Configuration.....	6	14. Symbols Glossary	17
b) Bra Configuration	7	15. Warnings	19
c) Bralette Configuration	8	16. Cautions	20
d) Chest Band Configuration.....	8	17. General Safety Precautions	20
e) Tank Top Configuration	9	18. Other	21

1. Introduction

This *Operation and Safety Manual* for the Skiin Underwear provides instructions for the safe and efficient use of the Skiin Underwear for biometric monitoring application.

The Skiin Underwear is authorized for sale by Health Canada.

Skiin Underwear uses textile electrodes embedded in the elastic band/garment with electronics in the Skiin Pod to measure and transmit your ECG. The underwear is machine washable & dryable and can be laundered with your other clothes (use the included wash bag).

The data can also be used by individuals curious about their health and physiology.

The Skiin Underwear is used with a user-supplied compatible smart device. Operators of the device will be able to benefit from the Skiin Underwear's non-intrusive form factor and ability to record continual data which can be analysed for cardiac abnormalities.

WARNING EN-W1		Unauthorized modification of this equipment is not permitted and may compromise the safe operation of the equipment.
NOTE EN-N1		Follow the instructions in this booklet to learn how to use and take care of your Skiin Underwear.

2. Essential Performance

The essential performance of the Skiin Underwear is the ability to record and export ECG raw data to generate an accurate report of the patient's ECG through a 3rd party tool.

3. Indication for Use as a Medical Device

The Skiin Underwear is indicated for users in need of remote monitoring and aiding the identification of potential cardiovascular issues by skilled medical professionals through a 3rd party reporting tool. The medical features for ECG monitoring of this product are not to replace traditional methods of diagnosis or treatment.

Contraindications

The Skiin Underwear is not for use by:

- Persons under 12 years of age
- Pacemaker and/or (ICD) implantable cardioverter defibrillator users
- Pregnant Women

Do not leave the device near babies, infants or children. This device is not intended for use by individuals with limited physical capacities or limited experience and / or knowledge unless they are supervised by an individual responsible for their safety or receive instructions from the latter as to how the device should be used.

 Risks for children and individuals requiring assistance

4. Intended Use as a Medical Device

The Skiin Underwear is an over the counter or prescription medical device intended for users 12 years of age or older to record and transmit continuous ECG at static positions, and other behavioral data for use by any 3rd party display and reporting software.

5. Safety Information

Before using the Skiin Underwear, please become familiar with the information in this manual.

WARNING EN-W2		Failure to follow safety instructions and/or using the equipment for purposes other than those described in Skiin Underwear labeling constitutes improper use.
WARNING EN-W3		Use of accessories, and cables other than those specified or provided by the manufacturer of this device could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

6. Contents

The standard configuration for Skiin Underwear includes the following components:

(A) 1 Skiin Pod

(B) 1 USB Charging Kit:

- 1 USB Charging Dock
- 1 Micro-USB Cable
- 1 Wall Plug

(C) **Skiin Garments:** Each kit has one "Try me On" garment that you must try first to check for fit before opening the other garment boxes.

(D) 1 Wash Bag

(A) Skiin Pod

(B) USB Charging Kit

(D) Wash bag

NOTE
EN-N2

Some features described in this guide may not be available with your configuration of Skiin Garments.

7. Concept of Operation

When you put on the Skiin Underwear and place the Pod in, it can start measuring your ECG, which will be stored as data in the Skiin Pod and can then be transmitted to a 3rd party app.

8. Getting Started

STEP 1: Try on the Garment

a) Underwear Configuration

Open the 'Try Me On' box and put on the Underwear to check the fit. It is important to ensure your Underwear is the correct size, as good contact of the sensor zones in the waistband with the skin surface is required to get a good signal for ECG recording.


Proper fit will ensure accurate heart rate readings. Your Underwear should be snug, that means it will not ride up, twist, or slip down. When trying on the Underwear, the band should not constrict or dig into the body.

The waistband should sit comfortably on your body and not be too tight or too loose. Please wash the garment before wearing it for the first time.

NOTE
EN-N3

The garment without the 'Try me On' label on the box, cannot be returned if opened.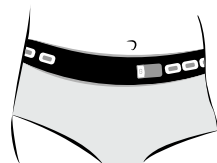

Sensor Zones

Located in the elastic band are 5 sensor zones, 3 on the left hip and 2 on the right hip. Make sure the five sensor zones located in the waistband of the Underwear are in contact with your skin. If your skin is too dry, it may affect the performance of the electrodes.


Underwear Sizing Chart

Size	Range (Inches)	
	Women's	Men's
XS	29.5 - 32	30 - 32.5
S	32 - 34.5	32.5 - 35
M	34.5 - 37	35 - 37.5
L	37 - 40.5	37.5 - 41.5
XL	40.5 - 45	41.5 - 45

How to measure: The above chart shows waist measurement by size. The waist measurement should be taken with a soft tape measure, around the top of the hip where the waistband of an elastic would sit. Keep a relaxed posture without holding in the abdomen.

b) Bra Configuration

Open the 'Try Me On' box and put on the Bra to check the fit. The Bra has 5 sensors, 4 on the front and 1 on the back.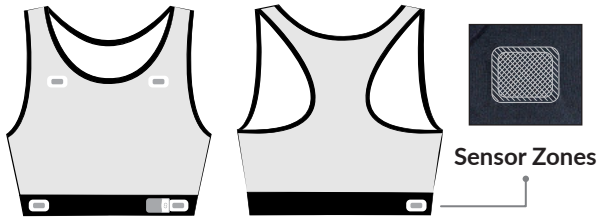

How to measure:

- **Under Bust** - place measuring tape below your bust, make sure the tape is tight and as level as possible.
- **Full Bust** - place measuring tape across the apex of your bust (the fullest part), make sure the tape is tight and level.

How to use the Size Chart:

Once you've measured your Under Bust and your Full Bust align the 2 measurements on the chart and find your Skiin size. If your size falls between 2 sizes (e.a. XS/S) select the smaller size based on your fit preference, if you prefer a more snug Bra select the smaller size, if you prefer a less compressive fit, up size.

Bra Sizing Chart

		Full Bust (in inches)																
		26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42
Under Bust (in inches)	25	XS	XS	XS	XS	XS												
	26		XS	XS	XS	XS/S	XS/S											
	27			XS	XS	XS/S	XS/S	XS/S										
	28				XS	XS/S	XS/S	XS/S	XS/S	S								
	29					S	S	S	S/M	S/M	M							
	30						S	S	S/M	S/M	S/M	M						
	31								M	M	M	M/L	L	L				
	32									M	M	M/L	L	L	L			
	33											L	L	L	L	L/XL	L/XL	
	34												L	L	L/XL	L/XL	XL	XL
	35													XL	XL	XL	XL	XL
	36														XL	XL	XL	XL


How it should fit:

The Bra is designed to provide you with a comfortable support, it should fit tight but still be comfortable during prolonged wear. This is a 3D knitted garment with engineered placement of different knit structures to allow

for stretch in some zones and provide support in others. The bra has no padding.

c) Bralette (Series 2 Bra Configuration)

Open the 'Try Me On' box and put on the Bralette to check the fit. Bralette has 5 sensors, 3 on the left and 2 on the right.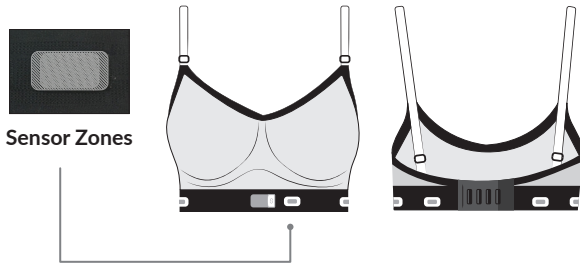

Skiin Series Bralette - Sizing Chart


Size	Range based on the under bust measurement (Inches)
XS	26 - 29
S	29 - 31
M	31 - 33
L	33 - 35
XL	35 - 37

How it should fit:

The Bralette is designed to provide you with improved comfort and support. The band should be worn snug around the underbust with the pocket for the Pod located in the center and facing outward. The band on the Bralette is adjustable- wear it on the loosest setting and tighten as needed.

d) Chest Band Configuration


Open the 'Try Me On' box and put on the Chest Band to check the fit. The Chest Band has 5 sensors, 3 on the left and 2 on the right.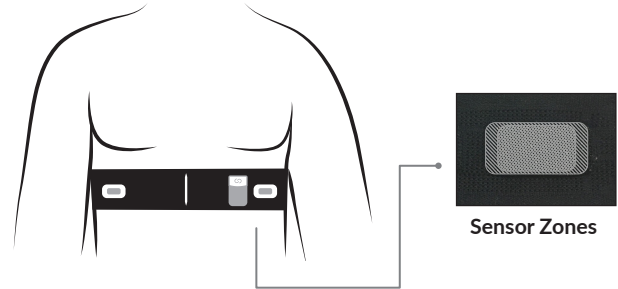

How to use the Chest Band

- 1) Wrap the band around your chest on the loosest setting and tighten as needed.

- 2) Locate the centre front of the band by the raised line after the SKIIN logo (See photo). Position the band below your Pectoral muscles or under your bustline.
- 3) You will hear a snapping sound when you open and close the closure (See images).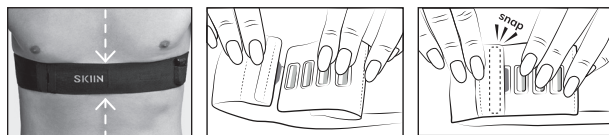


How to measure: Place measure tape under your bust/pectorals, make sure the tape is snug and level. Once you've measured, use the measurement and the chart to find your Skiin size. If your size falls between two sizes (e.a XS/S) select the smaller size.

Unisex Chest Band Sizing Chart

Size	Range (Inches)	Size	Range (Inches)
2XS	27.5 - 29.25	L	36.25 - 38.5
XS	29.5 - 31.5	XL	38.75 - 41.25
S	31.75 - 33.75	2XL	41.5 - 44.25
M	34 - 36	3XL	44.5 - 47.5

e) Tank Top Configuration

Open the 'Try Me On' box and put on the tank top to check the fit. The Tank Top has 5 sensors on the front. If your chest is hairy, it may prevent a good ECG signal. We recommend shaving the area in contact with the sensors.

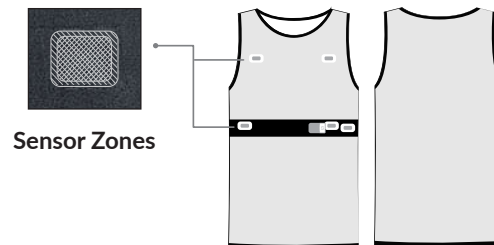How to measure: Place measure tape under your pectoral muscles, make sure the tape is snug and level.

Tank Top Men's Sizing Chart

Size	Range based on the under bust measurement (Inches)
XS	29 - 31.25
S	31.5 - 33.25
M	34 - 36.25
L	36.5 - 38.25
XL	39 - 41.75

To ensure your data is captured:

For best results, we recommend moisturizing your skin at the textile sensor sites to improve signal quality prior to donning the garment.

Wash the garment daily or after wearing it for 24 hours. Regular washing is necessary to restore proper sizing of the garment which ensures high quality of your recorded ECG. Ensure all components are dry prior to use, do not operate while wet or moist.

If the garment does not fit, visit: www.Skiin.com/returns for instructions on how to return or exchange for the correct size. You may return the unopened boxes of garment to exchange for the correct size.

For additional styles not shown here, visit <https://skiin.com/pages/support#/>

STEP 2: Find and Charge the Skiin Pod

Use the supplied charger to charge the Pod. Insert the Pod into the Charging Dock. You should feel the magnets pulling the Pod followed by an audible “Snap”. Connect the Charger and the Wall Plug using the micro-USB cable and plug it into a power source.

When the light on Pod shows a magenta (purple) colour, the Pod is activated. Take out the Pod from the charger, you should see a blinking yellow light and it is ready to be paired. The blinking yellow light on the Pod indicates that it is now ready to be paired with your smart device.

NOTE
EN-N4

It takes approximately 1.5 hours for the Pod to be fully charged from a discharged state.

9. Skiin Underwear Components

WARNING
EN-W4


Only the components listed in this section may be connected to the Skiin Underwear.

9.1 Skiin Pod

Your Skiin Pod records and securely sends your health information to a 3rd party app. Pair it with your smart device, then slide it into the Garment Dock. The Pod will easily 'snap' with the magnets when inserted in the Garment Dock. The Skiin Pod has an LED light that changes colours to indicate different states of charge and connection.

Remove the Pod from the garment when not using it. This is a good time to charge the Pod as well.

Picture of Pod with Model Number, FCC ID, IC, Serial Number and markings.
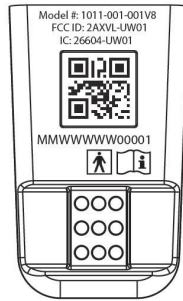
<p>CAUTION EN-C1</p>		<p>Remember to remove the Pod from the Underwear before washing.</p>
---------------------------------	--	--

No Charge in Battery

The Pod's LED will not appear or blink when the battery is depleted.

Low Battery

The Pod's LED blinks RED when the battery is low and the Pod should be charged.

Pod Connected State

Once the Pod is connected to your phone via Bluetooth

the Pod's LED turns into a BLUE blink, once every second, indicating that it is connected.

Disconnected and trying to pair/connect with a smartphone

If the Pod has no Bluetooth connection, the Pod's LED blinks yellow indicating that the Pod is disconnected and is trying to pair/connect with a smartphone. Please check your smart device settings to ensure that Bluetooth is activated.

Pairing the Pod to your Phone

When prompted by the App, place the Pod on your device screen to pair.

If your Pod is blinking yellow, it is ready to pair. Remember to unpair the Pod before attempting to pair it with a new account.

Pod Error State

The Pod's LED blinks GREEN when there is an error. Please contact Customer Support for assistance.

Turning the device on/off

The Skiin Pod is always “ON” as long as it is charged and has power. You cannot turn the Skiin Pod OFF, but you can unpair it from your phone in order to stop transmitting data. Refer above to Pod Disconnected State.

Cleaning the Pod

Do not wash the Pod or Charging Dock, USB Cable or Wall Plug, or immerse them in water. The surface of the Pod can be wiped clean with a soft, moistened cloth. Do not use any abrasive or solvent based cleaners.

Storing the Pod

Store the Pod in a clean, dry, safe location. Do not store in extremely hot, cold, humid, wet, or bright conditions.

Resetting the Pod


Insert the Pod into the charging dock, locate the pinhole under the charger. Use a pin to press into the hole and hold until the LED flashes to indicate it has successfully been reset.

Storage Capacity

Keep your phone nearby to collect Skiin data: the Pod

will save up to 45 minutes of ECG and up to 12 hours of non-medical data.

LED colours on Pod	
Solid Blue	The software on the Pod is being updated.
Solid Yellow	The pod is currently charging.
Single Blink Blue	The Pod is connected to the smart device.
Single Blink Yellow	The Pod is NOT connected to the smart device and it's ready to pair.
Single Blink Red	Low battery level & the Pod is connected to the smart device.
Rapid Blink Red	Low battery level & the Pod is NOT connected to the smart device.
Rapid Blink Green	Error state. Contact Customer Support.

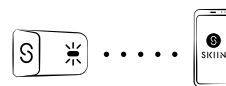
9.2 Skiin Charging Kit

Charging the Pod

When the Pod is inserted into the Charging Dock and connected to a power source, the Pod's LED will appear SOLID YELLOW to indicate that the Pod is currently charging. The Pod's LED will remain on and display a yellow light until the Pod has fully charged. We recommend that you fully charge the Pod for 1.5 hours before using.

Fully Charged

If the LED appears GREEN while charging, this indicates the Pod is fully charged and ready to use.

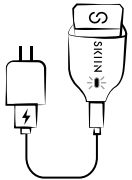LED colours on Pod		
	Magenta	The Pod is activated.
	Yellow	The Pod is not fully charged.
	Green	The Pod is fully charged.
	Blue	The software on the Pod is being updated.
	Red	A hardware issue is detected, contact support@skiin.com for assistance.
NOTE EN-N5 	While the Pod is charging it will not indicate Bluetooth connection status through the LED light. Do not try to pair the device while charging.	

9.3 Skiin Underwear

Materials: Our Skiin Garments can be washed like regular clothing. The elastic band and electrodes in all Skiin garments contain Carbon and Silver yarns. Carbon yarn is used in the textile electrodes of the garments to serve as a conduit to the skin. Conductive Silver yarn is used to connect textile electrodes within the band to the textile integration system.

- The Underwear band is made of 69% Nylon, 26% Spandex, 5% Other. The bottom parts are made of:

95% bamboo rayon, 5% Spandex. Bamboo is naturally antibacterial, comfortable, very breathable, moisture-wicking, fast drying and thermal regulating. Bamboo fabric is also anti-static so it doesn't cling to other materials.

- The Chest Band is made of 69% Nylon, 26% Spandex, 5% Other.
- The Tank Top is made of 82% Polyester, 17% Spandex, 1% Other.
- The Bra is made of 74% Polyester, 10% Nylon, 14% Spandex, 2% Other.
- The Bralette body is made of 95% Nylon, 5% Spandex. Band: 70% Nylon, 25% Spandex & 5% Other.

EN-W5		Warning: If you have an allergic or skin reaction please discontinue use and contact customer support at support@skiin.com . If necessary you should seek medical attention.
NOTE EN-N6		If the Skiin Pod is not inserted into the underwear you are wearing, no data will be collected or transmitted.

Laundry Instructions

- Please wash before wearing for the first time.
- Wash daily or after wearing the garment for 24 hours.
- Remove the Pod and put all garments in the washbag, use cold water when washing.
- Do not use bleach or fabric softener.
- When you change your underwear, take out the Skiin Pod and place it in a fresh pair.

Machine Wash on Cold

Tumble Dry on Medium

Do not Wring the Garment

Do Not Iron

No Bleach, No Softener

Do Not Dry Clean

EN-C2		Remember to remove the Pod from your underwear before washing.
NOTE EN-N7		Do not air-dry the underwear. A machine dryer should be used to ensure elasticity and long use of the underwear.

NOTE
EN-N8

To care for and maintain your Skiin Garments, place them in the wash bag before putting them in the washing machine.

Storing the Garments

Store garments in a clean, dry, safe location. Do not store in extremely hot, cold, humid, wet, or bright conditions.

10. Getting Help

If you have any questions, please contact the Skiin Customer support at support@skiin.com 24 hours/day, 7 days/week.

Send us an e-mail at feedback@skiin.com or visit www.Skiin.com to let us know how we can improve your experience.

Want to order more Garments or see additional styles? Visit www.skiin.com

NOTE
EN-N9

For Technical Support contact us at: +1 (844) 501-0252 or support@skiin.com

NOTE
EN-N10

In the event that an unexpected software error occurs, an error message will be displayed on the App.

11. Traveling with Skiin Garments

Flying while wearing a Skiin Garment is straightforward. Simply show the security statement (below) during screening, and if required/or asked to remove the Skiin Pod from the Skiin Garment Dock to disable the Skiin health data tracking for the duration of flight. Skiin Pods contain lithium rechargeable battery and should not be included in checked luggages.

SECURITY SCREENING STATEMENT

This person is wearing a Skiin Garment. This garment worn by the user tracks the user's ECG via the Skiin Pod. The Pod can be removed from the garment to stop the transmission of data to their Bluetooth connected smart device for ease of screening while travelling if required.

12. Limited Warranty

The underwear garment has a limited warranty of 30 days (on the date of purchase), and the Skiin Pod and accessories have a limited warranty of 1 year. If during this time the product malfunctions or does not perform as intended the user can request a new product to be shipped to them. Myant will only replace the defective part of the product after it has been returned and found to be defective due to normal use. Myant will not replace any products that have been tampered with or found to have been broken due to user negligence. For information on how to submit a warranty claim please email support@skiin.com.

13. Device Specifications

Performance Characteristics

- ECG Channel 3 Channels
- ECG Dynamic Range..... +/- 400mV
- Memory Storage Format..... 64Mb Flash Memory
- Pod Operational Time 1 year warranty
- Garment Operational Time 30 day warranty
- Frequency Range DC to 65 Hz
- CMRR..... 80 - 100 dB
- Input Impedance > 1 GΩ
- Sampling Rate.....320 samples/second
- ADC Resolution 24 bits DC
- Communication Interface Protocol
..... Bluetooth Low Energy 4.2
- Operating Frequencies.....2.4-2.48GHz
- Frequency Deviation 2 MHz/channel

Physical Characteristics

- Skiin Pod.....12.0 grams +/- 0.5g 27.9x46.5x10.6mm
- Skiin Underwear Men's and Women's sizes
range from 24" to 47"

Power Requirements

- Battery TypeRechargeable Lithium Polymer
- Power Supply Model.....KA1803A-US
(Ke Yu Dian Yuan (HongKong) International Ltd.)
- Battery Capacity200mAh, 2-3 days of use
- Battery Life minimum 24 Hours
- Operational Time..... 1 year warranty

Environmental Specifications

- Operational Temperature 5 to 40°C
- Operational Humidity..... 15% to 90%,
non-condensing, water vapour partial pressure <50hPa
- Operational Altitude..... 10kPa to 106kPa
- Storage Humidity..... 15% to 90%,
non-condensing, water vapour partial pressure <50hPa
- Storage Temperature -10°C to 45°C (for 1 month),
-10°C to 35°C (for 6 months), or -10°C to 28°C
(over 6 months)

14. Symbols Glossary

The tables in the following sections contain brief explanations of safety and other symbols information used to label underwear Skiin, its accessories and consumables.


Warnings, Cautions and Notes are used to describe serious and non-serious conditions at the level of the system. The symbols used to describe these conditions of safety are:

<p>WARNING EN-WO</p> 	<p>Messages with this heading indicate serious adverse reactions, potential safety hazards, and limitations in use imposed by a condition. The warning will identify steps that should be taken if the incident occurs.</p>
<p>CAUTION EN-CO</p> 	<p>Messages with this heading indicate information regarding any special care to be exercised by the operator and/or patient for the safe and effective use of the device and to prevent damage. All caution statements should be followed to ensure data and system integrity.</p>
<p>NOTE EN-NO</p> 	<p>Messages with this heading provide additional information that will increase the operator's understanding of the operation of the device.</p>

Symbols on the Skiin Underwear Device Label (Skiin Pod)		
Symbol	Meaning	Reference
MMWWWWWW 00001	Serial Number	
	Code for the Serial Number	
2AXVL-UW02	FCC Number	
26604-UW02	IC ID	
	Refer to Operating Instructions	ISO 15223-1, Clause 5.4.3 IEC 60601-1 symbol 10 in table D.2
	Type BF applied part	IEC 60601-1, Table D.1, Symbol 20 IEC 60417- 5333
1011-002-001V2	Model Number	

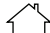Symbols on the Skin Underwear Package		
	Legal manufacturer.	ISO15223-1 symbol 5.1.1
	Device Serial Number.	ISO 15223-1 symbol 5.1.7
	Refer to Instruction Manual.	IEC 60601-1 Table D.2, Symbol 10
	Indicates authorization under the FCC Declaration of Conformity Procedure.	FCC title 47, Part 18.209 b (symbol shown in 47 CFR 15.19 b)
	Temperature Range Limitation.	ISO 15223-1 symbol 5.3.7
	Do not use the Skin Underwear if its packaging has been damaged.	ISO 15223-1 symbol 5.2.8
	Non Sterile.	ISO 15223-1, Clause 5.2.7 ISO 7000-2609
	MR Unsafe. Items should not enter the MRI scanner room. Patients with MR Unsafe devices should not be scanned.	ASTFM F2503
	Non-ionizing Radiation.	ISO 7010

	When disposing of this equipment it must be sent to the appropriate facilities for recovery and recycling.	ANNEX IX of directive 2012/19/EU for WEEE
	Voltage/frequency/current standard For NA (US and Canada): 100-240 VAC 50/60Hz 600VA.	Alternating current is indicated using symbol 1 table D.1 in IEC 60601-1
	The Safety Test Symbol is issued for the application of the selected test criteria.	Logo of the company that performs electrical safety tests, TUV.

Symbols on the Wall Plug		
	Indicates authorization under the FCC Declaration of Conformity procedure.	FCC title 47, Part 18.209 b (symbol shown in 47 CFR 15.19 b)
	UL Classification Marks for Canada and the United States.	60950-1
	When disposing of this equipment it must be sent to the appropriate facilities for recovery and recycling.	ANNEX IX of directive 2012/19/EU for WEEE
	External Power Supply energy efficiency standard.	U.S. Department of Energy, International Efficiency Marking Protocol
	For indoor use only.	IEC 60335-2-29
	Class II equipment.	IEC 60051-1

Source	Full Name of Standards
ISO 15223-1	IEC 60601-1:2012-Ed.3.1 Medical Electrical Equipment Part 1: General Requirement for Basic Safety and Essential Performance.
IEC 60417-5333	IEC 60601-1-2:2014-Ed.4.0 Medical Electrical Equipment Part 1: General Requirement for Basic Safety and Essential Performance, Collateral standard Electromagnetic Compatibility (EMC) Requirement and Test.
ASTM F2503	IEC 60601-1-11:2015 Medical Electrical Equipment – General Requirements for Basic Safety and Essential Performance – Collateral Standard: Requirements for Medical Electrical Equipment and Medical Electrical Systems used in the Home Healthcare Environment.
IEC 60335-2	IEC 60601-2-47:2012 © 2016 Medical Electrical Equipment Part 2-47: Particular Requirements for the Basic Safety and Essential Performance of Ambulatory Electrocardiographic Systems.
IEC 60051-1	IEC60601-1-6:2013 Medical Electrical Equipment - Part 1-6: General Requirements for Basic Safety and Essential Performance - Collateral Standard: Usability.

15. Warnings

EN-W6		Risks for children and individuals requiring assistance. This device is not intended for use by children under the age of 12. This device is not intended for use by individuals with limited physical capacities or limited experience and / or knowledge unless they are supervised by an individual responsible for their safety or receive instructions from the latter as to how the device should be used.
EN-W7		<u>Contraindication</u> The Skiin Underwear is not for use by: Persons under 12 years of age. Pacemaker and/or (ICD) implantable cardioverter defibrillator users.
EN-W8		<u>DO NOT expose the device to a magnetic resonance (MR) environment.</u>
EN-W9		Do not ingest the Skiin Pod, Skiin Charging Cable, Skiin Wall Charger, Skiin Charging Dock.
EN-W10		Warning: If you have an allergic or skin reaction, please discontinue use and contact customer support at support@skiin.com . If necessary, you should seek medical attention.
EN-W11		Do not operate the device if the device feels hot to touch.

16. Cautions

EN-C3		Remember to remove the Pod from your underwear before washing.
EN-C4		Do not drop the device. It is important that the user takes care when handling the Pod. Repeated drops from 1m or over may damage and/or affect the performance of the Pod.
EN-C5		DO NOT use in the presence of flammable anesthetics, drugs or pressurized oxygen.
EN-C6		The Skiin Pod contains magnets, use with caution around magnetic strips and metallic objects.

17. General Safety Precautions

Explosion Hazard

WARNING EN-W12		The equipment is not designed to be used in potentially explosive environments. Do not operate the Skiin Underwear in the presence of flammable liquids or gases.
--------------------------	---	---

Electrical Safety

CAUTION EN-C7		Do not spray or drip any liquid onto the Skiin Underwear's Pod, as this could affect reliable operation.
CAUTION EN-C8		Dispose of the Skiin Pod and Skiin Garments according to policies set out by your local governing authority.
EN-W13		Do not use the Skiin Pod, or Skiin Charging Kit or Skiin Underwear if they are damaged in any way.
EN-W14		Do not put your finger into the Skiin charger.
EN-W15		Do not operate the device while wet or moist.
EN-W16		Do not hold the Skiin Pod, the USB Charging Dock, the Micro-USB Cable and the Wall plug directly to skin for extended periods of time.

18. Other

FCC Regulatory Statements

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference.
- (2) This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the Federal Communication Commission (FCC) rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment causes harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by doing one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE: THE GRANTEE IS NOT RESPONSIBLE FOR ANY CHANGES OR MODIFICATIONS NOT EXPRESSLY APPROVED BY THE PARTY RESPONSIBLE FOR COMPLIANCE. SUCH MODIFICATIONS COULD VOID THE USER'S AUTHORITY TO OPERATE THE EQUIPMENT.

RF Exposure Warning

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

ISED Regulatory Statements

This device complies with ISED Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

CAN ICES-3 (B) / NMB-3 (B)

RF Exposure Information

This equipment complies with ISED RSS-102 radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Table 1 – Emission Class and Group Compliance

The Skiin Underwear is intended for use in the electromagnetic environment specified below. The customer or the user of the Skiin Underwear should assure that it is used in such an environment.		
Emissions Test	Compliance	Electromagnetic Environment – Guidance
RF Emissions CISPR 11	Group 1	The Skiin Underwear uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions CISPR 11	Class B	The Skiin Underwear is suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	Class A	
Voltage Fluctuations/ Flicker Emissions IEC 61000-3-3	Complies	

Table 2 – Example Immunity Test Level Compliance

<p>The ME EQUIPMENT is intended for use in the electromagnetic environment specified below. The customer or the user of the ME EQUIPMENT should assure that it is used in such an environment.</p>			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment – Guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	±8 kV Contact ±15 kV Air	±8 kV Contact ±15 kV Air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical Fast Transient/Burst IEC 61000-4-4	±2 kV for power supply lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±0.5 kV, ±1 kV line to line	±0.5 kV, ±1 kV line to line	Mains power quality should be that of a typical commercial or hospital environment.

<p>Voltage Dips, short interruptions and voltage variations on power supply input lines</p> <p>IEC 61000-4-11</p>	<p>0% U_T (100% dip in U_T) for 0,5 cycle</p> <p>0% U_T (100% dip in U_T) for 1 cycles</p> <p>70% U_T (30% dip in U_T) for 25 cycles</p> <p>0% U_T (100% dip in U_T) for 5 sec</p>	<p>0% U_T (100% dip in U_T) for 0,5 cycle</p> <p>0% U_T (100% dip in U_T) for 1 cycles</p> <p>70% U_T (30% dip in U_T) for 25 cycles</p> <p>0% U_T (100% dip in U_T) for 5 sec</p>	<p>Mains power quality should be that of a typical commercial or hospital environment. If the user of the ME EQUIPMENT requires continued operation during power mains interruptions, it is recommended that the ME EQUIPMENT be powered from an uninterruptible power supply or a battery.</p>
<p>Power Frequency Magnetic Field (50/60 Hz)</p> <p>IEC 61000-4-8</p>	<p>30 A/m</p>	<p>30 A/m</p>	<p>Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.</p>
<p>NOTE U_T is the a.c. mains voltage prior to application of the test level.</p>			

Table 3 – Example Immunity Test Level Compliance

The ME EQUIPMENT is intended for use in the electromagnetic environment specified below. The customer or the user of the ME EQUIPMENT should assure that it is used in such an environment.			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment – Guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz 6 Vrms ISM/Amateur Radio bands inside 150 kHz to 80 MHz	3 Vrms 150 kHz to 80 MHz 6 Vrms ISM/Amateur Radio bands inside 150 kHz to 80 MHz	Portable and smart RF communications equipment should be used no closer to any part of the ME EQUIPMENT including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.2\sqrt{P}$ $d = 1.2\sqrt{P}$ 80 MHz to 800 MHz $d = 2.3\sqrt{P}$ 800 MHz to 2.7 GHz
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,7 GHz RF communication equipment inside 80 MHz to 6 GHz	3 V/m 80 MHz to 2,7 GHz RF communication equipment inside 80 MHz to 6 GHz	where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a should be less than the compliance level in each frequency range ^b
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land smart radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the ME EQUIPMENT is used exceeds the applicable RF compliance level above, the ME EQUIPMENT should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the ME EQUIPMENT. b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

Table 4 – Example Recommended Separation Distances

Recommended separation distances between portable and smart RF communications equipment and the ME EQUIPMENT			
The ME EQUIPMENT is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the ME EQUIPMENT can help prevent electromagnetic interference by maintaining a minimum distance between portable and smart RF communications equipment (transmitters) and the ME EQUIPMENT as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter w	Separation distance according to frequency of transmitter		
	150 kHz to 80 MHz $d=1.2\sqrt{P}$	80 MHz to 800 MHz $d=1.2\sqrt{P}$	800 MHz to 2.7 GHz $d=2.3\sqrt{P}$
0.01	0.12	0.12	0.24
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
<p>For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.</p> <p>NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			

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