



MYANT VIRTUAL CLINIC PORTAL (MVCP)


QUICK START GUIDE FOR PHYSICIANS

1.

Accessing and navigating the MVCP

You will receive an invitation email from your clinic's admin.
Follow the link to complete the sign up: Enter your Name and set password.
To log onto the MVCP, visit <https://scwp.skiinserver.com/>

Navigate to key areas using the blue sidebar.
The sidebar will expand when hovered over.

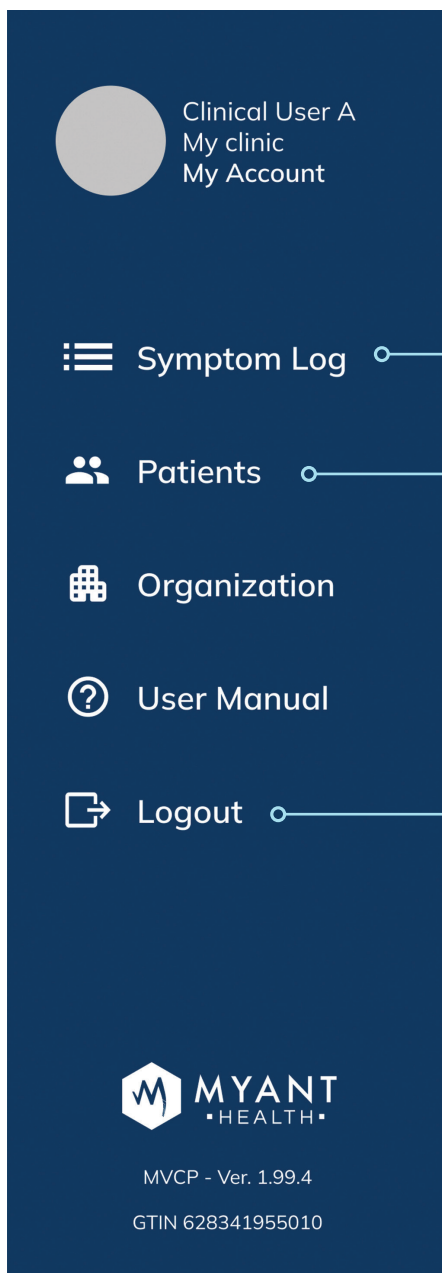

Symptom log: List of recent symptoms logged by your patients.

Patients: List of all patients using Skiin that are registered with your clinic.

Logout: To securely exit the MVCP.



IMPORTANT

The MVCP is not used for diagnostic ECG analysis, Holter applications and interpretation. The MVCP should not be used with pediatric patients and patients admitted for hospital care.

Please read the indications for use, contraindications, intended use, and safety information by visiting <https://myanthealth.com/pages/web-portal-user-manual>

2.

Symptom logs

Event Log

Review events recently reported by patients.

Filter by Date MD To Review

Page 1 of 4 > >|

PATIENT NAME	SYMPTOM DATE	TIME ↓	STATUS
Patient A	2022-11-23	16:02	<input type="button" value="MD To Review"/>
Patient B	2022-11-23	15:32	<input type="button" value="MD To Review"/>
Patient A	2022-11-23	14:45	<input type="button" value="MD To Review"/>
Patient C	2022-11-23	13:56	<input type="button" value="MD To Review"/>
Patient D	2022-11-23	13:39	<input type="button" value="MD To Review"/>
Patient E	2022-11-23	11:47	<input type="button" value="MD To Review"/>

Reviewing symptom log workflow

- 1- Patient logs symptom on the app
- 2- Symptom log is reviewed by cardiac tech on the MVCP
- 3- If review by an MD is needed, the cardiac tech would mark status as “MD to Review”

As an MD, by default, you will see logs marked for your review.

To view all logs:

- 1- Click on the “MD to Review” dropdown
- 2- Select “All Status”

Also, by default, only logs reported in the last 30 days will be displayed.

To view older logs:

- 1- Click on the “Filter by Date” dropdown
- 2- Select a date from the calendar

To clear the date filter, click “Reset”.

2.1 Viewing a symptom log

Clicking on a symptom log directs you to a page that displays additional information relating to the selected log, as seen below.

Each of these sections will be explained further in the subsequent pages.

Patient's symptom logs

Patient A Verified

Date of Birth: 1969-08-11 | Gender: M | Health card number: 1111-222-333 | Version: PP | Email: patienta@gmail.com | Physician: Clinician B

SYMPTOM LOG | PATIENT PROFILE | PATIENT METRICS | PROGRAM | CONSULTATIONS

Event Log Page 1 of 4 > |

DATE	TIME	STATUS	LAST UPDATED	
07-04-2022	12:24	MD To Review	07-04-2022, 12:24	Holter dates
07-04-2022	10:21	Reviewed by MD	07-04-2022, 10:21	Holter dates
21-03-2022	15:42	Reviewed by MD	22-03-2022, 08:30	Holter dates

Symptom details

Palpitations 8/10

Duration: 10m

Possible triggers: Stress

Patient's note
Felt palpitations during lunch. Morning was hectic with deadlines

ECG Viewer

Amplitude: 10mm/mv | Date: 07-04-2022 | Time: 12 : 24 | **Update**

12:24:25 - 12:24:30 PM

Garment type: Skin-underwear

5 seconds (10mm/mv) - Channel 1

5 seconds (10mm/mv) - Channel 2

5 seconds (10mm/mv) - Channel 3

Metrics

- Heart Health**
Heart Rate: 91 bpm
- Activity**
Posture: Resting
- Steps**
60 steps
- Core Body Temperature**
37.1 °C

Analysis

Rhythm

<input type="checkbox"/> 1AVB	<input type="checkbox"/> IRD	<input type="checkbox"/> Other
<input type="checkbox"/> 2AVB1	<input type="checkbox"/> IVR	<input type="checkbox"/> Pause
<input type="checkbox"/> 2AVB2	<input type="checkbox"/> JR	<input type="checkbox"/> SA
<input type="checkbox"/> 3AVB	<input type="checkbox"/> MAT	<input type="checkbox"/> SR
<input checked="" type="checkbox"/> AF	<input type="checkbox"/> NNS	<input type="checkbox"/> SVT
<input type="checkbox"/> AFL	<input type="checkbox"/> NNV	<input type="checkbox"/> VFib
<input type="checkbox"/> BBB	<input type="checkbox"/> NS	<input type="checkbox"/> VT
<input type="checkbox"/> EAT	<input type="checkbox"/> NV	<input type="checkbox"/> WAP

Interpretation
Rhythm irregular. 3 instances of AFib.

Status
MD To Review | **Save**


Symptom details •
3-channel ECG viewer •
Wellness metrics •

Analysis

2.2 ECG viewer

The ECG viewer displays waveforms across three channels if the patient was wearing their Skiin garment at the time of symptom.

By default, ECG from the time of symptom will be displayed. To jump larger timescales, you may directly input a date and time.

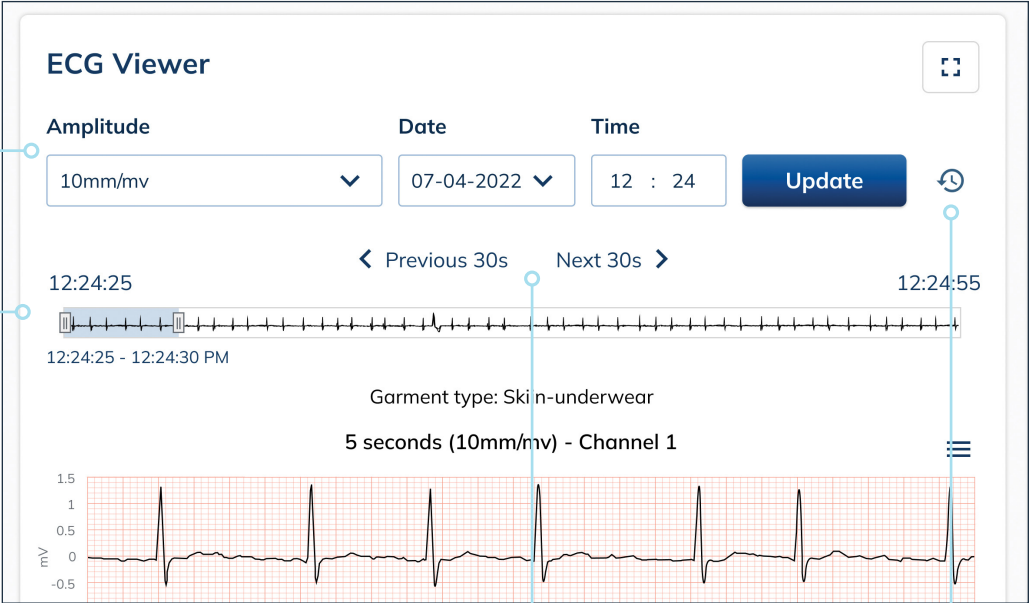

The screenshot shows the ECG Viewer interface with the following components and annotations:

- Amplitude scaling and temporal controls:** A callout points to the 'Amplitude' dropdown (set to 10mm/mv), the 'Date' dropdown (set to 07-04-2022), the 'Time' dropdown (set to 12 : 24), and the 'Update' button.
- Scrubber:** A callout points to the scrubber bar, which is a horizontal timeline with a blue highlight and a vertical playhead. The timeline is labeled with '12:24:25' on the left and '12:24:55' on the right. Below the scrubber, it says '12:24:25 - 12:24:30 PM'.
- Navigation:** Callouts point to the '< Previous 30s' and 'Next 30s >' buttons, and a circular refresh icon on the right side of the scrubber.
- ECG Strip:** The main display shows a 5-second ECG waveform on a grid. The y-axis is labeled 'mV' and ranges from -0.5 to 1.5. The x-axis is labeled '5 seconds (10mm/mv) - Channel 1'. Above the strip, it says 'Garment type: Skin-underwear'.

2.3 Symptom details & contextual wellness metrics

As part of a symptom log, the patient may have provided additional details regarding their experience.

If the patient was wearing their Skin garment at the time of the symptom, you may view contextual wellness metrics. These metrics pertain to the 30s timeframe seen in the ECG viewer and will update automatically as you navigate every 30s.

The screenshot displays a symptom log interface. On the left, the 'Symptom details' section includes 'Palpitations' (8/10), 'Duration' (10m), 'Possible triggers' (Stress), and a 'Patient's note' stating 'Felt palpitations during lunch. Morning was hectic with deadlines'. The center shows a blurred ECG waveform. On the right, the 'Metrics' section displays: Heart Health (Heart Rate 91 bpm), Activity (Posture Resting), Steps (60 steps), and Core Body Temperature (37.1 °C).

2.4 Analysis

Analysis section at the bottom of each symptom log, for use by cardiac techs and MDs.

The screenshot shows the 'Analysis' section. Under 'Rhythm', there are checkboxes for 1AVB, 2AVB1, 2AVB2, 3AVB, AF (checked), AFL, BBB, EAT, IRD, IVR, JR, MAT, NNS, NNV, NS, NV, Other, Pause, SA, SR, SVT, VFib, VT, and WAP. The 'Interpretation' box contains the text 'Rhythm irregular, 3 instances of AFib.'. The 'Status' dropdown is set to 'MD To Review', and there is a 'Save' button.



IMPORTANT






For billing and record keeping purposes, once you (MD) review a symptom log:

1- Update status to "Reviewed by MD"

2- Click "Save"

2.4 Analysis - Continued

1AVB: Atrioventricular Block 1st degree	NNS: Supraventricular Trigeminy
2AVB1: Atrioventricular Block 2nd degree Type I	NNV: Ventricular Trigeminy
2AVB2: Atrioventricular Block 2nd degree Type II	NS: Supraventricular Bigeminy
3AVB: Complete Heart Block	NV: Ventricular Bigeminy
AF: Atrial Fibrillation	Other: Other
AFL: Atrial Flutter	Pause: Pause
BBB: Bundle Branch Block	SA: Sinus Arrhythmia
EAT: Ectopic Atrial Tachycardia	SR: Sinus Rhythm
IRD: Isorhythmic Dissociation	SVT: Supraventricular Tachycardia
IVR: Idioventricular Rhythm	VFiB: Ventricular Fibrillation
JR: Junctional Rhythm	VT: Ventricular Tachycardia
MAT: Multifocal Atrial Tachycardia	WAP: Wandering Atrial Pacemaker

Icon & Status	Description
	Symptom newly logged by a patient, and is pending review by Cardiac Tech
	Symptom was updated by patient after initial logging, and is pending review by Cardiac Tech
	Symptom marked by Cardiac Tech for further review by MD
	Symptom reviewed by MD
	Symptom reviewed by Cardiac Tech




MYANT
HEALTH



MYANT VIRTUAL CLINIC PORTAL (MVCP) Quick Start Guide for Physicians

You're All Set!

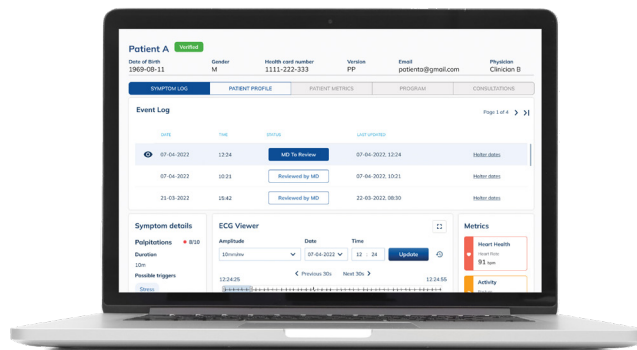

Get help at www.myanthealth.com
or contact us at: +1 (416) 423-7906 or info@Myant.ca

Myant Inc.

200 Ronson Drive, Suite 500, Etobicoke, ON | M9W 5Z9 Canada | +1 (416) 423-7906
info@Myant.ca | www.myanthealth.com