

1. SETTING UP THE QH2 HUB FOR CORDED TELEPHONES

For full manual visit www.clearsounds.com



STEP 1: Remove the curly cord from the phone base and connect it to the HANDSET port on the bottom of the hub.



STEP 2: Connect the short Hub-to-base cable to the phone base port where the curly cord had been connected.



STEP 3: Connect the unattached end of the Hub-to-base cable to the BASE port on the top side of the hub. The green light will illuminate.



STEP 4: Connect the round end of the AC power adapter to the DC 9V port on the left side of the hub, then plug the other end of the AC adapter into a power outlet in the wall.



STEP 5: Pick up the telephone handset. **If you hear a dial tone:** On the hub, slide the VOLUME slider or TONE slider until you hear a noticeable change in volume and tone quality.



STEP 6: **If you do not hear a noticeable dial tone:** On back of QH2, click the dial until you hear a dial tone. On the hub, slide the VOLUME slider or TONE slider until you hear a change in volume and tone quality. Repeat these steps until you find a compatible setting.

2. TO PAIR THE QH2 HUB WITH A BLUETOOTH HEADSET, STREAMER OR NECKLOOP



STEP 1: Wear your Bluetooth listening device and remain within 33 feet of the QH2 hub.



STEP 2: On the hub, press and hold the multi-function button (MFB) until the blue and red lights flash.



STEP 3: Turn on your listening device, then place it in Bluetooth pairing mode. When hub and listening device are paired, the hub flashes a blue light every three seconds. Your listening device also indicates when pairing is successful.



STEP 4: Confirm that you hear a dial tone in the handset by manually lifting the handset from the base.



STEP 5: On the hub, press and release the MFB to hear the dial tone in your Bluetooth headset, neckloop or streamer.



STEP 6: Press and release the MFB. The dial tone stops. Manually return the handset to the base.

3. TO MAKE OR ANSWER A CALL USING A CORDED PHONE



STEP 1: Wear your Bluetooth listening device and remain within 33 feet of the QH2 hub.



STEP 2: Manually lift the handset from the base.



STEP 3: On the hub, press and release the MFB. You will hear a dial tone or the caller in your listening device. Dial or talk.



STEP 4: Slide the TONE slider on the hub until you hear the tone that you prefer.
[OPTIONAL]



STEP 5: To end the call, press and release the MFB.



STEP 6: Manually hang up the handset.

Please Note: You will need to manually remove the handset from the base to make a call and replace it to hang up.

OPTIONAL LIFTER SET UP FOR THE QH2 HUB *Lifter not included*



STEP 1: Remove the handset from the phone base.



STEP 2: On the underside of the lifter, remove the paper backing from the two pieces of tape.



STEP 3: Firmly press the bottom of phone base lifter, directly below the earpiece, so that it lies beneath the handset arm.



STEP 4: Connect the L-shaped connector to the LIFT port on the bottom of hub...



STEP 5: ...then connect the other end of the cable firmly to the port on the top of the lifter. Look for a red glow the on lifter.

TO MAKE OR ANSWER A CALL USING A CORDED PHONE WITH HANDSET LIFTER CONNECTED



STEP 1: Wear your Bluetooth listening device and remain within 33 feet of the QH2 hub.



STEP 2: On the hub, press and release the MFB. The lifter raises the phone handset and you hear a dial tone or the caller in your listening device. Dial or talk.



STEP 3: Slide the TONE slider on the hub until you hear the tone that you prefer
[OPTIONAL]



STEP 4: To end the call, press and release the MFB and the lifter returns the phone handset to the base.

Please Note: Bluetooth connection required for lifter.