



INSTRUCTIONS FOR USING THE UNODOSE™ APPLICATOR



1 Remove the protective cover with a light counter-clockwise twist and pull while holding the barrel.



2 Turn the dosing knob in the direction marked for the number of prescribed clicks.



NOTE:
Be sure to leave the knob in a "lock" position, and not between click positions.



3 Confirm that cream has exited the holes of the applicator cap.



4 Apply the cream to the skin absorption site. Rub with the applicator cap until the cream is evenly spread (per physician or pharmacist direction).
Replace the protective cover with a light push and clockwise turn. You will hear and feel a click.



5 Check the scale at the blue line for the number of clicks remaining.
Refill your prescription as instructed by your pharmacy.

PRECAUTIONS:

- Store in a dry place at room temperature.
- Avoid exposure to sunlight and other bright lights.
- When traveling in an airplane or to a different elevation, seal the applicator in a zip lock bag to prevent soiling from any leakage due to pressure changes.

CUSTOMER SERVICE:
651.788.7907
Uno-Dose.com



Metered-Dose
Topical Applicator

COMPOUNDING IN THE APPLICATOR



1 Transfer half of the desired cream into the applicator in several steps using a spatula, ointment jar, or filling system. **Cover all of the bottom** of the barrel, being sure to fill the corners. To remove the air, **tap the bottom** of the applicator on your palm or a countertop as you go.



2 Add the drug* to the barrel.
NOTE: Do not allow liquid drug to flow past the cream surface to the bottom of the barrel.
** Drug has been tinted with color for demonstration purposes only.*



3 Add the rest of the cream to the barrel while removing all air.
NOTE: Cover the drug completely with cream. Do not allow liquid drug to flow to the top of the cream.



4 Attach the disposable mixing paddle to the electronic mortar and pestle (EMP machine) shaft, and insert the shaft into the mixing cap.
NOTE: A standard reusable mixing blade may also be used.



5 Insert the mixing paddle into the barrel and **plunge it to the bottom**. Then, while holding the barrel, **screw the mixing cap** onto the barrel.



6 Program and prepare the EMP machine, then attach the UnoDose™ Applicator to it. **Start** the mixing cycle.
NOTE: Observe the mixing paddle through the wall to confirm complete mixing, top to bottom, of the barrel contents.



7 When the cycle completes, **remove the applicator** and **unscrew the cap**.



8 Wipe the mixing paddle blades on the rim of the barrel, if needed, to transfer any cream remaining on the blades into the applicator. **Remove and discard the paddle**.
NOTE: The cream should show signs of proper mixing action.

Refer to the "FILLING THE APPLICATOR" instructions to:

- Attach the applicator cap
- Prime the applicator
- Wipe away excess cream
- Attach the protective cover
- Apply a prescription label

CUSTOMER SERVICE:
651.788.7907
Uno-Dose.com



Metered-Dose
Topical Applicator

FILLING THE APPLICATOR



1 Transfer the desired amount of compounded cream into the UnoDose™ Applicator barrel using a spatula, ointment jar, or filling system.



2 For best results, remove air during filling. Start with a small amount to **fill the corners** in the bottom of the barrel. **Tap the bottom** of the applicator on your palm or a countertop to settle the cream and remove air.



3 Add the remaining cream in several steps while continuing to **tap the bottom** of the applicator to settle the cream and remove air. Leave the top of the cream flat and smooth.



4 Put the applicator cap on the barrel and **push downward to snap** into place. The applicator cap will butt up against the ring on the barrel. **Tap the bottom again** and confirm that no air remains in the cream.



5 Prime to remove air. For a fast prime, use the **prime tool supplied in each box**. Push slowly until cream just comes out the holes, or the tool stops.
NOTE: The short end of the tool leaves 32 usable milliliters in the applicator; the long end leaves 17.




6 Finish priming by turning the **dosing knob** several clicks until cream exits all holes smoothly and equally.
NOTE: Be sure to leave the knob in a "lock" position, and not between clicks.



7 Wipe away the cream from the applicator cap, then **lock the protective cover** in place with a light push and clockwise turn. You will hear and feel a click.



8 Confirm the correct volume on the scale by **reading at the blue line**.
NOTE: The scale shows the usable volume and clicks remaining, and excludes the residual volume of 1.5 milliliters.

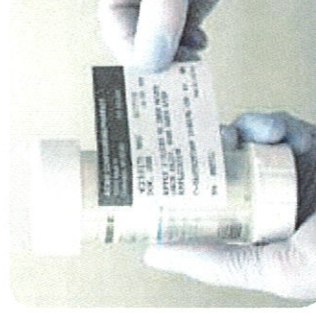

9 Apply a prescription label to the barrel.
NOTE: Do not cover the CLICKS scale.

CUSTOMER SERVICE:
651.788.7907
Uno-Dose.com