



Metered-Dose  
Topical Applicator



## USING THE UNODOSE™ APPLICATOR



Remove the cover with a light counter-clockwise twist and pull while holding the barrel. You will hear and feel a click.



Turn the dosing knob for the prescribed number of clicks.

*NOTE: Leave the knob in a "lock" position, not between clicks.*



Confirm that cream has come out the holes of the applicator cap.



Apply cream to the skin.  
Rub with the applicator cap until evenly spread (per physician or pharmacist direction).  
Replace the cover.



Check the scale at the blue line for the number of clicks remaining.  
Refill as instructed by your pharmacy.

### PRECAUTIONS:

- Store in a dry place at room temperature.
- Avoid exposure to sunlight and other bright lights.
- When traveling in an airplane or to a different elevation, seal the applicator in a zip lock bag to prevent soiling from any leakage due to pressure changes.