



FACE SHIELD FITTING INSTRUCTIONS



Usage

The Face Shield is intended to be worn by HealthCare Professionals to protect the face from liquid splashes and bodily fluids when treating infectious patients. It is not designed to be used as a respiratory device or to prevent any airborne infectious materials entering through the nose or mouth.

Always check the product for defects before use.

It is highly recommended that safety glasses/goggles and a face mask are used with the Face Shield when appropriate to further reduce the chance of liquid splashes and bodily fluids causing harm and add another level of protection against respiratory infections or diseases which may be present.

Maintenance and Servicing

The face shields are recommended to be used for short to mid term periods (6-72+ hours). The usage is dependent on application and user care and maintenance. See usage guidance below:

- NHS healthcare setting 6-8 hours. They should be disposed with medical waste after each usage period of 8 hours.
- Clinical medium risk 24-48 hours
- Retail low risk 48-72 hours
- Public very low risk 72hrs +.

Storage

To be stored at room temperature.



Medworx Face Shield
Model No: FM001
Classification: PPE Class I of REGULATION (EU) 2016/425
Manufacturer: Mauveworx

Mauveworx Ltd. Registered No England: 10989151
England Registered Office: 1 Nimrod Way, Wimborne BH21 7SH



Designed and
Manufactured in the UK

FITTING INSTRUCTIONS



Remove the Visor from the packaging. Wipe down the Visor before use.



Feed the long head strap through the large slot on the side of the Visor.




Put the Visor on your head and pull the head strap through the slot until its tight.



Holding the position of the strap with your fingers, remove the Visor and fold the strap over itself. Lock the semi circle tab into the slot.



Lock the second semi circle tab into the adjacent rectangle slot. **The strap should now be locked firmly into position by two semi circle tabs.**

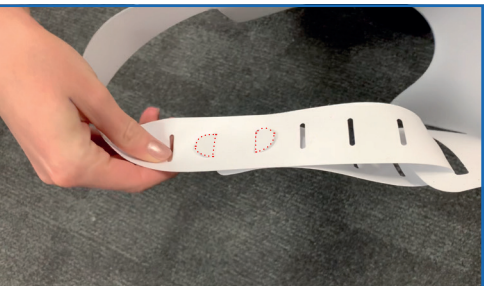

6 Once **both tabs** are locked into place the head strap will be secure. If the fit is still loose repeat the above process until you are comfortable you have a good fit.

Lastly sign your name in the white box on the front of the Visor.