

# Commode Chair

Thanks for using our product.  
Before using, please read the manual carefully.



Seat Height (Seat to floor height): 445-600mm


Net Weight: : 7. 2kg

Weight Capacity: 250kg

## Elements of the chair and manual for installation:



1. Seat cover
2. Armrest
3. Seat pan
4. Supporting bracket
5. Bucket
6. Chair leg
7. Back support

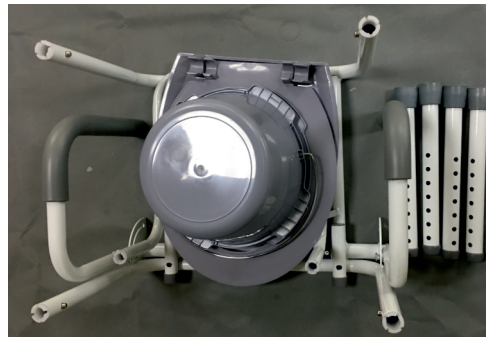


NO.1: Take all the items out of package.




No.2: Insert the leg tube into the supporting bracket in turn and push it to the marble pop.

Note: 4 leg tubes must be at the same level

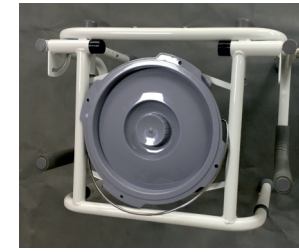

NO. 3: Put the bucket on the bucket frame. (as shown)

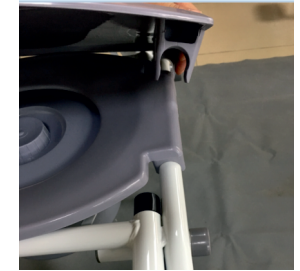

NO. 4: Align the toilet seat clamps onto the back frame rail (both clamps should sit outside the pale brackets). Push down on seat until clamps snap firmly into place. (as shown)



NO. 5: The seat pan and cover setup complete



NO. 6: Insert the backrest tube into the tube under the seat frame. Push to the marble to pop up.



NO.7: armrest opening operation. Lift the armrest tube to the button to pop up.



NO.8: Setup Complete State of serviceability