

# Shower chair with arms

Thanks for using our product.  
Before using, please read the manual carefully.



Diameter of the chair arm: 46.5cm

Chair back: 42\*18cm

Seat: 40\*34cm

Seat Height (Seat to floor height): 41-53.5cm

Net weight: 3.3kg

Weight Capacity: 450LBS


## Elements of the chair and manual for installation:



1. Chair back
2. Tube for chair back
3. Chair arm
4. Seat
5. Chair leg
6. Anti-slip pad for chair



1. Take all the items out of package.

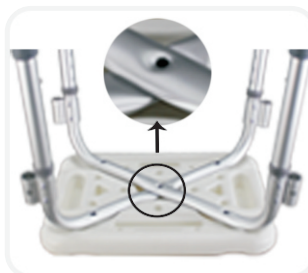

2. Put the seat on the seat frame and fix with short screws. (please make clearly which side is the back/front side of the seat)



3. Put the two straight tubes in the corresponding place and fix with long screws. (remember to use the plastic ring with the screw)

The dimension with 3 holes should face outside because it's used to connect to the tube of the chair back.



4. Fix the chair arm, should adjust the steal ball on the hole when install the arm.



5. Fix the chair back and the tube with short screw. (with plastic ring)



6. Put the tube of chair back into the tube of the seat, and adjust the steal ball.



7. Setup Complete;