

Deluxe bariatric shower chair

Thanks for using our product.
Before using, please read the manual carefully.



Seat : 510*305mm

Seat Height (Seat to floor height) : 360-485mm


Net Weight : 2.2kg

Weight Capacity : 226kg(500lbs)

Elements of the chair and manual for installation:




1. Seat
2. Leg Receivers
3. C-Clips
4. Chair leg
5. Cross Brace

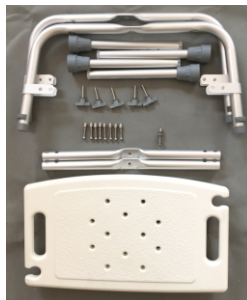

NO.1: Take all the items out of package.




NO.2: Put the seat on the seat frame and fix with short screws . (please make clearly which side is the back/front side of the seat)

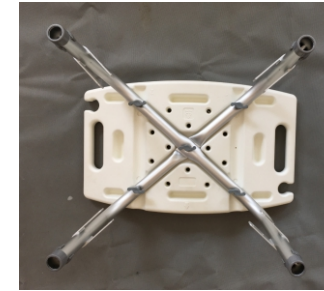


NO.3: Leg Receivers
Setup Complete



NO.4 Assemble the leg cross-brace
Take the two leg cross-braces and place one on top of the other with the grooves properly aligned.
Place 2 washers, one on both sides of the cross-braces. Insert screw through washer and into the center hole and secure with a nut.

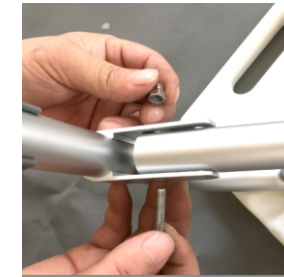

NO.5 Position the cross-brace assembly in between the four legs and align the cross-brace screw holes with the cross-brace bracket screw holes.
Insert a screw into each of the 8 screw holes and secure with a nut.

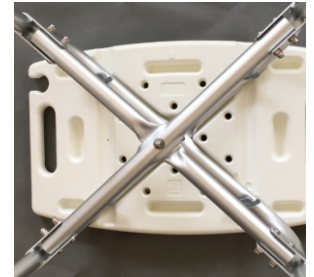

NO.6 Cross Brace
Setup Complete



NO.7: Insert a leg into each of the leg receivers use a C-shaped clip to secure each of the legs to the leg receiver.



NO.8: Setup Complete
State of serviceability