

Shower chair

Thanks for using our product.
Before using, please read the manual carefully.



Chair back: 41*17cm

Seat: 51*31 cm

Seat Height(Seat to floor height): 35.5-51cm

Chair back Height: 71-87cm

Net weight: 2.7kg

Weight Capacity: 450LBS

Elements of the chair and manual for installation:



1. Chair back
2. Tube for chair back
3. Seat
4. Chair leg
5. Anti-slip pad for chair




4. Fix the chair back and the tube with short screw. (with plastic ring)



1. Take all the items out of package.



5. The backrest tube is inserted into the seat plate under the straight hole, out of copper beads.

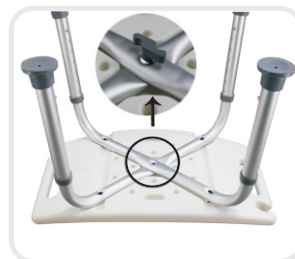

2. Put the seat on the seat frame and fix with short screws. (please make clearly which side is the back/front side of the seat)



6. Setup Complete;



3. Put the two straight tubes in the corresponding place and fix with long screws. (remember to use the plastic ring with the screw)

The dimension with 3 holes should face outside because it's used to connect to the tube of the chair back.

Thanks for your purchase. Any concerns please don't hesitate to contact us, we will resolve all the issues timely. Customers are always the best teachers for us to keep growing!