




# **Entry Ramp Instruction**

**Model No. TR-4IR**

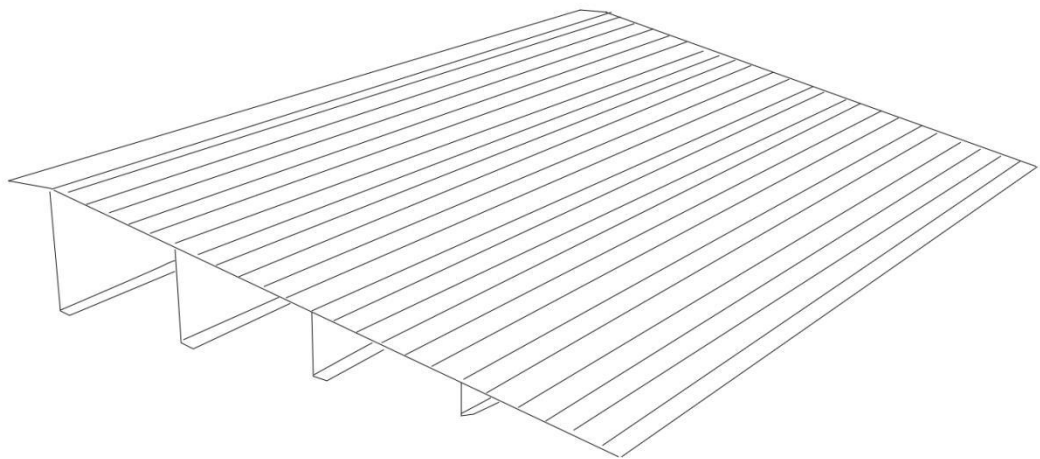


## Product Info

1. Material: Aluminum
2. Product Size: 34" L×22.2" W×4.25" H
3. Product Weight: 11 lbs/5 kg
4. Max. Loading Capacity: 700lbs

## Safety Instruction

1. The maximum weight capacity of the entry ramp is 700lbs. Please never exceed the maximum loading weight.
2. Please check whether all the parts have been secured properly before use.
3. Please make sure the ramp surface is totally dry before use. And also please never use the entry ramp by placing it on ground which is wet, icy or slippery.
4. Please use the ramp with the help of qualified health care workers or others.

## Parts

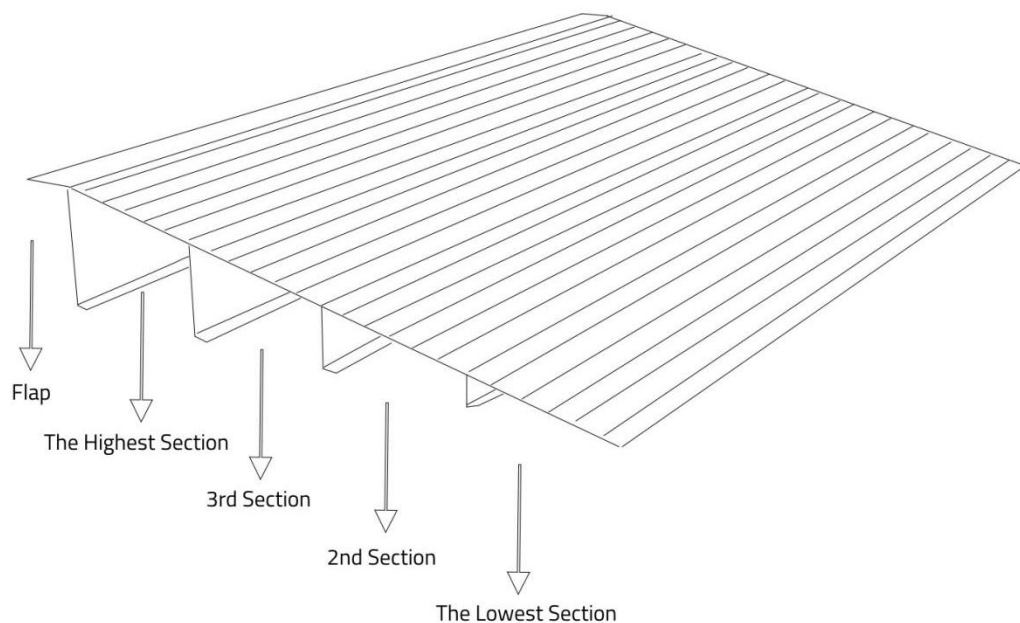

## **How to Assemble**

Line up the extended part in the 2nd section with the slot in the lowest section and then slide it into the slot. Assemble the rest sections in the same way.

Align the extended part in the flap with the slot in the highest section and then slide it into the slot.

Put the screws into the holes on the right side of the ramp and tighten the screws.

## **How to Use**

Put the entry ramp against the edge of the threshold and then place the flap on the threshold. And please make sure the flap is placed on a horizontal surface when you need to use it.

If the flap is not needed, you can take it off or just fold it downward.