



FOLDING WALKING CANE WITH LED LIGHTS & SOS ALARM

Operating Instructions

PRODUCT SPECIFICATIONS:

- Size of product itself:
 - folded length: 34cm (13.38")
 - unfolded length: 85-97cm (33.46"-38.19")
- Net weight of product itself: 485g
- Single package size: 16*10.5*35cm
- Net weight of product including package: 580g
- Materials: Aluminum alloy, ABS, TPR
- Any applicable Certifications: CE,GSG,FDA
- Battery Type: 2x AAA (not included)
- List of all the parts and pcs included per item:
 - 1x cane
 - 1x carry bag
 - 1x manual

USE

1. Remove all parts from packaging.
2. Connect all 3 part of the cane.
3. Unscrew the battery compartment cap (see photo) and insert 2 x AAA batteries (not included).
4. To turn the LED lamp on or off, simply press the On/Off button.
5. To adjust the lamp angle, tilt the front of the lamp up or down to achieve the desired position.
6. To turn the SOS Alarm on or off, simply press the On/Off button on the left side of the handle. An alarm will sound.
7. Fold the cane by gently disconnecting the 3 parts.

SAFETY AND CARE:

1. Danger of suffocation – keep all packing materials away from children and animals.
2. Never look directly at the LED light beam and do not point the beam at other people or animals.
3. Keep batteries out of reach of children and animals. If ingested, seek immediate medical attention.
4. Use only the specified battery size.
5. Always replace the batteries at the same time and be sure to use the same brand and capacity.
6. Remove batteries when depleted or when not in use for an extended period of time.
7. In case of a battery leak, remove immediately. In the event of any direct contact with battery acid, immediately flush the area with a generous amount of water and seek medical treatment.
8. When using the cane, keep a safe distance from heat sources, such as: stovetop, fire, lit candles.
9. Handle the cane with care. Avoid dropping or any aggressive use of the cane.
10. Avoid use in extreme temperatures, fluctuating temperatures, direct sunlight or moisture.
11. Protect the LED light from moisture and wetness.
12. To clean use a soft, damp cloth. Avoid any cleaning solutions or abrasive products.
13. If the cane breaks, do not attempt to repair.



DISPOSAL:

Prior to disposal, remove the batteries and dispose of according to Federal guidelines at designated collection points.

Dispose of the product in an environmentally friendly manner. This product is recyclable. Contact your local waste management authority for further instructions.

LIMITED LIFETIME WARRANTY:

Your Medic Therapeutics Folding Walking Cane with LED Lights & SOS Alarm is backed by a limited lifetime manufacturer's warranty. Medic Therapeutics will repair or replace your device at any time should it fail due to a defect in material or workmanship, subject to the certain limitations.

This limited warranty does not cover any damage that results from unauthorized or improper use, service, or repair. Further, it does not cover damage caused by accident, impact, negligence, or normal wear and tear. Should you discover your Medic Therapeutics Walking Cane is not functioning properly, please send your device to our repair center for evaluation. If your product cannot be repaired or serviced, we will reserve the right to change it for a similar or newer model.

Please note that a flat rate of \$8.00 will be charged to cover a service evaluation fee and return shipping of your device. All warranty claims must be accompanied by a copy of your proof of purchase from an authorized retailer. Please send your device, proof of purchase, and a check or money order in the amount of \$8.00 made out to Medic Therapeutics to:

Address:

Medic Therapeutics Service Center
3069 Taft Street
Hollywood, FL 33009

Contact:

warranty@medictherapeutics.com