



**MEDIC
THERAPEUTICS**

MULTI-FUNCTIONAL ADJUSTABLE WALKER

Instruction Manual

Usage and Maintenance Instructions



**N.B. This equipment must be installed by a competent person.
Do not exceed the weight limit stated - doing so could put the user at risk.**

BEFORE USE

- Carefully remove all packaging. Avoid use of any knives or other sharp instruments as they may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a default, do not use your product and contact your supplier for support.

INTENDED USE

Your Walker has been designed to assist when walking and carrying small items of shopping. The bag was designed specifically to fit your Walker, please only use the supplied bag for your Walker. Do not use or combine bags and/or baskets from other manufacturers.

When not in use, simply fold the Walker and store. The Walkers are designed to be used both inside and outside the home.

ASSEMBLY INSTRUCTIONS

Carefully remove all packaging, including the cardboard tubes covering the knurled tube part of the brake assembly. Check the product thoroughly for damage or obvious faults.



! If you suspect a default, please do not use, and contact your supplier immediately for service.

COMPONENTS OF THE WALKER



- 1 Push handle
- 2 Brake lever
- 3 Front wheel folding mechanism knobs
- 4 Backrest
- 5 Mesh seat
- 6 Personal Item Bag
- 7 Rear wheel folding mechanism knobs
- 8 Front wheel
- 9 Rear wheel
- 10 Cane holder

ASSEMBLY PROCEDURE

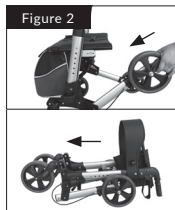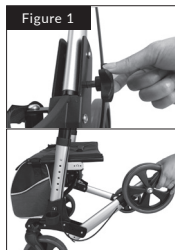

PART 1 - UNFOLD YOUR WALKER

1. Undo hook & loop, remove rear wheel folding mechanism knob. (Figure 1)
2. Rotate the rear wheel arms backward. (Figure 2)
3. Insert and tighten rear wheel folding mechanism knobs firmly, ensuring proper alignment.
4. Remove front wheel folding mechanism knobs. (Figure 1)
5. Rotate the front wheel arms forward. (Figure 2)
6. Insert and tighten front wheel folding mechanism knobs firmly, ensuring proper alignment.
7. Pull the two push handles apart.
8. Open the frame until the crossbar locking mechanism locks automatically, verify seat is locked in place by pushing down on both seat edges. (Figure 3)



CAUTION - Make sure the crossbar locking mechanism and rear wheel folding mechanism knobs locked firmly before use.

ASSEMBLY PROCEDURE


Figure 4



PART 2 - FOLDING

Remove the accessories and lower down the handle if necessary. Lift the crossbar locking mechanism to release belt on the mesh seat to release the crossbar lock. Fold the frame as shown on left (Figure 4). Push the two push handles together. Release the front and rear wheel folding mechanism knobs and rotate the rear wheel arms inward, replace folding mechanism knobs and attach hook and loop to rear legs.

Figure 5

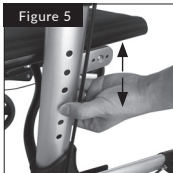

PART 3

Adjusting the height level of the handles to fit different users. Press the buttons for handgrips and pull the handgrip upwards or downwards. As the handle moves, release the buttons. This automatically locks in the next possible height. Repeat this process until the desired height is reached. (Figure 5)

Figure 6




PART 4

Attach the bag to the hook of the seat frame. The Personal Item Bag can be easily detached when using. (Figure 6)

OPERATING THE BRAKES

Figure 7

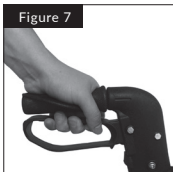

1. To stop, gently apply a gripping pressure to both brakes simultaneously (Figure 7), this will slow down and stop your Walker.

2. The brakes on your Walker have a park option (Figure 8). To operate the park option, push downwards on both brakes simultaneously. You will feel the brake make a very slight click. Your brakes are now in the 'lock' position.


Figure 8



3. To unlock, simply pull up the brake again, you will hear a slight click as the 'park' disengages.

BRAKE ADJUSTMENT

Figure 9

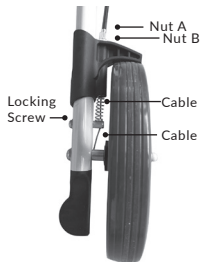

Your Walker brake system will require periodic adjustment. To adjust the brakes, loosen Nut B and then screw/unscrew Nut A. When adjusted re-tighten Nut B. If further adjustment is required then loosen the Locking Screw and then using a pair of pliers pull the cable down by approximately a millimeter. Re-tighten Locking Screw. The brakes can then be adjusted further by loosening Nut B and screwing/unscrewing Nut A as before (Figure 9)

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components.

This includes:

- Bent or damaged tubes or cracks at the tube joints.
- Loose or worn horseshoe clips
- Elongated adjustment holes.
- Loose handles.
- Loose or worn Wheels and wheel fasteners (where fitted)
- Smooth, worn or split ferrules (rubber tips).

Replace all worn, missing or damaged parts where necessary. If in doubt, please do not use and immediately contact your supplier for service support.

CARE AND MAINTENANCE & YOUR OBLIGATION

1. For intended use only as a walking aid. Your Walker should only be used on standard pavements or at home. It should not be used on rough terrain or pot-holed surfaces. The use of your Walker on these surfaces can seriously compromise your safety and may damage your Walker beyond repair.
2. Keep your Walker clean by wiping down with a damp cloth and mild detergent. Do not under any circumstances use an abrasive or solvent based cleaner as this will seriously damage the Walker and void your warranty.
3. Wipe off all surfaces with a dry cloth and use a water dispersing agent such as WD40 on all metal joints.
4. Check that all wheels and the folding mechanism function correctly and that all wheels move freely. Pay particular attention to the front wheels, ensuring that they rotate freely and without obstruction and that they are attached securely to the frame.



Withdraw your Walker immediately if any of the wheels become difficult to turn or do not feel securely attached to the frame or if you suspect damage or any other fault.

5. Monthly Checks - check that all locking pins are properly secure . Check that no components are loose. Check the adjustment to your brakes. Check castors.
6. Do not use any parts or accessories to your Walker manufactured by another company. To do so will annul your warranty and may seriously compromise your safety.
7. We suggest that you check your Walker visually every day. We recommend that you should have your Walker serviced every three months or less if used by, or issued to, various users.

CLEANING

Clean your Walker using a non-abrasive cleaner or mild detergent and a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may seriously damage the product beyond repair and should not be used. Always ensure that you wipe the equipment dry after cleaning.

MAINTENANCE

Please conduct a safety check of the product at regular intervals or if you have any Concerns.

PRODUCT SPECIFICATIONS

- Size of product itself: L 770 x W 665 x H 815-940 (mm)
- Net weight of product itself: 8.5kg
- Single package size: L635 x W280 x H340 (mm)
- Net weight of product including package: 9.4 kg
- Materials: Aluminum frame + PA plastic part + PVC wheel
- Certifications: (FDA, UL, ETL) CE MDR, ISO13485, ISO9001, BSCI, SGS test report (11199:2-2005)
- List of all the parts and pcs included:
 - Rollator x 1pc
 - Shopping bag x 1pc
 - Cane holder x 1pc
 - User manual x 1pc
 - Flyer x 1pc

LIMITED LIFETIME WARRANTY

Your Medic Therapeutics Multifunctional Adjusting Walker is backed by a limited lifetime manufacturer's warranty. Medic Therapeutics will repair or replace your device at any time should it fail due to a defect in material or workmanship, subject to the certain limitations.

This limited warranty does not cover any damage that results from unauthorized or improper use, service, or repair. Further, it does not cover damage caused by accident, impact, negligence, or normal wear and tear. Should you discover your Medic Therapeutics Multifunctional Adjusting Walker is not functioning properly, please send your device to our repair center for evaluation. If your product cannot be repaired or serviced we will reserve the right to change it for a similar or newer model.

Please note that a flat rate of \$35.00 will be charged to cover a service evaluation fee and return shipping of your device. All warranty claims must be accompanied by a copy of your proof of purchase from an authorized retailer. Please send your device, proof of purchase, and a check or money order in the amount of \$35.00 made out to Medic Therapeutics to:

Medic Therapeutics Service Center
3069 Taft Street
Hollywood, FL 33009